# Proclinic Expert Porcelain Etch

---

9% HYDROFLUORIC ACID

- de Gebrauchsanweisung
- en Instructios for use
- es Instrucciones de uso
- fr Mode d'emploi
- it Istruzioni per l'uso
- pt Instruções de uso
- nl Gebruiksaanwijzing
- pl Instrukcja postępowania
- da Brugsvejledning
- ro Instrucțiuni de utilizare
- sv Bruksanvisning
- no Bruksanvisning
- cs Návod k použití
- sk Návod na použitie
- sl Navodila za uporabo
- et Kasutusjuhised
- hu Használati utasítás
- hr Upute za uporabu
- bg Инструкции за употреба
- el Οδηγίες χρήσης
- fi Käyttöohjeet

**proclinic**  
EXPERT

### **PATIENTENPOPULATION**

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## **en** Instructios for use / HYDROFLUORIC ACID FOR PORCELAIN ETCHING

### **PATIENT POPULATION**

Any human undergoing dental treatment related to the procedures listed below under “Indications”. The definitive patient group is diagnosed on a case-by-case basis by a licensed dental professional and there are no exclusions for specific patient groups.

Cementation of ceramics in general, for example for:

- Preparation of porcelain crowns, veneers and inlays before definitive cementation.
- Repair of ceramic pieces or fractured metal-ceramic.

### **PROPERTIES**

Proclinic Expert Porcelain Etch is a viscous hydrofluoric acid for etching any type of ceramic which creates micropores in the treated surface, thus achieving a strong bond to composite. When applied to the etched ceramic, Proclinic Expert Silane improves the restoration’s bonding strength and longevity.

### **INSTRUCTIONS FOR USE**

1. Isolate the working field intra-orally with rubber dam.
2. For fractured ceramic repairs, roughen the surface of the ceramic to be etched using a medium grain bur. Remove the shiny surface and create enough space for the composite. Rinse and dry the ceramic surface.
3. Apply Proclinic Expert Porcelain Etch to the ceramic with a brush and leave to act for 60 seconds.
4. Rinse the ceramic with copious amounts of water to remove the acid and dry. The ceramic should have a matt, frosted appearance. If this is not the case, repeat the etching process until this appearance is achieved.
5. Apply Proclinic Expert Silane onto the etched ceramic and dry well with a stream of oil-free and water-free air.

### **CONTRAINDICATIONS**

- Not to be used on Tooth Surfaces
- Avoid contact with Oral Mucosa
- The use of the medical device is contraindicated if the patient is known to be allergic or sensitive to any of its ingredients.

### **PRECAUTION!**

Use with caution. Avoid contact with mucosa, eyes and skin. In case of accidental contact, wash immediately with copious amounts of water and consult a specialist. It is recommended to isolate the working field intra-orally with a rubber dam.

Any serious incident that occurs in relation to this device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established. You may also contact our EU Authorised Representative: Obelis S.A. (contact details below).

### **COMPOSITION**

Proclinic Expert Porcelain essential ingredients - Etch % wt.: Water <90% / Hydrofluoric acid <10% / Thickening agent <5%

### **STORAGE AND DISPOSAL**

- Do not store above 24°C (75°F).
- Keep product in original packaging.
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.

- If refrigerated, allow the syringe to reach room temperature.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.
- Empty product can be safely disposed after use in an intended regular waste container. The product does not require special disposal instructions or precautions and is harmless to the environment.

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**OBELIS S.A.**
**OBELIS SWISS GmbH**
**B.J.M. Laboratories Ltd.**
**PROCLINIC S.A.U.**