Ceramic Primer

ONE COMPONENT

- de Gebrauchsanweisung
- en Instructios for use
- es Instrucciones de uso
- fr Mode d'emploi
- it Istruzioni per l'uso
- pt Instruções de uso
- nl Gebruiksaanwijzing
- pl Instrukcja postępowania
- da Brugsvejledning
- ro Instrucțiuni de utilizare
- sv Bruksanvisning
- no Bruksanvisning
- cs Návod k použití
- sk Návod na použitie
- sl Navodila za uporabo
- et Kasutusjuhised
- hu Használati utasítás
- hr Upute za uporabu
- bg Инструкции за употреба
- el Οδηγίες χρήσης
- fi Käyttöohjeet

proclinic
EXPERT

PATIENTENPOPULATION

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PATIENT POPULATION

Any human undergoing dental treatment related to the procedures listed below under “Indications”. The definitive patient group is diagnosed on a case-by-case basis by a licensed dental professional and there are no exclusions for specific patient groups.

INDICATIONS

Proclinic Expert Ceramic Primer is a surface conditioner for ceramic materials such as porcelain and zirconia restorations.

It is used for

- Preparation of ceramic crowns, veneers and inlays prior to cementation
- Preparation of fractured ceramic crowns for repair with resin materials

PROPERTIES

Proclinic Expert Ceramic Primer is a one bottle primer material, for surface preparation of ceramic crowns, prior to bonding with resin cements. This primer improves cementation of zirconia crowns and does not require additional conditioning such as sandblasting. Its low viscosity and excellent wetting properties promote ideal surface coverage.

STORAGE CONDITIONS

The shelf life is 2 years when stored below 24°C (75°F). Do not freeze.

PREPARING CERAMIC SURFACE FOR BONDING

1. Clean ceramic surface with alcohol and dry well with oil-free air.
2. Open Proclinic Expert Ceramic Primer and dispense 1-2 drops into mixing well, replace bottle cap immediately.
3. Use applicator to apply generous amount of primer onto ceramic surfaces.
4. Wait 1 minute.
5. Dry gently for 20 seconds with oil-free air.
6. Continue with cementation.

CONTRAINDICATIONS

The use of the medical device is contraindicated if the patient is known to be allergic or sensitive to any of its ingredients.

ESSENTIAL INGREDIENTS

Ethanol, water, organic acid, silane coupling agent, 4-META

CAUTIONS

Avoid contact with skin, oral soft tissues and eyes.

If accidental contact occurs, rinse with copious amount of water and seek medical attention immediately. Intraorally it is mandatory to isolate the work area with a rubber dam.

Any serious incident that occurs in relation to this device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established. You may also contact our EU Authorised Representative: Obelis S.A. (contact details below).

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OBELIS S.A.
OBELIS SWISS GmbH
B.J.M. Laboratories Ltd.
PROCLINIC S.A.U.