



Product information

Oil spray - Universal

Properties

- excellent lubrication -> practice tests in cooperation with several handpiece and turbine manufacturers with excellent testing results
- extends the economic life-time of your instruments
- environment-friendly and High efficiency
- stable to sterilisation, no influence of sterilisation process
- approved as a medical device

Product type

The Universal Oil Spray is used for cleaning and lubricating turbines, tool holders, angle pieces and air motors.

Special lubrication formula for the maintenance especially of dental turbines, handpieces.

General references:

Today's high-precision systems are very sophisticated and demand for a premium maintenance. A professional cleansing and lubricating maintenance are the key and the main condition for dental drill instruments to function correctly and for a long economic life-time.

Universal Oil Spray supports your daily work being environment-friendly at the same time. The spray provides premium maintenance for your precise high quality instruments. It was especially designed for the extreme conditions in dental treatment units. Its additives are very well harmonised making it a lubricant for a broad variety of drives – fast as well as slow drives.

Dental drives may reach speeds of up to 350.000 rpm. Universal Oil Spray provides for an excellent lubrication and internal cleaning. Thus, it provides for your instruments to function flawlessly.

Universal Oil Spray does not contain toxic ingredients. It is physiologically unobjectionable and environment-friendly. All ingredients comply with the specific international regulations and are conform to FDA standards.

Very good compatibility to materials, the product also supports other materials such as plastics, coatings, seals, glass etc..

Complementary products

- **RCS ENDO System** (rapid clean system for inside cleaning and disinfection of handpieces and turbines)
- disinfection wipes for outside cleaning of handpieces and turbines
- various oilspray adapters (e.g. INTRAMatic, Roto Quick, Multiflex and ISO interfaces)

Packaging sizes

- aluminium can oder tinmetal can from 200ml - 500ml
- special packagings like machine suitable valves etc.
- Bulk drums for self filling customers

label size/ printing area

- 500ml aerosol can: 207 x 175 mm (please aks for templates)

Universal Oil Spray is completely sterilisable, stable to hydrolysis and oxidation and shows a very best compatibility with disinfection products. A sterilisation effect without restrictions may be accounted for by Prof. Dr. Werner. (expert report upon request)

If used regularly, Universal Oil Spray helps your instruments needing to be repaired less often.

Application references:

Apply before sterilising or at least twice per day for approx. two seconds. Keep can upright and use the specific adapter for your system.

Safety references:



Gefahr

R 12 Extremely Flammable

S 9 Keep container in a well-ventilated place. S 33 Take precautionary measures against static discharges. S 51 Use only in well-ventilated areas. S 60 This material and its container must be disposed of as hazardous waste.

Pressurized container: protect from sunlight and do not expose to temperatures exceeding 50°C. Do not pierce or burn, even after use. Do not spray on a naked flame or any incandescent material. Keep away from sources of ignition – No smoking. Keep out of the reach of children.

Expertise:

Our products are scientifically examined and tested according to the latest methods. We will gladly send you copies of our expertise upon request.