




A use film is available here: www.youtube.com

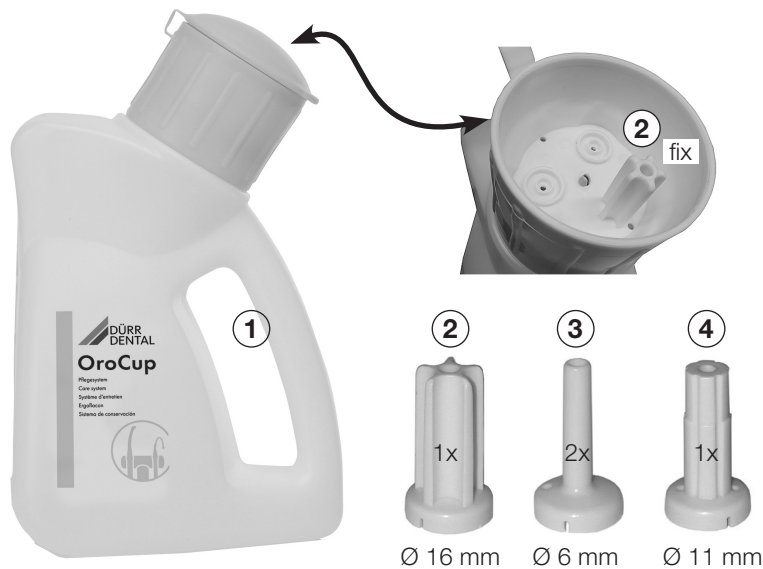

Scope of delivery

- ① OroCup, Order number 0780-350-00
- ② Ø 16 mm cartridge (for large suction hoses), 2x (1x hose, 1x fix)
- ③ Ø 6 mm cartridge (for saliva ejector), 2x
- ④ Ø 11 mm cartridge (other), 1x

Disposable materials

- Orotol® plus Disinfection of suction systems.....CDS110P6150
Liquid concentrate
- Orotol® ultra Disinfection of suction systems.....CDS120A6750
Powder concentrate
- MD 555 cleaner Special detergent for suction systems...CCS555C6150
foam-free concentrate for dental suction units and waste water connections



Orotol® plus



Orotol® ultra



MD 555 cleaner

Product description

The OroCup care system is a closed dosing system for the simple preparation and suction of disinfectants and special cleaners. Use OroCup to clean and disinfect the suction system and its components and the spittoon. 1 L of ready-to-use solution is required for 1 treatment unit. OroCup is suitable for conventional suction hoses of various diameters. The OroCup cover contains 1 fixed cartridge for Ø 16 mm. Two further cartridges can be selected and used depending on the diameter. Depending on requirements, you can connect 1 - 3 suction hoses at the same time; non-used connections have no influence on the function.

OPERATION

3. Selecting and inserting cartridges

- Select and insert the cartridge in accordance with the diameter of the suction hose (e.g. Ø 16 mm for larger suction hoses, Ø 6 mm for saliva ejectors), see example. Unused connections do not exert an influence on the function.


4. Cleaning and disinfecting the suction unit

Use at the end of all treatment, or twice a day if contamination levels are higher (e.g. afternoons and evening or as required).

- Wear personal protective equipment.


4.1 Pre-cleaning with water

- Suck out 2 l of water.

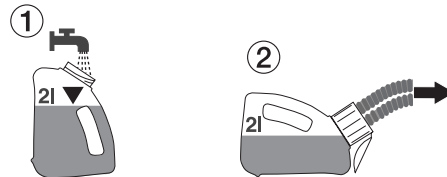


4.2 Place in the OroCup

- Insert 1 or 2 litres of ready-to-use solution as required. Comply with manufacturer information.

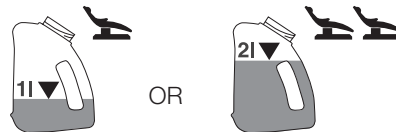

- Close the OroCup cover and shake the OroCup.

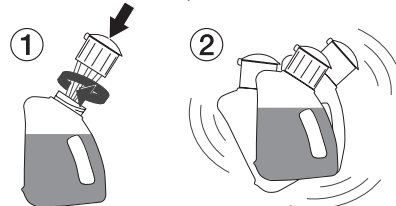


4.3 Position the OroCup and engage suction

- Open the cover of the OroCup.
- Place on the disinfected suction handpiece and remove the ready-to-use solution.

Up to 2 litres of ready-to-use solution can be poured into the OroCup.


Position the OroCup as follows to remove the ready-to-use solution:

- Place in an upright position when sucking out the 1. L (for the 1 treatment unit).
- Place in an horizontal position when sucking out the 2. L (for the 2 treatment unit).

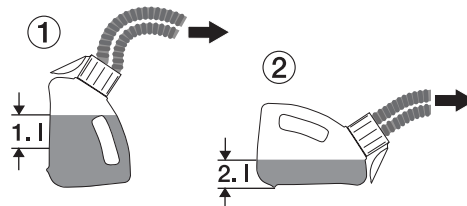

4.4 Cleaning and disinfecting the spittoon

- Use min. 250 ml ready-to-use solution.

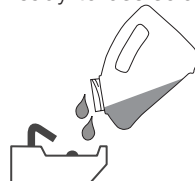

4.5 Final rinse

- Suck off 2 K water after the reaction time see 2.1 Pre-cleaning with water.

