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25 °C
77 °F

Made in
Germany
0483
07/2018

Ready-to-use one component temporary filling material, with CaF₂

Fig. 1

Fig. 2

Indications

Provisional temporary filling of dental cavities.

Application

Take the required quantity of **Detaferm®** and directly insert it into the cavity (wet/dry) with the filling instrument (Fig. 1); product dries after approx. 15 minutes under the effect of saliva; avoid the application of chewing pressure for approx. 1 hour. Do not return the material to the glass jar.

Removing the temporary

Detaferm® can be removed from the cavity by means of a probe or an excavator (Fig. 2).

Contra-indications

Do not apply directly on exposed pulp. Place suitable lining to protect the pulp, e.g. with a calcium hydroxide preparation.

Attention

- The application should be performed under aseptic conditions.
- After application of the cement, pulpal irritations may occur. Therefore a lining should be applied to the cavity when working in close proximity to the pulp, before applying **Detaferm®**.
- Only for intended use by qualified personnel.
- Contains calcium fluoride.
- Please follow the instructions on the safety data sheet!

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MD Dispositivo médico

Temporary sealing material for tooth cavities

Fig. 1

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Application

Take the required quantity of **Fermin** and directly insert it into the cavity (wet/dry) with the filling instrument (Fig. 1); product dries after approx. 15 minutes under the effect of saliva; avoid the application of chewing pressure for approx. 1 hour. Do not return the material to the glass jar.

Fig. 2

Removing the temporary

Fermin can be removed from the cavity by means of a probe or an excavator (Fig 2).

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Attention

- The application should be performed under aseptic conditions.
- After application of the cement, pulpal irritations may occur. Therefore a lining should be applied to the cavity when working in close proximity to the pulp, before applying **Fermin**.
- Only for intended use by qualified personnel.
- Please follow the instructions of the safety data sheet!

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