MICRO-MEGA®

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Sterile : *Sold in boxes of 10 blisters packs of 5 identical instruments.*
Sold in blister of 5 identical instruments
Sold in assortment of 2xTS1 + 2xTS2

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Non-sterile : *Sold in boxes of 10 plates of 6 identical instruments.*
Sold in plates of 6 identical instruments
Sold in assortment of 2xTS1 + 2xTS2

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Internet : www.micro-mega.com

Geometric specifications :

The 2Shape instruments have a .04 and .06 taper.

They have a profile with 3 helixes with an asymmetrical cross section.

The 2Shape instruments are available in « Classics » (handle Ø 2.35) for use on endodontic contra-angles.

The instruments comply with the applicable paragraphs of the standard EN ISO 3630-1.

The handle « Classics » complies with the applicable paragraphs of the standard EN ISO 1797-1.

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TS2 ; F35 = 2,5 N.cm

Specifications for use :

The 2Shape instruments are used with continuous rotary endodontic contra-angles.

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Maximum recommended number of uses under normal conditions :

5, if the instrument is not visually damaged

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Re-use of disposable instruments :

In case of re-use of a disposable instrument, the risks are a bad cleaning during reprocessing or a risk of breakage of the instrument during re-use.

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Pre-disinfection, cleaning and sterilization:

For hygienic and sanitary safety reasons, all root canal instruments marketed by MICRO-MEGA® must be pre-disinfected, cleaned and sterilized before each use on a patient (excepting sterile instruments).

This procedure is recommended for all instruments :

- before the first use for non sterile instruments
- between each use on patient for re-usable instruments

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Pre-disinfection: *Before the immersion of the instruments in the disinfectant bath sort them by type and material category in order to avoid electrolysis and corrosion. Comply with the instructions and concentrations recommended by the pre-disinfectant manufacturer (an excessive concentration can cause chemical damage to the materials). Respect the immersion time (do not leave the instruments in the disinfectant bath overnight). Only use disinfectant products recommended for the disinfection of dental root canal instruments (do not use products containing phenol, hydrogen peroxide, soda or sodium hypochlorite or extremely aggressive special disinfectants for burs).*

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Note: MICRO-MEGA® has tested the most common disinfectant products on the market. In view of the large number of available products, it is recommended to pre-test a product on a sample or to contact the manufacturer of the used product.

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Cleaning: *Brush the instruments for dentinal debris elimination. Do not clean the instruments loosely in an ultrasound device. Do not use soda-based solutions, solutions containing mercury salt or alkaline and acid solutions.*

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Internet : www.micro-mega.com

Rinsing: Rinse the instruments abundantly using demineralized or distilled water to avoid corrosion and alterations during sterilization. Dry with dry air. Discard deformed instruments (broken, twisted, despiralized instruments or instruments showing marks conducive to breakage).

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Wrapping : Put the instruments in a sterilization bag compliant with EN ISO 11607 standard to be used in a steam sterilizer up to 134°C and 2.2 bar during 18 minutes. Follow the instruction of the sterilization bag's manufacturer.

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Sterilization: Sterilize the instruments wrapped in a sterilization bag. Use an autoclave (saturated steam sterilizer): at 134°C and 2.2 bar during 18 minutes.

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Storage : Check the integrity of the sterilization bag before using the instruments

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Normal function of the Shape :

Instruments enable to make easy and quick root canal treatment.

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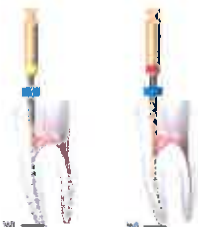
 : Code du lot / *Lot number*

Dispositif médical de classe IIA stérile (Directive 93/42)

Medical device class IIA sterile (93/42 Directive)

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VALIDATION :

FUNCTION : Regulatory Affairs Manager Micro-Mega

NAME : Philippe CONCHE

DATE (YYYY-MM-DD) : 2017-06-01

SIGNATURE :

FUNCTION : Technical Director Micro-Mega

NAME : Dr Laurent BATAILLARD

DATE (YYYY-MM-DD): 2017-06-01

SIGNATURE :

po D. ROUSSEAU Resp BE