BM 45

D	Blutdruckmessgerät	
	Gebrauchsanweisung	2-15
GB	Blood pressure monitor	
	Instructions for use	16-27
F	Tensiomètre	
	Mode d'emploi	28-40
E	Tensiómetro	
	Manual de instrucciones	41-53
I	Misuratore di pressione	
	Istruzioni per l'uso	54-66
TR	Bilgisayarli tansiyon ölçer	
	Kullanım kılavuzu	67-78
RUS	Прибор для измерения кровяного давления в плечевой артерии	
	Инструкция по применению	79-92
PL	Ciśnieniomierz	
	Instrukcja obsługi	93-105
	Electromagnetic Compatibility Information	106-108

Inhalt

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Contents

1. Getting to know your instrument.....	16	7. Displaying and deleting measurements	24
2. Important information.....	17	8. Cleaning and storing the instrument	25
3. Device description.....	19	9. Rectifying faults.....	26
4. Preparing the measurement	20	10. Specifications.....	26
5. Measuring blood pressure.....	22	11. Mains part	27
6. Evaluating results	23		

Dear Customer,

Thank you for choosing one of our products. Our brand stands for well-engineered, quality made products; driven by innovation and safety. Our product range encompasses a wealth of products for health and well-being in the heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air sectors.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,
Your Beurer Team

1. Getting to know your instrument

Check that the packaging of the Beurer BM 45 blood pressure monitor has not been tampered with and make sure that all the required contents are present. Before use, ensure that there

is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

The upper arm blood pressure monitor is used for non-invasive measurement and monitoring of adults' arterial blood pressure. You can use it to measure your blood pressure quickly and easily, storing the results and displaying the progression of readings together with the average.

A warning is issued for anyone suffering from cardiac arrhythmia.

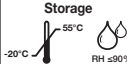
The values determined are classified and graphically evaluated according to WHO guidelines.

Keep these instructions carefully for further use and also let other users have access to them.

2. Important information

Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

	Caution
	Note Note on important information
	Follow instructions for use
	Type BF applied part
	Direct current
	Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).
	Manufacturer
	Permissible transport and storage temperature. Permissible transport and storage humidity.

	Permissible operating temperature and humidity
	Keep dry
SN	Serial number
 0483	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products.

Advice on use

- In order to ensure comparable values, always measure your blood pressure at the same time of day.
- Before every measurement, relax for about five minutes.
- If you want to perform several measurements on the same person, wait five minutes between each measurement.
- Do not take a measurement within 30 minutes after eating, drinking, smoking or exercising.
- Repeat the measurement if you are unsure of the measured value.
- The measurements taken by you are for your information only – they are not a substitute for a medical examination! Discuss the measurements with your doctor, and never base any medical decisions on them (e.g. medicines and their administration)!
- Do not use the blood pressure monitor on newborns or patients with preeclampsia. We recommend consulting a doctor before using the blood pressure monitor during pregnancy.

- Cardiovascular diseases may lead to incorrect measurements or have a detrimental effect on measurement accuracy. The same also applies to very low blood pressure, diabetes, circulatory disorders and arrhythmias as well as chills or shaking.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Only use the device on people who have the specified upper arm measurement for the device.
- Please note that when inflating, the functions of the limb in question may be impaired.
- During the blood pressure measurement, blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions, remove the cuff from the arm.
- Avoid any mechanical restriction, compression or bending of the cuff line.
- Do not allow sustained pressure in the cuff or frequent measurements. The resulting restriction of the blood flow may cause injury.
- Ensure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intra-vascular access or therapy, or an arteriovenous (AV) shunt.
- Do not use the cuff on people who have undergone a mastectomy.
- Do not place the cuff over wounds as this may cause further injury.
- You can either use the blood pressure monitor with batteries or with a mains part. Please note that data transfer and data storage is only possible when your blood pressure monitor is supplied with power. As soon as the batteries are empty or

the mains part is disconnected from the power supply, the blood pressure monitor loses the date and time.

- To conserve the batteries, the monitor switches off automatically if no buttons are pressed for 3 minutes.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

Storage and Care

- The blood pressure monitor is made up of precision electronic components. Accuracy of readings and the instrument's service life depend on careful handling.
 - You should protect the device from impact, moisture, dirt, major temperature fluctuations and direct exposure to the sun's rays.
 - Never drop the device.
 - Do not use near strong electromagnetic fields, i.e. keep it away from any radio systems and mobile phones.
 - Only ever use the cuffs provided with the monitor or original replacement cuffs. Otherwise erroneous results will be recorded.
- Do not press any buttons until the cuff is in position.
- If the instrument is not used for any length of time, we recommend removing the batteries.

Advice on batteries

- If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.

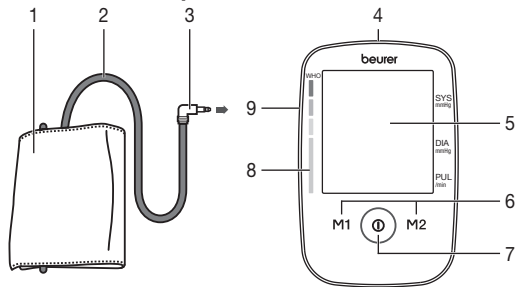
- Choking hazard! Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Risk of explosion! Never throw batteries into a fire.
- If the device is not to be used for a long period, take the batteries out of the battery compartment.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.

i Repair and disposal

- Batteries do not belong in domestic refuse. Used batteries should be disposed of at the collection points provided.
- Never open the instrument. If these instructions are not heeded, the warranty will be null and void.
- Never attempt to repair the instrument or adjust it yourself. We can no longer guarantee perfect functioning if you do.
- Repairs may only be performed by Customer Service or authorized dealers. However, always check the batteries and replace them if necessary prior to making any complaint.
- For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.


3. Device description

1. Cuff
2. Cuff line
3. Cuff connector
4. Connection for mains part
5. Display
6. Memory buttons **M1/M2**
7. Start/stop button **ⓘ**
8. WHO scale
9. Connection for cuff connector (left-hand side)

Information on the display:

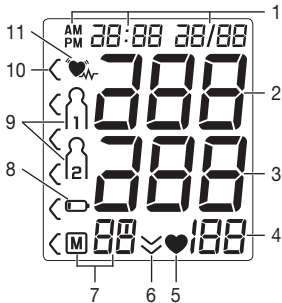
1. Date/time
2. Systolic pressure
3. Diastolic pressure
4. Calculated pulse value
5. Pulse symbol 
6. Release air (arrow)
7. Number of memory space/memory display average value (M), morning (M^m), evening (M^e)
8. Battery replacement symbol 
9. User memory  
10. WHO classification 
11. Cardiac arrhythmia symbol 

4. Preparing the measurement

Inserting the batteries

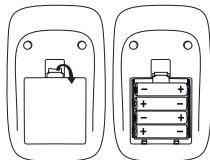- Remove the battery compartment lid on the rear of the device.
- Insert four 1.5 V AA (alkaline type LR 06) batteries. Make sure that the batteries are inserted the correct way round. Do not use rechargeable batteries.
- Close the battery compartment lid again carefully.

4 x 1,5V AA (LR06)

If the battery replacement symbol  is permanently displayed, you can no longer perform any measurements and must replace all batteries. Once the batteries have been removed from the device, the date and time must be set again. Any saved measurements are retained.

The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

-  The codes below are printed on batteries containing harmful substances:
- PB = Battery contains lead,
 - Cd = Battery contains cadmium,
 - Hg = Battery contains mercury.

Setting the hour format, date and time

This menu allows you to set the following functions, one after another.

Hour format

Date

Time

It is essential to set the date and time. Otherwise, you will not be able to save your measured values correctly with a date and time and access them again later.

i If you press and hold the **M1** or **M2** memory button, you can set the values more quickly.

Hour format

- Press and hold the Start/stop button **ⓘ** for 5 seconds.
- Choose the desired hour format with the **M1/M2** memory buttons and confirm with the Start/stop button **ⓘ**.

Date

The year flashes on the display.

- Choose the desired year with the **M1/M2** memory buttons and confirm with the Start/stop button **ⓘ**.

The month flashes on the display.

- Choose the desired month with the **M1/M2** memory buttons and confirm with the Start/stop button **ⓘ**.

The day flashes on the display.

- Choose the desired day with the **M1/M2** memory buttons and confirm with the Start/stop button **ⓘ**.

i If the hour format is set as 12h, the day/month display sequence is reversed.

Time

The hour flashes on the display.

- Choose the desired hour with the **M1/M2** memory buttons and confirm with the Start/stop button **ⓘ**.

The minute flashes on the display.

- Choose the desired minute with the **M1/M2** memory buttons and confirm with the Start/stop button **ⓘ**.

Operation with the mains part

You can also operate this device with a mains part.

When doing so, there must not be any batteries in the battery compartment. The mains part can be obtained from specialist retailers or from the service address using order number 071.60.

- To prevent possible damage to the device, the blood pressure monitor must only be used with the mains part described here.
- Insert the mains part into the connection provided for this purpose on the right-hand side of the blood pressure monitor. The mains part must only be connected to the mains voltage that is specified on the type plate.
- Then insert the mains plug of the mains part into the mains socket.
- After using the blood pressure monitor, unplug the mains part from the mains socket first and then disconnect it from the blood pressure monitor. As soon as you unplug the mains part, the blood pressure monitor loses the date and time setting but the saved measurements are retained.

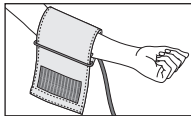
5. Measuring blood pressure

Please ensure the device is at room temperature before measuring.


The measurement can be performed on the left or right arm.

5.1 Positioning cuff


Fit the cuff round your bare left upper arm. Blood circulation in the arm should not be restricted by tight clothing or other objects.

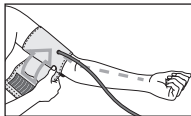
The cuff should be placed on the upper arm so that the lower edge is 2 to 3 cm above the bend of the elbow and above the artery. The tube should be in line with the centre of the palm.

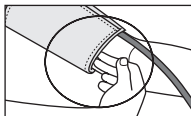Now place the free end of the cuff snugly, but not too tightly, around the arm, and fix it with the Velcro fastener. The cuff should be fitted tight enough to allow just two fingers to fit beneath the cuff.

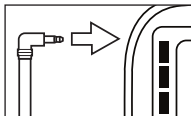Insert the cuff tubing into the socket for the cuff attachment.



- i** If the measurement is performed on the right upper arm, the line should be located on the inside of your elbow. Ensure that your arm is not pressing on the line.

Blood pressure may vary between the right and left arm, which may mean that the measured blood pressure values are different. Always perform the measurement on the same arm.

If the values between the two arms are significantly different, please consult your doctor to determine which arm should be used for the measurement.

Important: The unit may only be operated with the original cuff. The cuff is suitable for an arm circumference of 22 to 36 cm. A larger cuff for upper-arm circumferences of 35 to 44 cm can be obtained from specialist retailers or from the service address using order number 163.387.

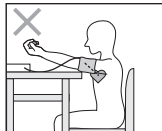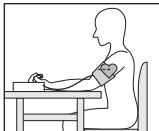5.2 Correct posture


- Rest for approx. 5 minutes before each measurement. Otherwise there may be divergences.
- You can perform the measurement either sitting or lying down. Always make sure that the cuff is on a level with your heart.
- To carry out a blood pressure measurement, make sure you are sitting comfortably with your arms and back leaning on something. Do not cross your legs. Place your feet flat on the ground.

- In order not to distort the result, it is important to keep still during the measurement and not talk.

Performing the blood pressure measurement

As described above, attach the cuff and adopt the posture in which you want to perform the measurement.

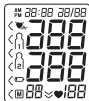- To start the blood pressure monitor, press the Start/stop button **ⓘ**. All displays will illuminate briefly.


The blood pressure monitor will begin the measurement automatically after 3 seconds.

The cuff automatically inflates.

- **ⓘ** Measuring can be cancelled at any time by pressing the Start/stop button **ⓘ**.

The cuff's air pressure is slowly released. If you already recognise a tendency for high blood pressure, you should reinflate the cuff and increase the cuff's pressure again. As soon as a pulse is found, the pulse symbol **♥** is displayed.

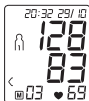- Systolic pressure, diastolic pressure and pulse measurements are displayed.

- **Er_** appears if the measurement could not be performed properly. Observe the chapter on error messages/troubleshooting in these instructions for use and repeat the measurement.

- Now select the desired user memory by pressing the **M1** or **M2** memory buttons. If you do not select a user memory, the measurement is stored in the most recently used user memory. The relevant symbol **M1** or **M2** appears on the display.
- Using the Start/stop button **ⓘ**, switch off the blood pressure monitor. The measurement is then stored in the selected user memory. If you forget to turn off the device, it will switch off automatically after approx. 3 minutes. In this case too, the value is stored in the selected or most recently used user memory.
- Wait at least 5 minutes before taking another measurement!

6. Evaluating results

Cardiac arrhythmia:

This instrument can identify possible cardiac arrhythmia disorders during measurement and if necessary indicates the measurement with the flashing icon **♥**.

This may be an indicator for arrhythmia. Arrhythmia is a condition where the heart rhythm is abnormal as a result of defects in the bioelectrical system controlling the heart beat. The symptoms (omitted or premature heart beats, slow or excessively fast heart rate) may be caused, among other things, by heart disease, age, physical predisposition, excessive use of stimulants, stress or lack of sleep. Arrhythmia can only be ascertained through examination by your doctor.

Repeat the measurement if the flashing icon  is displayed after the measurement. Please note that you should rest for 5 minutes between measurements and not talk or move during the measurement. If the icon  appears often, please contact your doctor. Any self-diagnosis and treatment based on the test results may be dangerous. It is vital to follow your doctor's instructions.

WHO classification:

In accordance with the guidelines/definitions of the World Health Organization and the latest findings, the measurements can be classified and assessed according to the following table. However, these standard values serve only as a general guideline, as the individual blood pressure varies in different people and different age groups etc.

Range of blood pressure values	Systolic (in mmHg)	Diastolic (in mmHg)	Measure
Grade 3: Severe hypertension	≥ 180	≥ 110	Seek medical advice
Grade 2: Moderate hypertension	160–179	100–109	Seek medical advice
Grade 1: Mild hypertension	140–159	90–99	Have it checked regularly by doctor
High-normal	130–139	85–89	Have it checked regularly by doctor
Normal	120–129	80–84	Check it yourself
Optimal	<120	<80	Check it yourself

Source: WHO, 1999

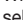
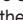
It is important to consult your doctor regularly for advice. Your doctor will tell you your individual values for normal blood pressure as well as the value above which your blood pressure is classified as dangerous.

The classification on the display and the scale on the unit show which category the recorded blood pressure values fall into. If the values of systole and diastole fall into two different WHO categories (e.g. systole in the 'High normal' category and diastole in the 'Normal' category), the graphical WHO classification on the unit always shows the higher category; for the example given this would be 'High normal'.

7. Displaying and deleting measurements

User memory

The results of every successful measurement are stored together with the date and time. If there are more than 60 measurements, the oldest measurements are lost.

- To access memory recall mode, the blood pressure monitor must first be started. To do this press the Start/stop button .
- Within 3 seconds of the full-screen display appearing, select the desired user memory ( ) with the **M1** or **M2** memory button.

- To view the measurements for user memory M_1 , press the **M1** memory button.
- To view the measurements for user memory M_2 , press the **M2** memory button.

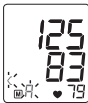
Your last measurement will appear on the display.

- Press the relevant memory button (**M1** or **M2**).
- ⓘ If you have selected user memory M_1 , the **M1** memory button must be pressed.
If you have selected user memory M_2 , the **M2** memory button must be pressed.

R flashes on the display.

The average value of all saved measured values in this user memory is displayed.

- Press the relevant memory button (**M1** or **M2**).

AM flashes on the display.

The average value of the morning measurements for the last 7 days is displayed (morning: 5.00 a.m. – 9.00 a.m.).

- Press the relevant memory button (**M1** or **M2**).

PM flashes on the display.

The average value of the evening measurements for the last 7 days is displayed (evening: 6.00 p.m. – 8.00 p.m.).

- When the relevant memory button (**M1** or **M2**) is pressed again, the last individual measurement is displayed (in this example, measurement 03).

- When the relevant memory button (**M1** or **M2**) is pressed again, you can view your individual measurements.
- To switch the device off again, press the Start/stop button **ⓘ** or wait 30 sec.
- ⓘ You can exit the menu at any time by pressing the Start/stop button **ⓘ**.

- To clear the memory of the relevant user memory, you must first select a user memory.
- Start individual measurement access.
- Press and hold the **M1** & **M2** memory buttons for 5 seconds.

All the values in the current user memory are deleted.

8. Cleaning and storing the instrument

- Clean your device and cuff carefully only with a slightly moistened cloth.
- The frequency of cleaning depends on the contamination level of the device. Clean the blood pressure device and cuff as soon as any dirt appears on the device.

- Do not use detergents or solvents.
- On no account must you immerse the computer in water, otherwise liquid can enter it and cause damage.

9. Rectifying faults

In case of faults, the E_r _ message appears in the display.

Error messages may appear if:

- systolic or diastolic pressure could not be measured ($E_r 1$ or $E_r 2$ appears on the display),
- systolic or diastolic pressure was outside the measurement range (**Hi** or **Lo** appears on the display),
- the cuff is fastened too tightly or loosely ($E_r 3$ or $E_r 4$ appears on the display),
- the pump pressure is higher than 300 mmHg ($E_r 5$ appears on the display),
- pumping up takes longer than 160 seconds ($E_r 6$ appears on the display),
- there is a system or device error ($E_r A$, $E_r B$, $E_r 7$ or $E_r 8$ appears on the display),
- the batteries are almost empty ,
- the data could not be sent to the PC ($P_C E_r$ appears in the display).

In such cases, repeat the measurement. Ensure that the cuff line is correctly attached and that you do not move or speak. If necessary, reinsert or replace the batteries.

10. Specifications

Model no.	BM 45
Type	M1502
Measurement method	Oscillometric, non-invasive blood pressure measurement on the upper arm
Measurement range	Cuff pressure 0–300 mmHg, systolic 60–260 mmHg, diastolic 40–199 mmHg, Pulse 30–180 beats/minute
Display accuracy	Systolic ± 3 mmHg, diastolic ± 3 mmHg, pulse $\pm 5\%$ of the value shown
Measurement inaccuracy	Max. permissible standard deviation according to clinical testing: systolic 8 mmHg/diastolic 8 mmHg
Memory	2 x 60 memory spaces
Dimensions	L 165 mm x W 107 mm x H 50 mm
Weight	Approx. 284 g (without batteries)
Cuff size	22 to 36 cm
Permissible operating conditions	+10 °C to +40 °C, $\leq 90\%$ relative air humidity (non-condensing)
Permissible storage and transport conditions	-20 °C to +55 °C, $\leq 90\%$ relative air humidity, 800–1050 hPa ambient pressure
Power supply	4 x 1,5V  AA batteries
Battery life	For approx. 250 measurements, depending on the blood pressure level and/or pump pressure

Accessories	Instruction for use, 4 x 1.5V AA batteries, storage pouch
Classification	Internal supply, IPX0, no AP or APG, continuous operation, application part type BF (cuff), CLASS II (when using a mains part)

Technical information is subject to change without notification to allow for updates.

- This unit is in line with European Standard EN 60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this unit. More details can be requested from the stated Customer Service address or found at the end of the instructions for use.
- This device is in line with the EU Medical Devices Directive 93/42/EC, the “Medizinproduktegesetz” (German Medical Devices Act) and the standards EN 1060-1 (non-invasive sphygmomanometers, Part 1: General requirements), EN 1060-3 (non-invasive sphygmomanometers, Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and IEC 80601-2-30 (Medical electrical equipment – Part 2–30: Particular requirements for the safety and essential performance of automated non-invasive blood pressure monitors).
- The accuracy of this blood pressure monitor has been carefully checked and developed with regard to a long useful life. If using the device for commercial medical purposes, it must be regularly tested for accuracy by appropriate means.

Precise instructions for checking accuracy may be requested from the service address.

11. Mains part

Model no.	FW 7575M/EU/6/06
Input	100–240V, 50–60 Hz
Output	6V DC, 600 mA, only in connection with beurer blood pressure monitor.
Supplier	Friwo Gerätebau GmbH
Protection	This device is double insulated and protected against short circuit and overload by a primary thermal fuse. Make sure to take the batteries out of the compartment before using the mains part.
	Polarity of the the DC voltage connection
	Double insulated/equipment class 2
Enclosures and Protective Covers	Equipment enclosed to protect against contact with live parts, and with parts which can become live (finger, pin, hook test). The operator shall not contact the patient and the output plug of AC mains part simultaneously.

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Electromagnetic Compatibility Information

Guidance and manufacture's declaration – electromagnetic emissions – for all EQUIPMENT and SYSTEMS

Guidance and manufacture's declaration – electromagnetic emission

The BM45 blood pressure monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the BM45 blood pressure monitor should assure that it is used in such an environment.

Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The BM45 blood pressure monitor uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	The BM45 blood pressure monitor is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.

Guidance and manufacture's declaration – electromagnetic immunity – for all EQUIPMENT and SYSTEMS

Guidance and manufacture's declaration – electromagnetic immunity

The BM45 blood pressure monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the BM45 blood pressure monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floor are covered with synthetic material, the relative humidity should be at least 30%.
Power frequency (50Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Guidance and manufacture's declaration – electromagnetic immunity – for EQUIPMENT and SYSTEMS that are not LIFE-SUPPORTING

Guidance and manufacture's declaration – electromagnetic immunity

The BM45 blood pressure monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the BM45 blood pressure monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 V _{rms} 150 kHz to 80 MHz	3 V _{rms}	<p>Portable and mobile RF communications equipment should be used no closer to any part of the BM45 blood pressure monitor, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance:</p> $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$ $d = \left[\frac{3.5}{E_1} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[\frac{7}{E_1} \right] \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	<p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of  equipment marked with the following symbol:</p>

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the BM45 blood pressure monitor is used exceeds the applicable RF compliance level above, the BM45 blood pressure monitor should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the BM45 blood pressure monitor.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the EQUIPMENT or SYSTEM – for EQUIPMENT or SYSTEM that are not LIFE-SUPPORTING

Recommended separation distances between portable and mobile RF communications equipment and the BM45 blood pressure monitor

The BM45 blood pressure monitor is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the BM45 blood pressure monitor can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BM45 blood pressure monitor as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = \left[\frac{3.5}{V_i} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3.5}{E_i} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{E_i} \right] \sqrt{P}$
0.01	0.117	0.117	0.234
0.1	0.370	0.370	0.740
1	1.170	1.170	2.340
10	3.700	3.700	7.400
100	11.7	11.7	23.4

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.