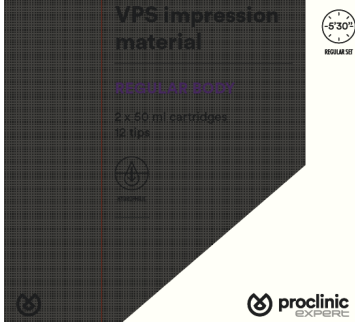
# CHL MEDICAL SOLUTIONS, SRL

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# CHL MEDICAL SOLUTIONS, SRL

TECHNICAL DATA SHEET	<b>Code:</b> TDS_004
LIGHT BODY REGULAR 2x50ml PROCLINIC	<b>Date:</b> 29/Abr/16
CHL	<b>Version:</b> 01

<b>PRODUCT</b>	<b>LIGHT BODY REGULAR 2x50ml PROCLINIC</b>	
<b>REFERENCE</b>	15153	
<b>COMPANY</b>	CHL MEDICAL SOLUTIONS SRL VIA FELICE CAVALLOTTI, 20122 MILANO - ITALY	
<b>CLASSIFICATION</b>	CLASS 1 MEDICAL DEVICE Directive 93/42/CEE	
<b>PRODUCT DESCRIPTION</b>	Addition silicone light bodied consistency elastomer precision material, based on vinyl polysiloxane, cristobalite, quartz, ethoxylated nonyl phenol.	
<b>SPECIFICATIONS</b>	Colour Base: Coral Colour Catalyst: Grey light Working time 2' Setting time 3'30'' Total working time 5'30''	
<b>PACKAGING</b>	2u.x 50 ml cartridges 12 tips	
<b>SHELF LIFE</b>	3 years	
<b>INTENDED OF USE</b>	Perform precision dental impressions.	
<b>SECURITY</b>	Do not use latex gloves. The use of latex gloves may disrupt the addition reaction. It is advised a careful hand washing before handling the material.	
<b>TRANSPORT AND STORAGE CONDITIONS</b>	Store between 5 ° C and 27 ° C. Store in a dry place.	

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# CHL MEDICAL SOLUTIONS, SRL

TECHNICAL DATA SHEET	<b>Code:</b> TDS_005
LIGHT BODY FAST SET 2x50ml/4x50ml PROC	<b>Date:</b> 29/Abr/16
CHL	<b>Version:</b> 01

<b>PRODUCT</b>	<b>LIGHT BODY FAST SET 2x50ml/4x50ml PROC</b>	
<b>REFERENCE</b>	15154  15155	
<b>COMPANY</b>	CHL MEDICAL SOLUTIONS SRL VIA FELICE CAVALLOTTI, 20122 MILANO - ITALY	
<b>CLASSIFICATION</b>	CLASS 1 MEDICAL DEVICE Directive 93/42/CEE	
<b>PRODUCT DESCRIPTION</b>	Light bodied consistency Silicone elastomer precision material, based on vinyl polysiloxane, cristobalite, quartz, ethoxylated nonyl phenol.	
<b>SPECIFICATIONS</b>	Colour Base: Orange Colour Catalyst: Grey light Working time 1' 30'' Setting time 2'30'' Total working time 4'	
<b>PACKAGING</b>	2u.x 50 ml and 4x50 ml cartridges 12 tips	
<b>SHELF LIFE</b>	3 years	
<b>INTENDED OF USE</b>	Perform precision dental impressions.	
<b>SECURITY</b>	Do not use latex gloves. The use of latex gloves may disrupt the addition reaction. It is advised a careful hand washing before handling the material.	
<b>TRANSPORT AND STORAGE CONDITIONS</b>	Store between 5 ° C and 27 ° C. Store in a dry place.	

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# CHL MEDICAL SOLUTIONS, SRL

TECHNICAL DATA SHEET	<b>Code:</b> TDS_006
ULTRA LIGHT BODY FAST SET PROCLINIC	<b>Date:</b> 29/Abr/16
CHL	<b>Version:</b> 01

<b>PRODUCT</b>	<b>ULTRA LIGHT BODY FAST SET PROCLINIC</b>	
<b>REFERENCE</b>	15156	
<b>COMPANY</b>	CHL MEDICAL SOLUTIONS SRL VIA FELICE CAVALLOTTI, 20122 MILANO - ITALY	
<b>CLASSIFICATION</b>	CLASS 1 MEDICAL DEVICE Directive 93/42/CEE	
<b>PRODUCT DESCRIPTION</b>	Light bodied consistency silicone elastomer precision material, based on vinyl polysiloxane, cristobalite, quartz, ethoxylated nonyl phenol.	
<b>SPECIFICATIONS</b>	Colour Base: Yellow Colour Catalyst: Grey light Working time 1' 30'' Setting time 2'30'' Total working time 4'	
<b>PACKAGING</b>	2u.x 50 ml cartridges 12 tips	
<b>SHELF LIFE</b>	3 years	
<b>INTENDED OF USE</b>	Perform precision dental impressions.	
<b>SECURITY</b>	Do not use latex gloves. The use of latex gloves may disrupt the addition reaction. It is advised a careful hand washing before handling the material.	
<b>TRANSPORT AND STORAGE CONDITIONS</b>	Store between 5 ° C and 27 ° C. Store in a dry place.	

# CHL MEDICAL SOLUTIONS, SRL

TECHNICAL DATA SHEET	<b>Code:</b> TDS_003
REGULAR BODY REGULAR 2x50ml PROCLINIC	<b>Date:</b> 29/Abr/16
CHL	<b>Version:</b> 01

<b>PRODUCT</b>	<b>REGULAR BODY REGULAR 2x50ml PROCLINIC</b>	
<b>REFERENCE</b>	15152	
<b>COMPANY</b>	CHL MEDICAL SOLUTIONS SRL VIA FELICE CAVALLOTTI, 20122 MILANO - ITALY	
<b>CLASSIFICATION</b>	CLASS 1 MEDICAL DEVICE Directive 93/42/CEE	
<b>PRODUCT DESCRIPTION</b>	Regular Silicone elastomer precision material, based on vinyl polysiloxane, cristobalite, quartz, ethoxylated nonyl phenol.	
<b>SPECIFICATIONS</b>	Colour Base: Purple Colour Catalyst: Grey light Working time 2' Setting time 3'30'' Total working time 5'30''	
<b>PACKAGING</b>	2u.x 50 ml cartridges 12 tips	
<b>SHELF LIFE</b>	3 years	
<b>INTENDED OF USE</b>	Perform precision dental impressions.	
<b>SECURITY</b>	Do not use latex gloves. The use of latex gloves may disrupt the addition reaction. It is advised a careful hand washing before handling the material.	
<b>TRANSPORT AND STORAGE CONDITIONS</b>	Store between 5 ° C and 27 ° C. Store in a dry place.	