Teeth whitening pen

16% CARBAMIDE PEROXIDE

Instructions for use

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TEETH WHITENING PEN

- 1.- Brush and floss the teeth prior to using the whitening pen.
- 2.- Press the top button a few times until the gel comes out.
- 3.- Place the gel on the surface of the teeth to be bleached until covering all of them.
- 4.- Avoid the contact of the lips with the product for 30 seconds.
- 5.- Leave the gel for 30 minutes.
- 6.- In case of tooth sensitivity or gum irritation, remove immediately the product and rinse with water.

Use 1-3 times daily until finishing the product (one pen lasts up to 2-3 weeks). Repeat the treatment again with another Proclinic Expert Whitening Pen after 2 months.

In case of accidental ingestion or poisoning, please contact the Medical Toxicology Information Service.

Warning: Exclusive distribution to odontologists. Exclusive dental use. Do not ingest. It is not recommended to receive long-term bleaching treatments with peroxide substances. It is not

recommended for regular consumers of tobacco or alcohol. Do not apply during pregnancy or to children. Its use is not recommended after dental restorations. Its use is not recommended if previous injuries of the gingival tissues exist. Avoid contact with the eyes. In case of contact with the eyes, rinse with plenty of water and consult a doctor.

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