Corewhite bleaching lamp

Instructions for use

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EN Instructions for use.

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INSTRUCTIONS FOR USE

EN

PROCLINIC EXPERT COREWHITE BLEACHING LAMP

INDEX

Symbols used in the manual and in the product	6
Safety warnings	6
Introduction.....	6
Technical description	7
Mechanical fixing	7
Electrical wiring	7
Use.....	7
Bleaching procedure	8
Maintenance.....	8
Use and storage conditions	9
Repairs.....	9
Warranty	9
Warning RAEE.....	9

SYMBOLS USED IN THE MANUAL AND IN THE PRODUCT:

	Type B applied part
	Alternating current
	Follow the instructions for use
	Ground
	Prohibition
	Warning
	Auxiliary power supply connector
	RAEE symbol
	ON/OFF

SAFETY WARNINGS:

Before using the operating light system, please read the instructions contained in this manual

and please pay attention to the following indications:

- dental bleaching light has to be used to illuminate patient dental arches for professional bleaching treatment and must be used only for this purpose by medical staff or by staff adequately trained and with an appropriate knowledge in the use of this kind of devices;
- dental operating light must be supplied in conformity to electric supply specifications exposed in this manual and with wiring in conformity to laws applied for medical local;
- do not stare into beam and wear the protection glasses during the treatment;
- do not realize maintenance or light cleaning during light turned on;
- do not introduce any objects in the ventilation holes that could realize a contact within voltage point;
- clean light surfaces according to the indications contained in this manual;
- do not cover the ventilation light holes; clean and control periodically holes to assure a correct ventilation;
- do not leave unguarded the light turned on for long period;
- spray or liquid vapour entry in the light envelope has to be avoid.

INTRODUCTION:

Thank you for purchasing the dental operating light "COREWHITE".

The bleaching light COREWHITE is a device which has to be used to illuminate patient dental arches for professional bleaching treatment of vital teeth. The emitted light acts to an hydrogen peroxide based gel in order to increase the bleaching reaction velocity and to reduce the treatment duration. To realize an optimal treatment following the instructions contained in the section BLEACHING PROCEDURE.

The bleaching light is realized with four high power LEDs (Light Emitting Diode).

COREWHITE is characterized by four bleaching programmes which makes the lamp suitable for every requests: by the uniform bleaching of both dental arches to the bleaching of only a part of those.
 The luminous emitted power is about 2000 mW. The emission spectrum is without UV and IR and solve all the problems concerned with the presence of these radiations in light.
 In fact, UV radiation causes mucosa membrane irritation and IR radiation produces an heating of treatment zone.
 Please read the following thoroughly and strictly observe the operating instructions provided.

DEVICE DESTINATION

COREWHITE has to be used to illuminate patient dental arches for professional bleaching treatment and must be utilized only for this purpose by medical staff or by staff adequately trained and with an appropriate knowledge in the use of this kind of devices.

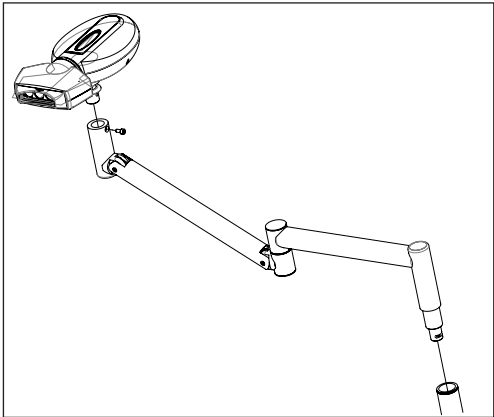
Proclinic Expert is not responsible of incorrect or im- MECHANICAL FIXING proper use of this dental bleaching light.

DEVICE CLASSIFICATION

Device of class I for European Directive 93/42/CEE.
 Device against direct and indirect contact: instrument which applies to type B part A, i.e. instrument that has a specific protection degree against electrical dangers with particular care for admissible currents dissipation and reliability of the ground connection.
 Device IPX0 not protected against liquid penetration and not suitable for the momentary use of inflammable anaesthetic mixtures with air, oxygen or nitrogen protoxide.
 Device for continuous work.
 Device of class II for CEI EN 60825-1:2003.
 Equipment not suitable for use in oxygen-rich environments.
 Equipment not suitable for use with flammable agents.

MECHANICAL FIXING:


COREWHITE lamp must be mechanically connected to the trolley.
 To mount the lamp on the trolley proceed as shown in the figure below.
 1 - Insert the pole in its place on the trolley going through the hole in the cover
 2 - Lock the 3 screws to fix the pole

TECHNICAL DESCRIPTION:

SPECIFICATIONS

Size and movement:	see Figure 1
Weight:	about 700 g
Power supply (without transformer):	12-24 Vac 50-60 Hz
Maximum power consumption:	50 VA
Current consumption:	1.5 A
Light pattern:	80 x 20 mm
Light source:	4 LEDs (3.5 W each)
Emission spectrum:	420-480 nm
Maximum light power emitted:	2000 mW
Maximum required distance from patient:	10 cm
Data transformer:	230V - 50 / -60Hz (Primary) - 17/24 Vac, 50VA min (Secondary)
Transformer protection:	2 fast fuses from 2.5A on the secondary

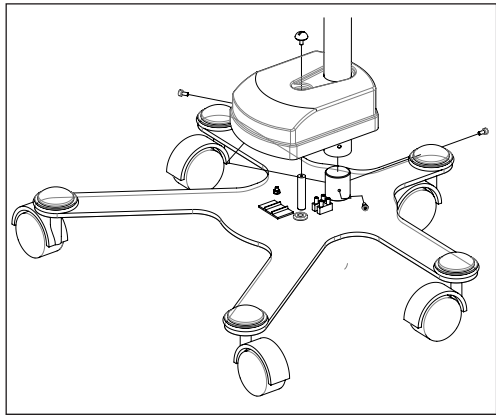
It is now possible to mount the arm on the pole letting the power supply cable slide through the pole and letting it exit under the dome.
 Connect the arm cable with the one on the COREWHITE lamp and insert the lamp on the arm.
 Fix both together with the screw supplied.


NO MODIFICATIONS ARE ALLOWED ON THIS DEVICE.

MAKE SURE THAT COREWHITE IS FIRMLY FIXED ON THE ARM.

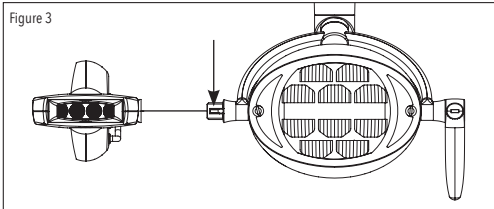MAKE SURE THAT ARM, POLE AND TROLLEY ARE UNITED WITH EACH OTHER.

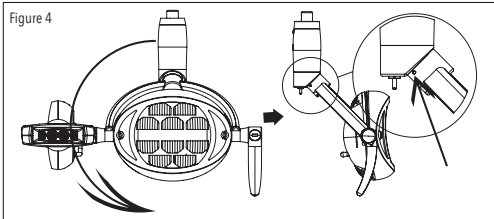For Polaris with 3rd Axis rotation, the attachment of COREWHITE needs an adjusttment of the insert screw with an allen key as for figure 4.

DO NOT FIX COREWHITE TO POLARIS WHEN POLARIS LEDS ARE TURNED ON.

⚠ BE SURE THAT COREWHITE IS STABLE FIXED TO POLARIS.

ELECTRICAL WIRING:

- 1- Fix the connector on the base with the cable going out from the pole.
 - 2 - Plug the power supply placed under the base with a current outlet.
- It's mandatory to use a medical power supply with an insulation from the 230 V 50/60 Hz electricity grid and a tension between 12-24 Vdc 6A compliant to CEI EN

60601-1 guaranting 2MOPP and is necessary that, on the primary winding an 1,6 A fuse is used.

THE POWER SUPPLY WITH WHICH THE LAMP IS POWERED MUST BE SEPARATED FROM

THE ELECTRICITY GRID (230V) WITH A MEDICALLY CERTIFIED COMPONENT ABLE TO GRANT 2MOPP.

VERIFY THAT THE CABLES ARE CORRECTLY FIXED AFTER THE ASSEMBLY AND CAN'T BE

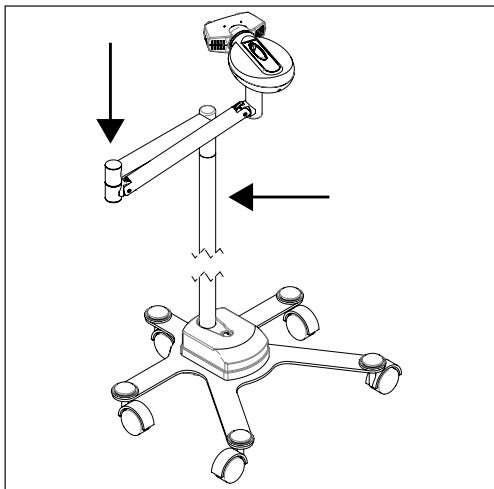
EXTRACTED FROM THE TERMINAL WITHOUT UNLOCKING THE SCREWS.

THE INSTALLATION AND ALL THE MAINTAINANCE OPERATIONS MUST ALWAYS BE PERFORMED WITH THE LAMP DISCONNECTED FROM THE ELECTRICITY GRID.

MOVING THE TROLLEY:

To move the trolley:

- 1- Unlock the brake on the braked wheels.
- 2- Bend the arm as shown in figure.
- 3- Move the trolley holding it from the pole and the arm.


USE:

Control board

The bleaching light presents a control board with a display. The control board presents 5 buttons (see Figure 6):

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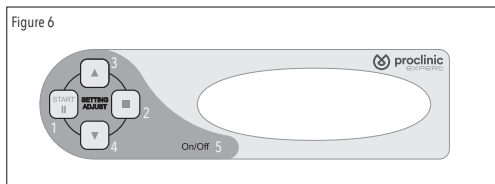With ON/OFF button it is possible to turn on/off the lamp.

With the START button it is possible to confirm a selection and to start a bleaching treatment.

UP/DOWN buttons allows to select programmes in the various menus and to increase/decrease the bleaching duration.

With STOP button it is possible to exit from a selection and to stop the execution of a bleaching treatment.

PAUSE button allows to stop temporarily the execution of a bleaching treatment.

Turn on/off light


COREWHITE has an ON/OFF button (see Figure 6) placed on control board which turns on/off the light.

Menu

When the light is on (see the section TURN ON/OFF LIGHT) it is possible to select through the display one of the following two menus (see Figure 7):

- bleaching menu;
- diagnostic menu.

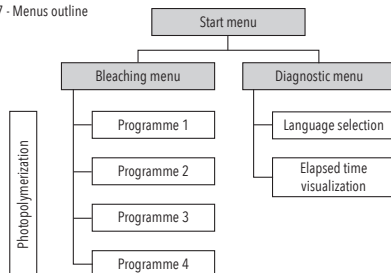Figure 7 - Menus outline

To select the desired menu use the buttons UP/ DOWN. To confirm the selection press START. To exit from a selection press STOP.

Bleaching menu

With the bleaching menu it is possible to select the desired bleaching treatment with a duration fixed by the user. There are four bleaching programmes (see Table 1):

Table 1 - Summarizing table of bleaching programmes

	Function	Left LED power	Right LED power	Central LEDs power
Photopolymerization	Liquid dam photopolymerization	100 %	100 %	100 %
Programme 1	Uniform bleaching of both dental arches	100 %	100 %	100 %
Programme 2	Right lateral bleaching of both dental arches	0 %	100 %	0 %
Programme 3	Left lateral bleaching of both dental arches	100 %	0 %	0 %
Programme 4	Tooth selective bleaching	0 %	0 %	100 %

Photopolymerization. This programme is for liquid dam photopolymerization. All four LEDs are at a power of 100%.

Programme 1. Uniform bleaching of both dental arches. All four LEDs are at a power of 100%.

Programme 2. Bleaching of right lateral part of dental arches. Right LED is at a power of 100%, left and central LEDs are at 0%.

Programme 3. Bleaching of left lateral part of dental arches. Left LED is at a power of 100%, right and central LEDs are at 0%.

Programme 4. Tooth selective bleaching. Central LEDs are at a power of 100%, right and left LEDs are at 0%.

When the programme has been selected, treatment duration, in minutes, must be set. For each programme there is a duration of 20 minutes by default. It is possible to change the treatment default duration through the buttons UP/DOWN in order to obtain the duration desired.

To confirm the duration press START.

Selecting the bleaching programme and the treatment duration, it is possible to start the treatment following the instructions on display.

DIAGNOSTIC MENU

Diagnostic menu presents two programmes:

- language selection;
- elapsed time visualization.

To select the desired menu push the buttons UP/ DOWN.

To confirm the selection push START.

To exit from a selection push STOP.

In selection language programme, it is possible to choose one of the following working language:

- English;
- Italian;
- French;
- Spanish;
- German.

In the elapsed time visualization programme, it is possible to visualize the using time of the light. The using time is referred to time in which LEDs are turned on for bleaching treatment.

BLEACHING PROCEDURE:

Dental bleaching treatment has to be executed adopting an appropriate operative protocol in order to assure an optimal effect. The operative protocol phases are in sequence:

1. execute an appropriate teeth cleaning with typical treatments;
2. dispose vaseline on patient lips;
3. introduce the apposite open-mouth to separate lips and maintain tongue down in order to avoid accidental contacts of patient soft tissues with bleaching gel;
4. insert cotton rolls in the upper and/or lower barrelvault to guarantee a better position control of soft tissues;
5. dry teeth using the dental unit syringe to facilitate the adhesion of the dam to the membrane mucous;
6. apply accurately the photopolymerizable liquid dam on gums and mucosa and insulate the other parts with salivary rolls and cotton lint (in case also on tongue);
7. start the liquid dam photopolymerization to harden the dam (select Programme 1);
8. to avoid an high dental sensibility in patient, use the apposite desensitizing product: put it on teeth to treat, wait 30 seconds (this is generally time to act) and blow air through dental unit syringe;
9. put bleaching hydrogen peroxide based gel on the teeth surface to treat, realize a uniform distribution of gel without using too much;
10. place light source of lamp at a maximum distance of 10 cm from the mouth and set the desired bleaching programme.

During the bleaching treatment, maintain the light source near to teeth. Select the bleaching programme and the its duration (see the section USEMENU).

⚠ ATTENTION:

Do not stare into beam when light is on.

Patient and operator must wear protection glasses during the treatment.

To stop temporarily the treatment push PAUSE button. To resume the treatment push START button. To exit definitely from bleaching programme, push STOP button.

When treatment is finished:

11. remove bleaching gel with saliva-aspirator without spout paying attention to avoid any contacts with mucosa;
12. remove laminated dam and cotton rolls;
13. remove open-mouth;
14. rinse the oral cavity abundantly with water;
15. remove vaseline from patient lips.

⚠ ATTENTION:

Radiotelephones may cause interferences with electromedical instruments. To guarantee the correct working of electromedical instruments, it is forbidden to use radiotelephones in medical offices and hospitals.

MAINTENANCE:

General hygiene regulations

"COREWHITE" has been designed to allow easy disinfection, with particular attention to hygiene. The following advices will simplify the cleaning process on your dental operating light.

⚠ Use water. DO NOT USE ABRASIVE SUBSTANCES

AND/OR MATERIALS, especially alcohol, trichloroethylene, gas, turpentine and similar.

Do not spray or spill liquids on the light.

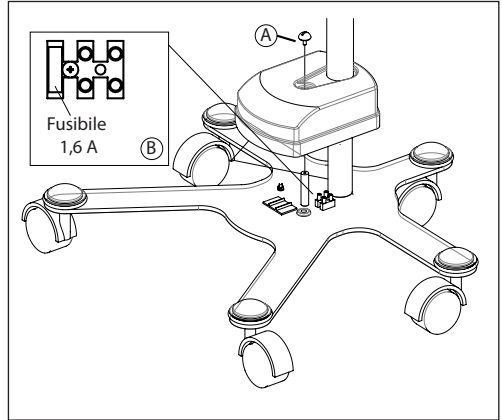
Use a very soft cloth soaked with distilled water and very soft movements. Dry with a soft cloth.

⚠ ATTENTION:

Do not carry out any maintenance or cleaning on the lamp when the power is on.

⚠ If the mobile arm does not respond appropriately to the stresses of the applied load open the door on the arm and gradually tighten the compression nut of the spring placed inside checking from time to time if the compression of the spring is adequate to balance lamp's load.
The spring's stiffness verification has to be done annually by a specialized technician.

Periodically, a qualified technician must check the grounding of the lamp arm. A QUALIFIED TECHNICIAN MUST VERIFY THE STIFFNESS OF THE SPRING ANNUALLY. To substitute the 1.6A fuse is necessary to lift the dome after the unlocking of the screw A and have access to the fuse on the terminal in position B.
Close back the dome and lock the screw A.

TRANSPORT AND STORAGE CONDITIONS:

Please keep the dental operating light IRIS in a closed, covered and dry place:

- temperature: 20 a +70 °C;
- relative humidity: dal 10 al 90%;
- atmospheric pressure: da 700 a 1060 hPa.

AMBIENTAL USE CONDITIONS:

Use in medical purposes rooms. Do not subject the lamp to thermal stress and do not expose it out of the following limits:

- temperature: +10°C a +35 °C;
- relative humidity: dal 30 al 75%;
- atmospheric pressure: da 700 a 1060 hPa.

REPARATUREN:

Proclinic Expert is not responsible to unauthorized repairs made by third parties. Please, contact us directly to have the address of authorized distributors and technicians.
Proclinic Expert responsibility is limited to:

- assembly operations, extensions, adjustments, modifications or repairs made by authorized staff and with the use of original replacement pieces provided by Proclinic Expert;
- electrical wiring, in which the device is installed, has to be conformed to safety laws in force;
- use of the light in accordance with the instructions given.

WARRANTY:

The manufacturer hereby certifies that this product has been properly manufactured in complete compliance with the European and international regulations. The product is covered by warranty for a period of 12 months from the date of delivery to the end user. Any part to be replaced under warranty shall be returned to Proclinic Expert within 8 days. Warranty is limited to replacement or repair of the single parts or components that will result as defective.

This warranty does not include:

- labour costs, travelling expenses of the technical personnel, transport, etc.;
- any damage or defect caused by improper use or by any other use than the one for which it was originally designed;
- any damage caused by repair, alteration or disassembly of the product or part of

the product carried out by the purchaser itself or by third parties not authorised by the manufacturer;

- normal wear of parts and components such as LEDs, etc.

This warranty does not cover direct or indirect damage of any kind to people and property caused by any malfunction of the product. In need of repair under warranty of the product the purchaser shall revert to the retailer, when it is officially authorised, to an authorised service centre or to the manufacturer itself. Any defective part covered by warranty will be replaced free of charge. This warranty does not entitle the replacement of the whole device. In case of dispute on this warranty, on the quality and efficiency of the delivered product the purchaser shall not stop or delay payment. No claim for damages shall be accepted in case the product stops working.

This warranty is not valid in case of:

- damage resulting from fall, fire, pouring of liquids, lightning, fatality, or from causes other than defects in workmanship;
- improper installation;
- incorrect connection to mains or lacking of protection devices;
- serial numbers or CE mark have been removed, deleted, altered.

Any component to be replaced under warranty is to be returned to the factory that will send the spare part. If not returned, the part sent in exchange will be debited to the customer.

WARNING RAEЕ:

Indications about disposal

The illegal or inadequate disposal of disused product by the user implies the application of economic / administrative sanctions accordingly to the Law.

Indications about disposal

The European directives 2002/95/CE, 2002/96/ CE and 2003/108/CE foresee the dental operating light in the category of electrical and electronic products that should be disposed separately from the municipal waste stream (RAEE). The disused product shall be disposed separately to optimise the recovery and re-cycle of materials that compose the unit, in order to obtain a significant energy saving and to prevent potential negative consequences for environment and human health.

This crossed-out bin symbol is shown to all products that request to be disposed via designated collection facilities.

The user can hand over the disused dental operating light to his/ her city office or to the designated collection facilities appointed by his/her government or local authorities.

The illegal or inadequate disposal of disused product by the user implies the application of economic / administrative sanctions accordingly to the Law.n.

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