FICHA TÉCNICA

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TECHNICAL DATA SHEET

BESTDENT BLUE PROTECTIVE FILM		
REFERENCES	20980	
MANUFACTURER	CHL MEDICAL SOLUTIONS, Srl Via Felice Cavallotti, 15 20122 Milano - Italy	
CLASSIFICATION	Medical Device, Class I, according to Regulation (EU) 2017/745 on medical devices.	
DESCRIPTION	Blue adhesive film. The product is non sterile and for single use. Easily removable	
SPECIFICACIONS	1.200 pre-cut sheets in one roll. Single sheet: 10x15 cm	
PACKAGING	Box with 1 roll	
INTENDED USE	For dental use. Protection of instruments and materials.	
INSTRUCTIONS FOR USE	<ul style="list-style-type: none"> • Remove dispensing-slot cover. • Leave the film protector in the box. Insert the end of the film protector into the slot. • Tear off film protector and open its end. Place film protector on turbine and tubing. • Remove and dispose film protector after each patient. 	
SAFETY INFORMATION	The product can ignite when in contact to a flame. If it burns, it will cause tear-producing smoke that could lead to suffocation.	
HANDLING AND STORAGE CONDITIONS	Store in a cool dry place, away from direct sunlight. Protect from moisture.	
SDS	Not applicable.	