



# EasyOil

## The best lubrication adapter. Ever.



Example configuration

### The universal solution for a trouble-free handpiece care.

**EasyOil** is the *universal support* that can conveniently gather up to five different adapters to meet all your dental handpiece maintenance needs.





An effective handpiece care results in fewer failures and *great savings* on repair costs.

**EasyOil** allows you to take care of your handpieces in an easy, fast and comfortable way because all you need is already there, readily at hand.

With just a few, simple gestures **EasyOil** will help you make the life of your handpieces longer.

**Patented – 100% made in Italy**

### Perfect handpiece care has never been easier.

-  Lower the adapter for the handpiece you want to lubricate.
-  Insert the handpiece onto the adapter.
-  Wrap the handpiece in a paper towel and press the button for a couple of second to spray the oil.
-  Spray the oil in the chuck system for 1 second.

### Did you know...?

To prevent early rotor failures and expensive repairs, constructors recommend to maintain the chuck system in turbines and red contrangles. Just spray a little lubricant in the chuck to keep it clean and perfectly efficient.

### Fit for any spray can.

**EasyOil** can be installed on all the sprays for dental handpieces available on the market. No special spray can is required.



### Stunningly flexible.

No matter what coupling your turbine is for, **EasyAdapters** are available in a wide range of models.



**EasyAdapters** are built in solid aluminium for durability, color coded and laser engraved for easy recognition: you will always be sure about which one to use.

**EasyOil** comes with 1 **EasyAdapter** for contrangle and handpieces with ISO/E-Type (INTRAmatic®) attachment and 1 **EasyAdapter** for chuck maintenance already included. **EasyAdapters** for turbines available separately.

Your dealer

**SILVER**<sup>®</sup>  
dental handpiece service

[www.silversnc.com](http://www.silversnc.com)  
[www.easyoil.it](http://www.easyoil.it)



KaVo, INTRAmatic and KaVo Spray are registered trademarks of Kattenbach & Veight GmbH & Co. Blen Air and Lubrifluid are registered trademarks of Blen Air Dental SA. Sirona and T1 Spray are registered trademarks of Sirona Dental Systeme GmbH. W&H and F1 Service Oil are registered trademarks of W&H Dentalwerk Barmosee GmbH. NSK and PANASPRAY are registered trademarks of NSK Nakajima Inc.

80-01130A102EU

