G-ETCH

Définition

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G-ETCH

Definition

G ETCH is a dental etching gel at 38% phosphoric acid. Non-drying, medium viscosity gel, thixotropic, washes off quickly and easily, excellent water solubility, very good contrast.

Safety notes

Warnings

G ETCH contains phosphoric which may cause respiratory irritation, severe skin burns and eye damage.

Avoid contact with skin to prevent serious skin irritation. In case of contact with skin, wash and rinse thoroughly with soap and water.

Avoid eye contact to prevent serious possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek for medical advice.

Harmful if swallowed. Rinse mouth thoroughly. Do not induce vomiting and seek for medical advice or call a Poison Center.

Precautions

Wear suitable protective eyewear, mask, clothing and gloves.

Protective eyewear is recommended for patients.

Contraindication

Not known.

Instruction for use

1. Prepare tooth by cleansing the enamel surface to be etched with a slurry of pumice without oil or fluoride.
2. Apply G-ETCH on both enamel and dentin. Recommended time is 20 seconds.
3. Rinse off G-ETCH with copious amount of water. Do not contaminate with other fluids like saliva.
4. Leave surface wet, or dry surface with uncontaminated air, depending on the instructions for use for the bonding you are using. Etched enamel should appear chalky white. If not, reapply G-ETCH following points 2 and 3 again.

Filling method for the jumbo kit

Important notice

Wearing protective goggles is compulsory during the filling operation.

1. Remove caps protecting syringe ends.
2. Prepare the 3ml syringe for loading by depressing plunger to the end of the barrel.
3. Secure the ends of the two syringes (the empty 3ml one and the 50ml bulk pack storage syringe) together by interlocking them, using the luer lock coupling device: hold barrel of the 50ml syringe securely handscrew the empty 3ml syringe with the other hand by turning clockwise; be sure the fit is tight but do not force.
4. Pull the plunger of the dispensing syringe (3ml) to fill it. Never push on the plunger of the storage syringe (50ml).
5. When the dispensing syringe (3ml) is full, slightly push its plunger to avoid any leaking when separating the 2 syringes from each other.
6. To disengage syringes, grip the luer lock with one hand and with the other; turn the barrel of the 3ml syringe anti clockwise.
7. Attach the appropriate protective caps on both syringes until ready to use.
8. To apply etching gel, remove cap from the 3ml syringe and insert a disposable needle application tip.

Storage & shelf-life

Store in a dry, cool and well-ventilated place. Protect from sunlight and keep away from heating sources, metals and alkaline-based products. Container should be kept tightly closed. Keep at a temperature not exceeding 24°C.

When stored under such conditions, the material has a shelf-life of 2 years.

JUMBO KIT: keep the 50ml syringe in a fridge,

References & presentations

MET- 24 - 24 x 1.2ml syringes.

MET-6 - 4 x 1.2ml syringes + 8 disposable needle tips.

JKB/JKV - 1 x 50ml storage syringe (blue or green) + 5 empty syringes + 40 disposable needle tips.

JRB/JRV - 1 x 50ml storage syringe (blue or green), only.

Important notice

- Do not throw away the box and the instruction for use until the product is completely finished.
- High humidity could damage the packaging on which important instructions and legal information are printed.
- Storage at a too high temperature could cause to age prematurely and inhibit its polymerization.
- Storage at a too low temperature leads to the decomposition of the product which will render it completely unusable.

Disclaimer

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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