


Concise™ Orthodontic Bonding System


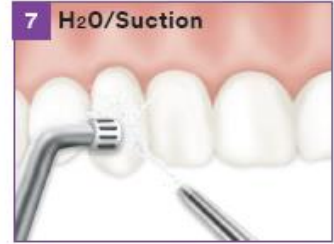


Mixing Ratio Paste A	Mixing Ratio Paste B	Approximate Working Time (after mixing)
1	1	1 min. 45 sec.
1	2	3 min. 30 sec.
2	1	55 sec.

More Paste A will accelerate the set
More Paste B will retard the set

Concise™ Orthodontic Bonding System

With Transbond™ MIP
Moisture Insensitive Primer

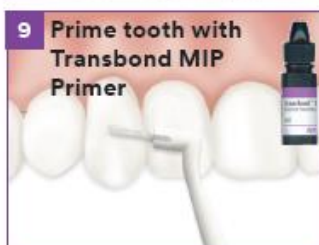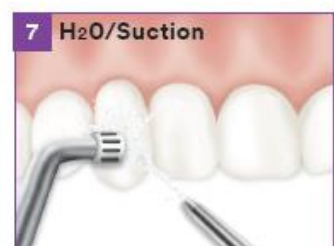

Mixing Ratio Paste A	Mixing Ratio Paste B	Approximate Working Time (after mixing)
1	1	1 min. 45 sec.
1	2	3 min. 30 sec.
2	1	55 sec.

More Paste A will accelerate the set
More Paste B will retard the set