3M Unitek

Transbond™ Plus Self Etching Primer

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REF 712-090, 712-091

ENGLISH Instructions For Use

Product Description and Indication for Use: Transbond™ Plus Self Etching Primer is both an etchant and a primer combined into one product. This product contains “methacrylated phosphoric acid esters.” Transbond Plus Self Etching Primer is indicated for use as a bonding mediator for bracket attachments. It is a combined etchant and primer in orthodontic treatment and is for use only with light-curing direct bonding orthodontic adhesives.

Note: One Transbond Plus Self Etching Primer unit contains sufficient material for etching and priming one arch only. Use two units when bonding both arches. Protect the material from light and heat prior to activation.

Transbond Plus Self Etching Primer is intended for single patient use only. Reuse, even with a new disposable applicator, could result in cross contamination.

Warnings:

- **IRRITANT** – may cause sensitization by skin contact.
- Possible acrylic sensitivity – this product contains acrylates which may cause allergic reaction in some individuals.

Important: All resin-based primers that penetrate into the enamel rods may under certain conditions temporarily alter the appearance of the enamel. It does not discolor the enamel; rather it creates a contrast to the chalky appearance of the surrounding enamel.

Note: The primer is not recommended for fluorosed or mottled teeth.

Precautions: This product contains phosphoric acid esters. Avoid contact with skin and eyes. Wear eye protection and gloves when handling this product.

CAUTION: For patients with poor oral hygiene, consider having the patient undergo a thorough dental cleaning prior to the bonding appointment.

First Aid for eye contact: Immediately flush with plenty of water. See a physician if irritation persists.








First Aid for mucous membrane and skin contact:

Flush mucous membrane with water. Superficial changes of the gingiva may occur in individual cases and are reversible. Wash affected area with soap and water. See a physician if irritation persists.

How to use:

1. Isolate teeth using the NOLA Dry Field System or cheek and tongue retractors, dri-angles and cotton rolls.
2. Prophyl teeth with an oil-free pumice or paste. Rinse with water. Remove excess water but do not dry tooth surface. Do not allow teeth to become recontaminated with saliva before applying the Transbond Plus Self Etching Primer. Figure 1.
- Note:** Do not activate the primer without the disposable applicator in place to prevent primer from spilling. If using the Transbond™ Plus Self Etching Primer Easy Roller, follow instructions 011-621.
3. Hold the sheath of the disposable applicator in one hand. Use the thumb and index finger of the other hand to completely squeeze the liquid out of the black reservoir towards the disposable applicator. Figure 2.
4. Completely empty the contents of the black reservoir into the white (middle) reservoir by squeezing the liquid forward and carefully fold back the package at the interface of the black and white reservoir. To keep the liquid from flowing back into the black reservoir, keep the black reservoir tightly squeezed with your thumb and index finger while bending the package. The black (largest) and white (middle) reservoir sections are now on top of each other. Figure 3.
5. Starting at the far end squeeze the liquid from the white (middle) reservoir into the purple (smallest) reservoir of the blister package using controlled pressure. Figure 4.
6. Once the liquid is transferred to the purple (smallest) section of the blister pack, **churn and swirl** the applicator inside purple reservoir for 5 seconds to completely mix chemicals and thoroughly coat applicator tip. Figure 5.
7. Remove the applicator from the reservoir.
The applicator tip must have a light yellow color. If not yellow, re-squeeze contents from outer reservoir to middle reservoir to small reservoir and mix well using a churning and swirling motion until yellow color is observed.
- Note:** Keep activated unit in an upright position during application to assure material stays in the reservoir.
8. **RUB** the saturated tip of applicator onto tooth surface. **Continue RUBBING liquid onto enamel while applying some pressure for a minimum of 3-5 seconds per tooth.** Figure 6.
Note: If bonding to aprismatic enamel, i.e. molars, deciduous teeth, lingual surfaces, increase the rubbing time per tooth to achieve a desirable etch pattern.
9. **Redip applicator into reservoir to saturate tip before rubbing it onto next tooth.** Figure 7.
10. Repeat steps 7 through 9 for each tooth.
11. When all teeth on one arch are primed, use an oil and moisture-free air source to deliver a **GENTLE air burst for 1-2 seconds** to each tooth to dry primer into a thin film. Figure 8.
Note: Direct air stream across the tooth mesial distal. Blowing the primer into the gingiva can cause irritation to the gingiva and possible contamination of the primer layer from wicking of gingival crevicular fluid.
Important: One Transbond Plus Self Etching Primer unit will etch and prime all teeth on **one arch only.**
12. Proceed immediately with bonding. If bonding is delayed, apply another coat, deliver a **GENTLE air burst** to dry primer into a thin film and bond.
13. Light cure.

Storage: Keep away from sunlight. Store at temperatures between 2°C/35°F and 24°C/75°F.

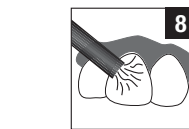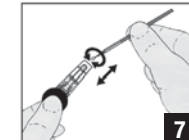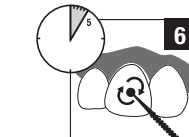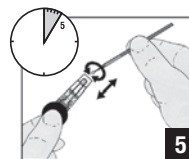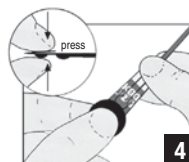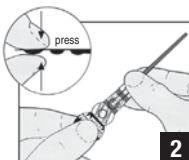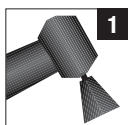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© 2008, 2012 3M. All rights reserved.
**3M Deutschland GmbH