For better dentistry

DENTSPLY

AH Plus®

AH Plus *jet*™
Mixing Syringe

Root Canal Sealing Material

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Caution: For dental use only.

In the USA: Rx only

| Content | Page |
|---|------|
| 1 Product description..... | 2 |
| 2 General safety notes..... | 3 |
| 3 Step-by-step instructions..... | 3 |
| 4 Cleaning, Disinfection and Sterilisation..... | 5 |
| 5 Lot number and expiration date..... | 5 |
| 6 Contact..... | 5 |

1 Product description

AH Plus® Root Canal Sealing Material (referred to below as “AH Plus”) is a two-component paste/ paste root canal sealer based on epoxy-amine resin, offering the following features:

- Long term sealing properties
- Outstanding dimensional stability
- Self-adhesive properties
- Very high radioopacity

1.1 Composition

AH Plus Paste A

- Bisphenol-A epoxy resin
- Bisphenol-F epoxy resin
- Calcium tungstate
- Zirconium oxide
- Silica
- Iron oxide pigments

AH Plus Paste B

- Dibenzyl diamine
- Amino adamantane
- Tricyclodecane-diamine
- Calcium tungstate
- Zirconium oxide
- Silica
- Silicone oil

1.2 Delivery forms

AH Plus comes in the following deliveries:

- as **AH Plus in tubes** for manual mixing of pastes A and B
- as **AH Plus Jet™ Mixing Syringe** for direct intra-oral application, offering a more precise, convenient and faster procedure.

1.3 Indications for use

Permanent obturation of root canals of teeth of the secondary dentition in combination with root canal points.

1.4 Contraindications

Hypersensitivity against epoxy resins, amines or other components of the root canal filling material.

2 General safety notes

2.1 Warnings

- AH Plus contains epoxy resins (paste A) and amines (paste B) which may cause sensitisation in susceptible persons. Do not use AH Plus in persons allergic to epoxy resins or amines or any other components of the products.
- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, immediately remove material with cotton and wash thoroughly with water and soap. In case of skin sensitisation or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, immediately remove material with cotton, flush mucosa with plenty of water and expectorate water. If inflammation of mucosa persists, seek medical attention.
- **Do not swallow or take internally.** If accidental swallowing occurs, drink lots of water. If nausea or illness develops, seek medical attention.
- **Avoid overfilling.** When overfilling occurs, the material is usually tolerated very well by the surrounding tissue. In cases where larger amounts of material are pressed into the mandibular canal, immediately remove the material.

2.2 Precautions

- This product is intended to be used only as specifically outlined in the Directions for Use. Any use of this product inconsistent with the directions for use is at the discretion and sole responsibility of the dental practitioner.
- Wear protective glasses, mask, clothing and gloves. Protective glasses are recommended for patients.
- Under certain storage conditions, AH Plus Paste B may slightly separate. It has been shown that this process does not negatively affect the performance of the mixed product.
- Interactions: None known.

2.3 Storage conditions

All products should be used at room temperature. Inadequate storage conditions may shorten the shelf life and may lead to product performance problems.

For tubes or syringes we recommend the following:

1. Store at temperatures between +10 °C and +24 °C.
2. Keep tubes or syringe tightly closed.

2.4 Adverse reactions

The product may cause allergic contact dermatitis.

3 Step-by-step instructions

3.1 Preparation

Prior to the application of AH Plus, prepare, clean, and dry the root canals to be filled using state-of-the-art endodontic techniques.

3.2 Using AH Plus in tubes

1. Mix equal volume units (1:1) of Paste A and Paste B on a glass slab or a mixing pad using a metal spatula¹. Mix to a homogeneous consistency.
2. Do not exchange caps of tubes. The coloured cap belongs to the coloured tube; the white cap belongs to the white tube.

3.3 Using AH Plus Jet Mixing Syringe

The Jet Mixing Syringe allows direct application of the sealer into the root canal orifices and into the coronal parts of the canals.

1. Remove the cap by turning it 90° counter-clockwise and pulling it.
Note: For AH Plus Jet Mixing Syringe, the syringe filling volumes of pastes A and B vary slightly. To ensure appropriate mixing, bleed the syringe prior to the first use.
2. Attach the mixing tip to the syringe, aligning the notch.
3. Turn the tip 90° clockwise and secure the mixing tip in place.

AH Plus Jet Mixing Syringe allows direct application of the sealer into the root canal orifices and into the coronal parts of the canals. The intra-oral tip can be rotated and adjusted in angle in order to meet individual anatomic conditions and treatment requirements. Use of a DisposaShield® is recommended for intraoral use (ReOrder No.: A880065). Discard DisposaShield after use. For application of the sealer, carefully apply steady pressure onto the plunger. The two pastes are automatically mixed in equal volume units.

Having finished the sealer application, the mixing tip must be discarded. Remove the mixing tip by turning it 90° counter-clockwise and pulling it. Replace the cap on the syringe head, aligning the notch, then turn it 90° clockwise.

3.4 Application of AH Plus

As a standard technique, AH Plus is used in combination with gutta-percha, silver or titanium root canal points.

For filling techniques where most of the canal is obturated by endodontic point material, apply only a light coating of AH Plus to the canal walls. For the Master-Point-Technique, select a gutta-percha point (or alternatively a paper point or a reamer) in the size of the last instrument used during apical preparation. Wet the canal walls with AH Plus through a pumping or simultaneously rotating movement in a counter-clockwise direction of the point/reamer. Alternatively, apply AH Plus onto the tip of a Lentulo spiral. Advance the Lentulo spiral slowly to the apex running at very low speed. Avoid the formation of air bubbles in the material and overfilling of the canal. Withdraw Lentulo very slowly still running at low speed. Dip disinfected and dry points into AH Plus and insert them into the canal with a slow pumping motion.

Working Time

The working time is 4 hours at 23 °C².

Setting Time

The setting time is 8 hours at 37 °C².

3.5 Removal of root canal filling

If AH Plus is used in combination with gutta-percha points, the root canal fillings can be removed using conventional techniques.

4 Cleaning, Disinfection and Sterilisation

4.1 Cleaning of Instruments

To prevent AH Plus Jet Mixing Syringe from exposure to spatter or spray of body fluids or contaminated hands, or oral tissues, use of a protective barrier is recommended to avoid package contamination. Repeated disinfection may damage label.

Spatulas, mixing slabs and instruments should be cleaned immediately after use with alcohol or acetone.

Dispose all products in accordance with the local regulations.

5 Lot number () and expiration date ()

1. Do not use after expiration date. ISO standard is used: "YYYY/MM".
2. The following numbers should be quoted in all correspondence:
 - Reorder Number
 - Lot number
 - Expiration date

6 Contact

If you have any questions, please contact:

Manufacturer ():
DENTSPLY DeTREY GmbH
De-Trey-Str. 1
78467 Konstanz
Germany
Phone +49 (0) 75 31 5 83-0

Distributor:
DENTSPLY Limited
Building 1
Aviator Park, Station Road
Addlestone, KT15 2PG
Phone (0 19 32) 85 34 22

© DENTSPLY 2008-07-15

¹ Mixing ratio by weight is 1 g of paste A to 1.18 g of paste B.

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DENTSPLY DeTrey GmbH
DENTSPLY International, Inc.