bestdent

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(EN)

INDICATIONS:

Cementation in inlays, onlays, crowns and bridges made from any kind of material used for operatoria and prosthetic dentistry.

COMPOSITION:

Alumino-Silicated Glass
Polyacrylic acid

INSTRUCTIONS FOR USE:**POWDER AND LIQUID DISPENSING**

- The standard powder to liquid ratio is 0.85g the powder and 0.45g the liquid.
- For accurate dispensing of powder, lightly tap the bottle against the hand. Do not shake or invert.
- Hold the liquid bottle vertically and squeeze gently.
- Close bottles immediately after use.

MIXING

Dispense powder and liquid onto the pad. Using the plastic spatula, divide the powder into 2 equal parts. Mix the first portion with all of the liquid for 10 seconds. Incorporate the remaining portion and mix the whole thoroughly for 15-20 seconds (total time within 30 seconds).

STORAGE

Store at temperature of 39.2 - 77.0°F

WARNING:

- Do not leave at reach of children.
- In case of any change on the original function of the product, stop using it and contact the manufacturer.

For Professional Dental use only

WARNINGS

Intention of use: Cementation in lays, onlays, crowns and bridges made from any kind of material used for operatoria and prosthetic dentistry.

Composition: Alumino-Silicated Glass and Polyacrylic acid.

The glass ionomer cement is not irritable to the skin. In case of contact with the eyes, wash immediately with abundant water during at least 15 minutes and if the bother persist consult a doctor. In case of contact with skin wash the area with soap and water. In case of inhalation clean the nose with water and medical assistance is required. Before the use of these products always use rubber dam.

Some patients can present allergy to the Glass Ionomer Cement Type I or any of its components. In this case suspend it use. Do not storage at temperature above 24°C. Close the packages immediately after to dispense the product.

Contraindication: The material must not be used when the techniques application described are not possible.

bestdent guarantees the product against any fault of manufacture or of the material. bestdent does not take responsibility for the use of the product out of the intention of use specified or out of the instructions of use, or out of the precautions, the warnings, and indicated contraindications.

Being the only obligation is the replacement of the defective product.

Information Post-production:

If the use of these product on a patient or any one else some anomaly occurs please send detailed report to MEDENTAL INTERNATIONAL. medental_inc@hotmail.es

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