For better dentistry

**DENTSPLY**

# Spectrum<sup>®</sup>TPH<sup>®</sup>3

**Universal Microhybrid Composite Restorative**

**Universelles Microhybrid-Komposit-Füllungsmaterial**

**Composite micro-hybride de restauration universel**

**Composito microibrido universale**

**Composite restaurador micro-híbrido universal**

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CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description.....	2
2 Safety notes .....	3
3 Step-by-step instructions .....	5
4 Hygiene.....	7
5 Lot number and expiration date.....	9

### 1 Product description

**Spectrum<sup>®</sup> TPH<sup>®</sup>3** restorative is a visible-light activated, radiopaque sub-micron hybrid composite for anterior and posterior restorations. The composite is pre-dosed in Compules<sup>®</sup> tips or delivered in traditional syringes and is available in a selection of precise VITA<sup>®</sup>1 shades.

#### 1.1 Delivery forms

- Predosed Compules<sup>®</sup> tips
- Syringes

#### 1.2 Composition

- TEGDMA
- Dimethacrylate
- Multifunctional polymethacrylate
- Camphorquinone
- Ethyl-4(dimethylamino)benzoate
- Butylated hydroxy toluene (BHT)
- UV stabilizer
- Barium-aluminium-borosilicate glass (mean particle size < 1 µm)
- Silanated barium-boron-fluoro-alumino-silicate glass (mean particle size < 1 µm)
- Highly dispersed silicon dioxide (particle size 10-20 nm)
- Fluorescent agent
- Iron oxide pigments
- Titanium dioxide

#### 1.3 Indications

- Restorations of all cavity classes in anterior and posterior teeth.

<sup>1</sup> VITA<sup>®</sup> is a registered trademark of VITA Zahnfabrik H. Rauter GmbH & Co. KG.

## 1.4 Contraindications

- Use with patients who have a history of severe allergic reaction to methacrylate resins or any of the other components.

## 1.5 Compatible adhesives

The material is to be used following application of a compatible (meth)acrylate-based dentin/enamel adhesive such as DENTSPLY adhesives designed for use with visible light-cured composite restoratives (e.g. the etch and rinse adhesives *XP BOND*<sup>®</sup>, Prime&Bond<sup>®</sup> *XP* and Prime&Bond<sup>®</sup> *NT* or Xeno<sup>®</sup>, self-etching adhesives).

## 2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

### Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

### 2.1 Warnings

The material contains methacrylates and polymerizable monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

### 2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Tightly close syringes immediately after use.
- Use of Compules<sup>®</sup> tips with the Compules<sup>®</sup> tips gun is recommended.
- The Compules<sup>®</sup> tips gun is only intended to be used with DENTSPLY Compules<sup>®</sup> or Compula<sup>®</sup> tips.
- Compules<sup>®</sup> tips may be used for direct intraoral application of restorative material into a cavity or for indirect application by first placing the restorative material on a pad. After intraoral use, discard Compules<sup>®</sup> tips and do not reuse in other patients in order to prevent cross-contamination.

- To prevent Compules® tips (when used for indirect application) and syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the devices are handled offside the dental unit with clean/disinfected gloves. Discard devices if contaminated.
- The material should extrude easily. **DO NOT USE EXCESSIVE FORCE.** Excessive pressure may cause the Compules® tip to rupture or to eject from the Compules® tips gun.
- Interactions:
  - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening of the product.
  - If refrigerated, allow material to reach room temperature prior to use.

### **2.3 Adverse reactions**

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

### **2.4 Storage conditions**

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store at temperatures between 2 °C and 28 °C (35 °F and 82 °F).
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date.
- Store in a dry environment (< 80% relative humidity).

### 3 Step-by-step instruction

#### 3.1 Shade selection

Shade selection should be made prior to the restorative procedure while the teeth are hydrated. Remove extraneous plaque or surface stain if any with prophylaxis paste. Use the Spectrum® TPH<sup>3</sup> shade guide provided with samples of original Spectrum® TPH<sup>3</sup> restorative material. The color coding dot on the shade guide matches the colored label on the syringe. Alternatively, a VITAPAN®<sup>2</sup> classical shade guide may be used. The Spectrum® TPH<sup>3</sup> shade corresponds to the central part of the respective VITA® tooth.

#### Avoid contamination of the shade guide.

To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder. Always make sure individual tabs are completely reprocessed before placing back into shade guide.

#### 3.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. Nupro® prophylaxis paste).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

#### 3.2.1 Matrix placement and wedging (Class II restorations)

1. Place a matrix (e.g. AutoMatrix® or Palodent® Plus matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BiTine® ring placement is recommended.

#### 3.3 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's Instructions for Use for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

#### 3.4 Application

Apply material in increments immediately after the application of the adhesive. The material is designed to be cured in increments up to a 3 mm depth/thickness.

#### Using Compules® tips

1. Insert Compules® tip into the notched opening of the Compules® tips gun. Be certain that the collar on the Compules® tip is inserted first.
2. Remove the colored cap from the Compules® tip. The Compules® tip may be rotated 360° to gain the proper angle of entrance into the cavity or to the mixing pad.  
**DO NOT USE EXCESSIVE FORCE.**

Indirect use: to prevent Compules® tip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the Compules® tip is handled offside the dental unit with clean/disinfected gloves. Dispense the necessary amount of material onto a mixing pad and protect against light. Apply material into the cavity with a plastic instrument.

- or -

Direct intraoral use: dispense the material directly into the cavity preparation using a slow, steady pressure.

<sup>2</sup> VITAPAN® is a registered trademark of VITA Zahnfabrik H. Rauter GmbH & Co. KG.

- To remove the used Compules® tip, be sure that the Compules® tips gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compules® tip and remove.

**Excessive force.**

Injury.

- Apply slow and steady pressure on the Compules® tips gun.
- Do not use excessive force. Compules® tip rupture or ejection from Compules® tips gun may result.

**Using syringes**

**Avoid contamination of the syringe.**

To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves.

- Remove the cap.
- Turn the handle of the syringe slowly in a clockwise direction and dispense the necessary amount of the material onto a mixing pad.
- Point the front tip of the syringe upwards and turn the handle anti-clockwise to prevent oozing of the material.
- Immediately close the syringe with the cap.
- Protect the restorative material on the mixing pad against light.

**3.5 Light-curing**

- Light-cure each area of the restoration surface with a suitable curing light designed to cure materials containing camphorquinone (CQ) initiator, i.e. spectral output containing 470 nm. Minimum light output must be at least 500 mW/cm<sup>2</sup>. Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.
- Cure each layer according to the table below.

**Insufficient curing.**

Inadequate polymerization.

- Check compatibility of curing light.
- Check curing cycle.
- Check curing output.

**Curing Light Output  $\geq$  500 mW/cm<sup>2</sup>**

Shades	Increment/layer	Curing time
A1; A2; A3; A3,5; B1; Incisal-B1; B2; C2; C3; D3	3 mm	20 sec
Opaque-A2; Opaque-A3,5; A4; B3; C4	2 mm	20 sec

### 3.6 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance® finishing devices for additional finishing.
3. Prior to polishing, check that surface is smooth and free of defects.
4. For achieving a very high luster of the restoration, we recommend Enhance® and PoGo®, finishing and polishing instruments, Prisma® Gloss™ Regular and Extrafine polishing paste.

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for Use.

## 4 Hygiene

### 4.1 Compules® tips gun

#### NOTICE

#### Wrong cleaning or disinfection method.

#### Damage to the device.

1. Always disassemble device before reprocessing.
2. Do not clean or disinfect with aggressive agents (e.g. solutions based on orange oil or acetone).

#### 4.1.1 Disassembly and removal of restorative material

1. Partially close the Compules® tips gun and place your thumb on the rear part of the hinge.
2. Push upwards and lift hinge. The Compules® tips gun is separated into two parts and the plunger is exposed.
3. Remove restorative material, if any, with a soft paper tissue and alcohol of 70% vol.

#### 4.1.2 Manual cleaning

1. Discard gloves according to local regulations.
2. Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.
3. Use a new clean pair of examination gloves.
4. Clean device with a soft brush, warm water, and an appropriate cleaning solution until free of visible contamination. Pay special attention to device seams and insertions.
5. Remove cleaning solution residue with a cloth soaked with water.
6. Dry device with a lint-free single-use cloth.

#### 4.1.3 Manual disinfection

1. After cleaning [4.1.2 Manual cleaning] thoroughly wipe all device surfaces with a single-use cloth in combination with an alcohol-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions.
2. Remove disinfectant solution residue with a cloth soaked with water.
3. Dry device with a lint-free single-use cloth.

#### 4.1.4 Automatic Washer-Disinfector

As an alternative to manual cleaning and disinfection, the device can also be processed in a washer-disinfector<sup>3</sup>. Use only properly maintained, calibrated, and approved washer-disinfector according to ISO 15883-1.

1. Disassemble device.
2. Remove residual adhesive with tissue.
3. Place device in the washer-disinfector allowing water and detergent to enter into and drain off from device openings during washer-disinfection program and follow washer-disinfector's Instructions for Use.
4. Run washer-disinfection program with A0 value  $\geq 3000$  (e.g. 5 min at  $\geq 90$  °C) using appropriate detergents<sup>3</sup>.

#### 4.1.5 Sterilization (optional)

The device can also be autoclaved.

1. Steam autoclave device unwrapped at 134 °C/ 2 bar = 273 °F/ 29 psi, for 3 minutes 30 seconds.

#### 4.1.6 Reassembly

1. After reprocessing insert plunger into the Compules® tips gun cylinder.
2. Press components together and snap hinge mechanism in place.

Do not continue to use damaged or soiled Compules® tips guns.

Compules® tips gun can be reprocessed for up to 100 times.

#### 4.2 Compules® tips – after intraoral use and/or if contaminated

##### **Cross-contamination.**

Infection.

1. Do not reuse Compules® tips after intraoral use and/or if contaminated.
2. Dispose the intraorally used and/or contaminated Compules® tips in accordance with local regulations.

#### 4.3 Individual shade guide tabs

##### **NOTICE**

##### **Cross contamination.**

##### **Infection.**

1. To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder.
2. Always make sure individual tabs are completely reprocessed before placing back into shade guide holder.

<sup>3</sup> For example Miele Washer-Disinfector G7835 CD using program vario TD with neodisher® MediClean (alkaline detergent) and neodisher® Z (acid neutralization and cleaning detergent); both detergents from Dr. Weigert, Hamburg, Germany (no products or registered trademarks of DENTSPLY International, Inc.).

## NOTICE

### Wrong cleaning or disinfection method.

#### Damage to the device.

1. Do not clean or disinfect with aggressive agents (e.g. solutions based on chlorine, glutaraldehyde, phenolic solvents, iodophors, alcohol, orange oil, or acetone) as they may affect the color shade over time.
2. Always make sure individual tabs are completely reprocessed before placing back into shade guide holder.
3. Never autoclave the individual tabs.

#### 4.3.1 Manual cleaning

1. Discard gloves according to local regulations.
2. Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.
3. Use a new clean pair of examination gloves.
4. Clean device with a soft brush, warm water, and an appropriate cleaning solution until free of visible contamination. Pay special attention to device seams and insertions.
5. Remove cleaning solution residue with a cloth soaked with water.
6. Dry device with a lint-free single-use cloth.

#### 4.3.2 Manual disinfection

1. After cleaning [4.3.1 Manual cleaning] thoroughly wipe all device surfaces with a single-use cloth in combination with a water-based, bactericidal, virucidal and fungicidal disinfection solution approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions.
2. Remove disinfectant solution residue with a cloth soaked with water.
3. Dry device with a lint-free single-use cloth.

## 5 Lot number ( ) and expiration date ( )

1. Do not use after expiration date. ISO standard is used: "YYYY-MM"
2. The following numbers should be quoted in all correspondence:
  - Reorder Number
  - Lot number
  - Expiration date

© DENTSPLY DeTREY 2013-08-16

[These Instructions for Use are based on Master Version 06]

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**DENTSPLY DeTrey GmbH**
**DENTSPLY Limited**
**DENTSPLY Italia S.r.l.**