For better dentistry

DENTSPLY

Spectrum[®]
TPH³

Resin-based Dental Restorative

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CAUTION

Cross-contamination

After intraoral use do not clean, disinfect or reuse Compules® tips.

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TPH[®]3

Resin-based Dental Restorative

Spectrum[®]TPH[®]3 – a visible-light activated, radiopaque sub-micron hybrid composite for anterior and posterior restorations. The composite is pre-dosed in Compules[®] tips or delivered in traditional Syringes and is available in a selection of precise Vita^{®1} shades.

CAUTION: For dental use only.

COMPOSITION

- Bis-GMA-adduct (adduct of 2,2-Bis[4-(2-hydroxy-3-methacryloyloxypropoxy)-phenyl]propane with hexamethylene diisocyanate)
- Bis-EMA (2,2-Bis[4-(2-methacryloyloxyethoxy)-phenyl]propane)
- TEGDMA (3,6-Dioxaoctamethylene-dimethacrylate)
- Photo initiators
- Stabilizers
- Bariumaluminiumborosilicate (mean particle size <1 µm)
- Bariumfluoroaluminiumborosilicate (mean particle size < 1 µm)
- Highly dispersed silicon dioxide (particle size 10-20 nm)

INDICATION

Spectrum TPH 3 is indicated for all cavity classes in anterior and posterior teeth.

CONTRAINDICATIONS

Spectrum TPH 3 is contraindicated for use with patients that have a history of severe allergic reaction to methacrylate resins.

WARNINGS

1. Spectrum TPH 3 contains methacrylates (polymerisable monomers) which may be irritating to skin and eyes. Avoid contact with oral tissues, eyes and skin. If accidental contact occurs, flush affected area with generous amounts of water. In case of contact with the eyes, immediately rinse with plenty of water and seek medical attention. After contact with skin, wash immediately with plenty of soap and water.
2. Spectrum TPH 3 may cause sensitisation by skin contact in susceptible persons. If skin sensitisation occurs, discontinue use.

PRECAUTIONS

Not applicable.

INTERACTIONS WITH OTHER DENTAL MATERIALS

Eugenol-containing dental materials should not be used in conjunction with these products because they may interfere with setting and cause softening of the polymeric components of these materials.

¹ Vita is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

ADVERSE REACTIONS

Allergic contact dermatitis and other allergic reactions may occur in susceptible individuals.

DOSAGE AND APPLICATION

Shade Selection

Shade selection should be made prior to the restorative procedure while the teeth are hydrated. Prophylaxis with pumice and water to remove any extraneous plaque or surface stain.

Use the Spectrum TPH 3 shade guide provided with samples of original Spectrum TPH 3 restorative material. The colour coding dot on the shade guide matches the coloured label on the syringe.

Alternatively, a Vita Lumin[®] Vacuum shade guide may be used. The Spectrum TPH 3 shade corresponds to the central part of the respective Vita tooth.

Cavity Preparation

1. Anterior Restorations. Use the cavity preparations for the acid etch technique (bevelled enamel margins), for all Class III, IV and Class V restorations.

2. Posterior Restorations. Cavity design requirements are essentially a conventional preparation with refinement of cavo-surface margins for enhancement of acid-etching.

No residual amalgam or other base material should be left on the internal surfaces of the preparation which would interfere with light transmission and the hardening of the restorative.

Moisture Control

Surface cleanliness is paramount for the development of adhesion. Isolate prepared tooth from contamination with saliva, sulcus fluid, or blood with adequate measures (rubber dam).

Pulp Protection

In deep cavities cover the dentine close to the pulp (less than 1 mm) with a hard-setting calcium hydroxide liner (Dycal[®]) leaving the rest of the cavity surface free for bonding with Prime&Bond[®]NT. Glass-ionomer or other eugenol-free base materials may be used, if wished.

Placement of the Matrix

Use a transparent matrix system with proper wedging for proximal contacts. Pre-wedging is advocated to achieve slight separation and facilitate optimal proximal contact.

Enamel and Dentine Pre-Treatment

It is recommended to use Spectrum TPH 3 in combination with the acid etch technique (DETREY[®] Conditioner 36) and an enamel or enamel/dentine adhesive (Prime&Bond NT). For further details, please see the Directions for Use provided with Prime&Bond NT.

Placement of Spectrum TPH 3

Incremental placement (in 3 mm levels or less) and curing of posterior composite restorations is recommended to minimise polymerisation shrinkage. Cure each increment separately.

IMPORTANT: BE SURE TO EXPOSE THE ENTIRE RESTORATION FOR THE APPROPRIATE TIME.

Curing

Expose each area of the restoration surface to a VLC polymerisation unit (e.g. DENTSPLY SmartLite™) for at least 20 seconds. The composite should be additionally exposed to a VLC polymerisation unit through lingual or buccal enamel walls. For pin build-ups the restorative material should be polymerised around each pin before additional material is added.

Specific exposure times and depths of cure for the different shades are given in the table below:

Depth of Cure

Using DENTSPLY VLC Polymerisation Units for 20 seconds

Shades ²	Depth of Cure
A1, A2, A3, A3.5, B1, Incisal-B1, B2, C2, C3, D3	3 mm
Opaque-A2, Opaque-A3.5, A4, B3, C4	2 mm

Finishing

Begin finishing immediately after curing. Gross excess composite may be removed with fluted finishing burs or diamonds. Additional finishing and polishing is obtained by using Enhance™ Finishing and Polishing Discs. A high final lustre can be obtained on Spectrum TPH 3 by applying PoGo® or Prisma®-Gloss™ or Prisma-Gloss Extrafine.

STORAGE

Do not store above 24 °C.

Keep out of direct sunlight.

Replace cap upon extrusion of composite to protect the contents from light.

BATCH NUMBER () AND EXPIRY DATE ()

The batch number should be quoted in all correspondence, which requires identification of the product.

Do not use after expiry date.

If you have any questions, please contact:

Manufacturer:
DENTSPLY DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
GERMANY
Phone +49 (0) 75 31 5 83-0

Distributor:
DENTSPLY Limited
Building 1
Aviator Park, Station Road
Addlestone, KT15 2PG
Phone (0 19 32) 85 34 22

© DENTSPLY DeTrey 2006-04

² Approximate shade match to Vita (body).

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DENTSPLY DeTREY GmbH