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INSTRUCTIONS FOR USE

EN

LIGHT CURE RESIN MODIFIED GLASS IONOMER REF. 35221 CONVENTIONAL (SELF-CURE) GLASS IONOMER REF. 35220

RECOMMENDED INDICATIONS (Light Cure Resin Modified Glass Ionomer)

- Deciduous cavity restoration.
- Class V/cervical lesions and small single surface class I or II cavities in adult teeth.
- Glass ionomer/composite "sandwich" restorations.
- ART or minimally invasive procedures.

RECOMMENDED INDICATIONS (Conventional Glass Ionomer)

- Class V cavities (particularly erosion lesions).
- Class III cavities.
- Posterior cavities in deciduous teeth and selected permanent teeth.
- "Sticky" pits and fissures.
- Core build-up.
- Minimal Intervention (MI) treatment.

CONTRA-INDICATIONS (Light Cure Resin Modified Glass Ionomer only)

- Pulp capping - use calcium hydroxide to line cavities where remaining dentin may be thin. Do not use eugenol-containing materials.
- In rare instances, some people may exhibit sensitivity to resins. If any such reactions are experienced, discontinue use of product and seek medical assistance.

APPLICATION

	Light Cure Resin Modified Glass Ionomer Restorative	Glass Ionomer Restorative
Powder: Liquid ratio	1 scoop: 1 drop 0,90 g; 0,20 g	1 scoop: 1 drop m.m 1,30 g; 0,50 g
Mixing time	20 seconds	30 seconds
Light Curing time (halogen, LED)	20 seconds	Not applicable
Working time in absence of light curing (at 23±1°C)	1 minute 30 seconds minimum	1 minutes 30 seconds minimum
Setting time in absence of light curing (at 37±1°C)	6 minutes 0 seconds maximum	4 minutes 30 seconds maximum

WARNUNG

Avoid contact of powder or liquid with the eyes. In case of contact, wash thoroughly with large quantities of water and obtain medical advice.

LOT STORAGE

Store in a cool, dry place away from light and heat (5-25°C). Close caps immediately after use. If desiccant becomes pink, discard the product.

BATCH CODE

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

The expiry date is shown in year, month format. Do not use the product after this date.

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