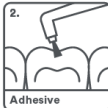
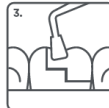
FICHA TÉCNICA

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TECHNICAL DATA SHEET

PROCLINIC EXPERT NANO COMPOSITE		
REFERENCES	According to Annex 1	
MANUFACTURER	SDI LIMITED 3-15 Brunson street Bayswater victoria 3153 Australia	
CLASSIFICATION	Medical Device, Class IIa, Directive 93/42/EEC	
DESCRIPTION	Combining nano-hybrid and micro filler particle technology together with non-stick handling, the radiopaque light cured, condensable and low shrinkage composite offers polishability and strength on an unprecedented scale for anterior and posterior restorations alike.	
COMPOSITION	Nano: 22.5% wt, multifunctional methacrylic ester. 77.5% wt, inorganic filler (40 nm – 1.5 micron). Nano Flow: 40% wt multifunctional methacrylic ester. 60% wt inorganic filler	
PACKAGING	Package with 1 syringe, 4 g Kit: 5x4 g syringes (A1, A2, A3, A3.5, B2), 1x5ml total etch dental adhesive, 2x2 ml etching gel, 40 microapplicators.	
SHELF LIFE	Nano: 5 years / Nano Flow: 3 years	
INTENDED USE	For use only by dentists. Direct anterior/posterior restorations for Class I, II, III, IV & V. Veneers. Core build ups. Inlays and onlays. Splinting. Composite and porcelain repairs. Sandwich technique with glass ionomers.	
INSTRUCTIONS FOR USE	<div style="display: flex; justify-content: space-around; align-items: center;">      </div> <p>1. Isolate and prepare tooth. 2. Apply a bonding agent as per manufacturer's instructions. 3. Apply Proclinic Nano Hybrid Composite in increment of 2 mm. 4. Light cure each increment for 20 seconds minimum. 5. Finish and polish using standard techniques.</p>	
PRECAUTIONS	Avoid prolonged contact with skin/soft tissues as it may cause inflammation or sensitisation. Discontinue use if irritation occurs or if have known acrylic resin allergies. When not in use, replace cap.	
CONTRAINDICATIONS	Pulp capping. Product may cause skin irritations to some people. In such cases, discontinue use and seek medical attention. Do not use in conjunction with any eugenol-containing materials.	
HANDLING AND STORAGE CONDITIONS	Store in a cool dry place between 10 and 25°C, away from direct sunlight. Protect from moisture. Keep out of the reach of children	

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