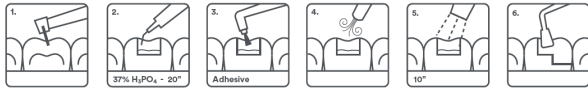
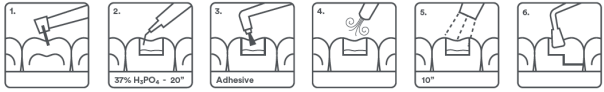
**FICHA TÉCNICA**

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**TECHNICAL DATA SHEET**

<b>DENTAL ADHESIVE TOTAL ETCH PROCLINIC EXPERT</b>		
<b>REFERENCES</b>	36599: 1 unit 36598: 2 units	
<b>MANUFACTURER</b>	CHL MEDICAL SOLUTIONS, Srl Via Felice Cavallotti, 15 20122 Milano - Italy	
<b>CLASSIFICATION</b>	Medical Device, Class IIa, Directive 93/42/EEC	
<b>DESCRIPTION</b>	Fluoride releasing single component dentin/enamel total - etch adhesive system, designed for direct bonding. The product completely and homogeneously infiltrates the hybrid layer to ensure superior bonding to the tooth.	
<b>COMPOSITION</b>	Acrylic monomer, Acetone, Fluoride, Stabilizer.	
<b>PACKAGING</b>	Package with 5ml bottle	
<b>SHELF LIFE</b>	3 years	
<b>INTENDED USE</b>	For use only by dentists. I) All direct restorative composites. II) Compomers. III) Composite, ceramic, metal, porcelain repairs. Bonding surface requires pretreatment with a silane primer.	
<b>INSTRUCTIONS FOR USE</b>	 <ol style="list-style-type: none"> <li>1. Isolate and prepare tooth.</li> <li>2. Apply Etch as per manufacturer's instructions. Wash thoroughly. Remove excess water. Keep moist.</li> <li>3. Apply the product to saturate all internal surfaces.</li> <li>4. Blow gently with dry, oil free air for 2 s to evaporate solvent. Leave surface glossy.</li> <li>5. Light cure for 10 seconds.</li> <li>6. Apply composite as per manufacturer's instructions.</li> </ol>	
<b>PRECAUTIONS</b>	Avoid prolonged contact with skin/soft tissues as it may cause inflammation or sensitisation. Discontinue use if irritation occurs or if have known acrylic resin allergies. When not in use, replace cap.	
<b>HANDLING AND STORAGE CONDITIONS</b>	Store in a cool dry place between 10 and 25°C, away from direct sunlight. Protect from moisture. Keep out of the reach of children	