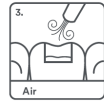
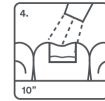
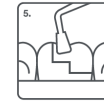
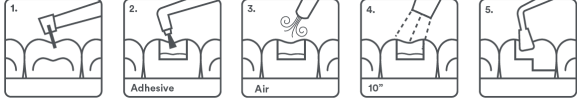
**FICHA TÉCNICA**

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**TECHNICAL DATA SHEET**

<b>NANO DENTAL ADHESIVE</b> <b>SELF-ETCH DENTAL ADHESIVE</b> <b>PROCLINIC EXPERT</b>		
<b>REFERENCE</b>	36600	
<b>MANUFACTURER</b>	CHL MEDICAL SOLUTIONS, Srl Via Felice Cavallotti, 15 20122 Milano - Italy	
<b>CLASSIFICATION</b>	Medical Device, Class IIa, Directive 93/42/EEC	
<b>DESCRIPTION</b>	One component, self-etching, fluoride releasing, light cured adhesive for use on both dentin and enamel.	
<b>COMPOSITION</b>	Phosphoric acid ester monomer, Dimethacrylate monomer, Monomethacrylate monomer, Silicon dioxide Filler, Water, Acetone, Photoinitiators, Stabilizer, Sodium Fluoride.	
<b>PACKAGING</b>	Package with 5ml bottle	
<b>SHELF LIFE</b>	2 years	
<b>INTENDED USE</b>	For use only by dentists. I) Direct filling restorations using light-cured composite and compomers. II) Composite repair of porcelain and hybrid ceramic restorations. III) Sealing exposed root surfaces causing hypersensitivity. IV) Cavity sealing prior to placement of indirect restorations. V) Core build-ups using light cure or dual cure composites. VI) Composite repair.	
<b>INSTRUCTIONS FOR USE</b>	 <ol style="list-style-type: none"> <li>1. Isolate and prepare tooth.</li> <li>2. Apply the product to saturate all internal surfaces and leave in place for 20 seconds.</li> <li>3. Blow firmly with dry, oil free air for 5 seconds. Leave surface glossy.</li> <li>4. Light cure for 10 seconds.</li> <li>5. Apply composite as per manufacturer's instructions.</li> </ol>	
<b>PRECAUTIONS</b>	Avoid prolonged contact with skin/soft tissues as it may cause inflammation or sensitisation. Discontinue use if irritation occurs or if have known acrylic resin allergies. When not in use, replace cap.	
<b>HANDLING AND STORAGE CONDITIONS</b>	Keep refrigerated until use. Allow product to reach room temperature just before use. Keep out of the reach of children	