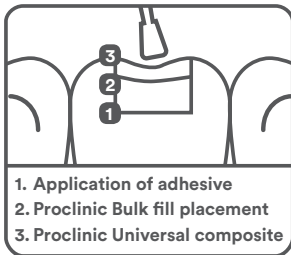
Composite restorative material

BULK FILL

1 syringe
4 g

- DE Bulk Fill Komposit.
- EN Bulk Fill Composite.
- ES Composite Bulk Fill.
- FR Composite Bulk Fill.
- IT Composito Bulk Fill.
- PL Bulk Fill Kompozyt.
- PT Composito Bulk Fill.

DEUTSCH:

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ENGLISH:

1. Isolate tooth. Prepare cavity using standard technique for composite resin restorations. Note: In cases where pulp protection is necessary, use a calcium hydroxide liner. 2. Apply a dentine / enamel bonding agent as per manufacturer's instructions. 3. Place Bulk Fill composite. 4. Light cure for 2 x 20 sec / 5 mm using a LED curing light or any other light curing device (470nm wavelength). 5. Finish restoration using standard technique. Precautions: 1. Avoid prolonged contact with skin / soft tissues as it may cause inflammation / sensitisation. 2. Discontinue use if irritation occurs or if have known acrylic resin allergies. 3. Use at room temperature. 4. When not in use, replace cap.

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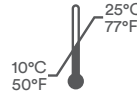
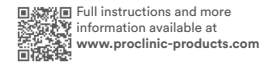
INDICATIONS: Posterior restorations. Core build ups. Splinting. Sandwich technique with glass ionomer.

FIRST AID:

Eye: Flush with water. Seek medical attention if symptoms persist. Skin: Remove with cloth soaked in alcohol, wash with water. Ingestion: Drink plenty of water/milk. Seek medical attention. Inhalation: No symptoms expected.

CAUTION: KEEP OUT OF REACH OF CHILDREN. FOR USE BY DENTISTS ONLY.

Ultra universal restorative material. Conforms to ISO 4049

CHL MEDICAL SOLUTIONS, SRL
Via Felice Cavallotti, 15
20122 Milano (Italy)