# Pressure Spot Indicator

Gebruksinformasjon	DE
--------------------	----

**Definition**

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**Definition**

Coltène Pressure Spot Indicator is a silicone-based pressure spot material.

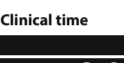
**Material type**

Polysiloxane, condensation-type silicone elastomer
Type 2, medium consistency
Base: white
Catalyst: green

**Indications**

- Recognition of pressure points on new or existing partial and total dentures
- Determining the need for relining
- Testing of the exact fitting of inlays, crowns, bridges and cast brackets

**Clinical time**

<b>≤ 2:00 min</b>	<b>≥ 5:00 min</b>
	
<b>≤0:30 min</b>	<b>≤1:30 min</b>
<b>23 °C / 73 °F</b>	<b>≥3:00 min</b>

**Contra-indications**

Provided that the product is used as directed, there are no known contra-indications.

**Side effects / Interactions**

Polysiloxanes have very good biological compatibility and up to now no harmful reactions o secondary effects on patients and/or dental personnel are known. Impression materials are intended to set in the mouth of the patient, however, they should remain in the mouth not more than twice the setting time. Although they have reasonably high tear

strength, care should be taken that no portion of the impression material remains in the interdental spaces or in the sulcus. Undercuts should, in certain instances, be blocked out before taking the impression.

**In case of contact of the catalyst with the eyes, wash out with plenty of water and then consult the ophthalmologist.**

**Dosage**

Squeeze out equal strand lengths of base material and catalyst onto mixing pad. Always dose catalyst first for better results.

**Dosage ratio by volume:**

10 : 1,4

**Dosage ratio by weight:**

10 : 1,4

**Close tube immediately after use!**

**Mixing**

- Take up one component with spatula and spread evenly on the other component. Do not mix.
- First take up half of the mass in one go with the spatula.
- Then take up the other half of the mass in the same way.

4. Using the flat part of the spatula, spread the mass on the mixing pad, spreading out to right and left with a little pressure. Repeat steps 2–4 until a uniform shade is achieved.

**Impression**

Coat interior surface and insert restoration. Hold in situ until the material has set completely. Remove restoration and indicate the pressure points with the enclosed marking pen (= interrupted layer). Remove material and correct marked surfaces by grinding. Material residue needs considerably more time to set at room temperature. You should therefore check intraorally if the material is set before removing from the mouth. Intensive mixing, high temperatures or overdosage of the catalyst speed up the setting; low temperatures and underdosage of the catalyst slow down the setting.

**Shelf life and storage**

Coltène Pressure Spot Indicator fulfills the intended purpose at least until the expiry date, which is displayed on the containers, in well sealed tubes at temperatures of 15–23 °C/59–73 °F and 50% relative air humidity.

**Marking**

The expiry date and 🔗<sup>ⓘ</sup>-number are shown on the package and cartridge.

**Technical data ISO 4823**

Measurements are based on 23 °C/73 °F room temperature and 50% relative humidity.

**Mixing time:**

30 s

**Total working time:**

135 s

**Oral setting time:**

180 s

**Instructions for use date**

01-2018

Only supplied to dentists and dental laboratories or upon their instructions.

**Caution**

Federal law restricts this device to sale by or on the order of a dentist.

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