## R-070170 / R-070180

E.....	REVELADORAS MANUALES .....	2
GB ....	MANUAL DEVELOPERS .....	5
F .....	DEVELOPPEUSE MESTRA .....	7
P .....	MÁQUINAS DE REVELAR MANUAIS.....	9

**MESTRA<sup>®</sup>**

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tfno. + 34 94 453 03 88 Fax + 34 94 471 17 25

E-mail: [mestra@mestra.es](mailto:mestra@mestra.es) - [www.mestra.es](http://www.mestra.es)

**48150 SONDIKA - BILBAO - ESPAÑA**

Rev. 25/04/06

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R-070170 and R-070180 are two practical developing chambers specifically designed for clinical or laboratory applications. R-070180 features a very convenient X-ray viewer which also provides light inside the developing chamber. To achieve optimum performance levels, proceed as follows:

## LOCATION

1. Remove packagmg materials with care. Inside the chamber you will the following components:
  - 1 collet holder bar (only R-070180).
  - 2 plate holder collets.
  - 2 cup holder strips.
  - 3 cups (red, blue and yellow).
2. Avoid placing the unit in the clinic or laboratory under direct sunlight. We recommend that the machine be placed m a room with subdued lighting.
3. Place the unit on a fiat surface, far away from heat and vibration sources. Select a comfortable working height to handle plates inside the chamber.
4. In the case of the R-070180 unit, place it close to a 220 V, 50/60 Hz ground power socket. Plug the X-ray viewer in to the mains.

## HOW TO USE THE DEVELOPER

1. Fill the three cups with developing, fixing and washing fluids. Place the cups on the cup holder inside the chamber.
2. Insert the X-ray you wish to develop inside the chamber and proceed accordingly. The transparent plastic lid allows to you to see inside the chamber. In the case of the R-070180 unit, it is possible to use the X-ray viewer to light up the interior of the chamber.
3. Slip you hands into the wool sleeves located on the front of the machine to manipulate the plates inside. X-ray pincers (supplied with the unit) can also be used during the developing process.
4. Remove the X-rays from inside the chamber. In the case of the R-070180, an X-ray viewer can be used.

## MAINTENANCE AND CLEANSING

Thoroughly clean the unit as detailed hereunder to achieve optimum X-ray viewer performance levels:

## **English**

---

1. Remove the top lid of the unit by detaching the two side hinges.
2. Remove the wool sleeves by releasing the fixing flanges.
3. Withdraw the holder plate plus the three cups from inside the chamber. Wash the unit with soap and water in a sink.
4. Use the same washing process for the cups and holder plate (only available in R-070170).
5. The wool sleeves are then removed from the flange once the rubber band is released. Once ready, conventional washing methods can also be used. To assemble the sleeve, proceed in reverse order. Make sure that the rubber band is folded twice over to ensure optimum tightness.
6. If the wool sleeves are in poor condition, contact your MESTRA dealer to order a spare set.

Sleeve fixation flanges are pressure-fit to simplify the dismantling process when the wool sleeves have to be cleansed. Should it be difficult to perform (i.e., dismantling sleeves during X-ray manipulation, unexpected flange shifts, etc) secure the flanges to the developing housing with a few drops of conventional plastic-to-plastic (R-070170) or plastic-to-metal (R-070180) glue.

## **WARNING**

- ◆ Plug the X-ray viewer (only R-070180) into a 220 V, 50/60 Hz ground connection.
- ◆ Always avoid direct sunlight in the clinic or laboratory.
- ◆ Prevent accidental spillages of processing fluids (developer, fixing agent and washing agent) inside the unit.
- ◆ Do not allow children or unqualified staff to use processing fluids and/or resulting waste.
- ◆ Never forget environmental hazards. Once used in full, dispose of processing fluids following the manufacturer's instructions.
- ◆ Wash the unit periodically as specified in this manual.

## **TECHNICAL CHARACTERISTICS**

	<b>R-070170</b>	<b>R-070180</b>
Size:	230 x 360 x 210	225 x 355 x 270
Voltage:	Not available	220 V, 50/60 Hz
Housing material:	Transparent plastic	Stainless steel & plastic
Number of cups:	3	3

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