R-100095

E	LUPA DE AUMENTO BINOCULAR	4
GB	BINOCULAR MAGNIFYING GLASSES	6
F	LOUPE BINOCULAIRE	8
D	STEREO-LUPENBRILLE	10
P	LENTE DE AUMENTO BINOCULAR	12
I	LENTE DI INGRANDIMENTO BINOCULARE	14

MESTRA[®]

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tfno. + 34 94 453 03 88 Fax + 34 94 471 17 25

E-mail: mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

Rev. 14/11/06

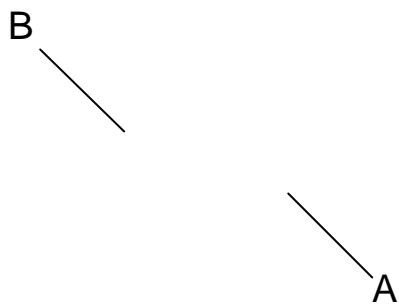
Fig. 1
Abb. 1

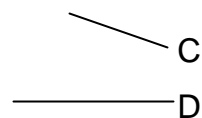
Fig. 2
Abb. 2

Fig. 4
Abb. 4

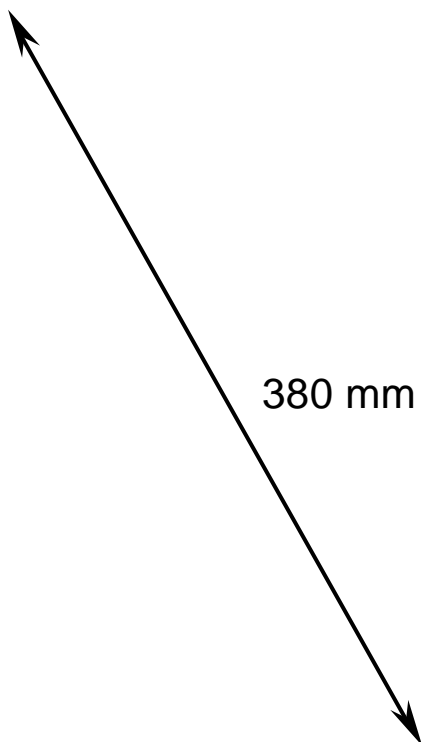
Fig. 3
Abb. 3

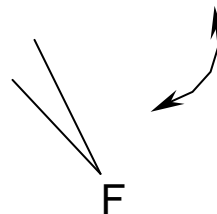
Fig. 5
Abb. 5

R-100095

- A Cristales / **Glasses** / Verres / Gläser / **Vidros** / *Lenti occhiali*
- B Protectores laterales / **Side shields** / *Protections latérales* / Seitenschutz / **Protectores laterais** / *Protettori laterali*
- C Lentes / **Lenses** / Verres / Okulare / **Lentes** / *Lenti*
- D Protectores lentes / **Lens protectors** / *Protections verres* / Okularschutzgläser / **Protectores lentes** / *Protettori lenti*
- E Tornillo distancia interpupilar / **Interpupillary distance adjustment screw** / *Vis* / Stellschraube Pupillendistanz / **Parafuso distancia interpupilar** / *Vite distanza interpupillare*
- F Articulación / **Joint** / *Articulation* / Gelenk / **Articulação** / *Snodo*

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Indispensable in the dental clinic and dental lab. In the clinic for exploration, diagnosis, minor surgery, etc. In the lab for precision works requiring greater visual sharpness.

Their main features are:

- High brightness optics, achieving outstanding clarity and sharpness.
- 2,5x magnification. Ideal for dental works.
- Working distance of 380 mm, together with an ideal depth of field.
- Side shields for protection against splashes and projections.
- Steel framed supporting goggles. Their glasses can be replaced for corrective glasses according to the user's needs.
- Comfortable and ergonomic nose-bridge.
- Interpupillary distance adjustment screw.
- Adjustable neck strap.
- Supplied with a practical black vinyl case and a screwdriver.

USE (see figures in previous pages)

1. If needed, the goggle's glasses (A) can be replaced for corrective glasses. Consult your optician.
2. Depending on the kind of work to be performed, use the side shields (B) for protection against splashes and projections.
3. The lenses are fitted with lens protectors (D), removable for cleaning. The lenses (C) should never be touched.
4. The ideal working distance is 380 mm (maximum sharpness) (Fig. 3).
5. Adjust the interpupillary distance by turning the central screw (E) until achieving a single vision.
6. The alignment of the lenses can be adjusted when needed.
7. The lenses can be easily folded upwards when they are not needed for the work being performed (Fig. 5).
8. If you need to tighten or loosen the joint (G), use the screwdriver supplied.

TECHNICAL SPECIFICATIONS

Dimensions of the case:	250 x 135 x 75 mm
Total weight of the case:	310 g
Weight of the glasses:	100 g
Focus distance:	380 mm
Magnification:	2.5 x

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