Prior to use, carefully read the instructions for use.

EN

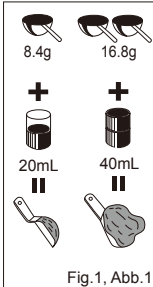
Fig. 1, Abb. 1

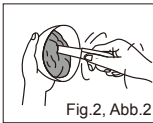
Fig. 2, Abb. 2

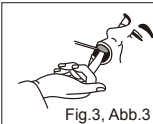
Fig. 3, Abb. 3

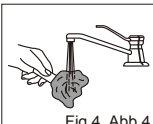
Fig. 4, Abb. 4

Fig. 5, Abb. 5

AROMA FINE PLUS

ALGINATE IMPRESSION MATERIAL

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

Preliminary impressions.

Impressions for the fabrication of:

- models for temporary inlays, onlays, crowns and bridges
- orthodontic models
- study models

CONTRAINDICATIONS

In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

	Normal set	Fast set
Mixing time	20-30 sec.	20-30 sec.
Total working time	90 sec.	70 sec.
Setting time	180 sec.	150 sec.
Minimum time in mouth	120 sec.	90 sec.

1. Loosen the powder well before every use. Take the necessary amount of powder and water. Standard powder to water ratio is 1 spoon (8.4g) of powder and 1/2 cup (20mL) of water (Fig. 1).
2. Mix for 20-30 seconds. Load the tray with the mixed material (Fig. 2).
3. Seat the loaded tray in the mouth. Refer to above table for recommended total working times (Fig. 3).
4. Wait for the material to set. Refer to above table for the minimum time in mouth.
5. Remove impression and rinse under running tap water (Fig. 4).
Note:
1. The setting time includes the mixing time and the minimum manipulation time. This is based on the mixing at 23°C with distilled or deionized water.
2. Higher temperatures of the mixing water will shorten the setting- and total working time. Variations in hardness of the water may influence the setting time.

DISINFECTION

The recommended disinfection method is to soak in a 0.5% sodium hypochlorite solution for 15 minutes. Refer to the respective manufacturer's instructions for use.

MODEL FABRICATION

1. Remove excess water before pouring cast. For the most accurate result, pour cast immediately after rinsing (Fig. 5).
2. If the impression cannot be poured immediately, wrap the impression in a wet paper towel or store the wet impression in a tightly sealed plastic bag. DO NOT STORE IN WATER.
3. AROMA FINE PLUS is compatible with type III and type IV stone (GC Fuji ROCK™ EP).

STORAGE

Store in a cool dry place, in a tightly closed container (4-25°C) (39.2-77.0°F). (Shelf life: 3 years from date of manufacture)

CAUTIONS

1. In case of contact of material or mixture with eyes, immediately flush with water and seek medical attention.
2. When using material, wear a dust mask to avoid inhaling powder dust.
3. After opening, transfer the contents into a container for exclusive use of the material.
4. Do not mix with any other product.

Last revised : 05/2011

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

GC ASIA DENTAL PTE. LTD.
19 Loyang Way, #06-27 Singapore 508724
TEL: +65 6546 7588

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