COE-COMFORT™

EDENTULOUS TISSUE CONDITIONER

For use only by a dentist or physician in the recommended indications.

RECOMMENDED INDICATIONS

Lining of ill-fitting dentures as a tissue conditioner to facilitate recovery of damaged soft tissue. Functional impression material for full dentures, relining the edentulous area of distal extension partials and construction of obturators. Lining of surgical and periodontal splints. Diagnostic aid to locate pressure areas on dentures. Liner to stabilize base tray during recording of jaw relations.

CONTRAINDICATIONS

Patients who have shown sensitivity to the material. In case of allergy, please refer to physician.

DIRECTIONS FOR USE

Product Handling:
Use caution when opening. COE-COMFORT Liquid is flammable; keep away from heat, sparks and flames. Use with adequate ventilation. Do not ingest or breathe vapor. Personal Protective Equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Check bottle with liquid before using. Make sure that upon removal of cap, the bottle opening has no obstructions. Some bottles may contain an insert (as a seal). The insert must be in the cap and not obstructing opening.

In case of contact, flush eyes with water for at least 15 minutes. Get prompt medical attention. If overcome by vapor, move to fresh air. If breathing is difficult, administer oxygen and seek medical attention. If swallowed, dilute by giving water to drink and seek medical attention. In case of contact with skin, wash thoroughly with soap and water. For more information, read the Safety Data Sheet (SDS).

Tissue Conditioning:

I. Pre-treatment considerations:
The restoration of sound condition to damaged soft tissue takes time, requiring several applications of COE-COMFORT. It is recommended that before starting treatment and at each subsequent visit an impression is taken and a study model poured. Comparison of such casts will demonstrate how well treatment is progressing. If the existing dentures do not provide good tissue coverage and are not in proper centric and vertical relationship, they should be rebased and corrected before beginning treatment. If tissues beneath both upper and lower dentures need treatment consider separate treatments. If not practical, then prepare the upper first and seat it. Then prepare the lower and seat it. Use separate mixes of COE-COMFORT.

II. Preparation of the denture:

Using a handpiece, remove undercuts to relieve the denture to prevent impingement. Wash and dry the denture thoroughly. Coat buccal and labial flanges with COE LUBRICANT to prevent adhesion of COE-COMFORT.

III. Mixing:

Dispense 5mL of COE-COMFORT Liquid into a disposable mixing cup. Fill the red measure level with COE-COMFORT Powder (6g) and add slowly to the liquid. Mix thoroughly for 30 seconds. Set aside the mixture for approximately 1 minute until it flows sluggishly from the spatula.

Note:
Using less liquid gives a thicker, faster setting mix and vice versa.

IV. Lining the denture:

Spread the mixture evenly on the surface to be coated, including the peripheral borders. During this time, ask the patient to rinse the mouth thoroughly with cold water. The reduced temperature of the mouth provides a more favorable working time. Insert the denture if taking an impression. Instruct the patient to close VERY GENTLY into occlusion. After the denture has been secured, encourage the patient to talk for a few minutes thus stimulating the tissues. After four or five minutes, remove the denture and rinse thoroughly in cold water. Trim away excess material. Inspect the contact area of the denture for areas of contact impingement where the pink denture resin shows through the white COE-COMFORT. In such cases, relieve the area with a handpiece and wash and dry the denture fully. Make up a small quantity of COE-COMFORT mixture and apply to the freshly relieved area. Instruct the patient to rinse with cold water, reinsert the denture and tell the patient to close VERY GENTLY into occlusion.

V. Further treatment:

Full restoration of tissue to a sound condition takes time and several repeat applications of COE-COMFORT may be necessary. Ask the patient to return 2-4 days after the initial treatment, and each subsequent treatment. At each visit, remove all the COE-COMFORT from the denture using a handpiece and place a new COE-COMFORT lining as described above. Two or three treatments may be sufficient, but continue as often as necessary until you judge that further treatment will bring no further improvement. At this stage, reline the denture using a hard chairside reline material.

OTHER INDICATIONS

In other indications, the same basic procedure as used for tissue conditioning are employed, being simply adapted to the particular application.

STORAGE

Recommended for optimal performance; store in a dry location at room temperature (70° to 77°F; 21° to 25°C).

PACKAGES

COE-COMFORT Professional Package
COE-COMFORT Powder, 6oz (170g)
COE-COMFORT Liquid, 6fl oz (177ml)

CLEANING AND DISINFECTION RECOMMENDATION

MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients the bottles and measuring devices require mid-level disinfection. Immediately after use inspect the bottles, measuring devices and label for deterioration. Discard if damaged. DO NOT IMMERSE. Thoroughly clean bottles and measuring devices to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at: <http://www.gceurope.com> or for the Americas: <http://www.gcamerica.com>

They can also be obtained from your supplier.

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