DESOCCLUSOL ORANGE

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(EN)

DESOCCLUSOL ORANGE

1 - PRESENTATION¹: Bottle of 14ml. Multi-use bottle.

2 - INDICATION²: Solvent for eugenate root canal filling cements (zinc oxide/eugenol + gutta-percha).

3 - CONTRAINDICATION³: Known allergy to Orange Essential Oil. Do not use in children or pregnant or breastfeeding women.

4 - COMPOSITION⁴: Orange essential oil.

5 - PRECAUTIONS FOR USE⁵: **DANGER**. Flam Liq 3. Flammable liquid and vapour. Asp. Tox. May be fatal if swallowed and enters airways. Skin Irrit 2. May cause skin irritation. Skin sens 1. May cause an allergic skin reaction. Aquatic chronic 1. Very toxic to aquatic life with long lasting effects. Keep away from heat/sparks/open flames/hot surfaces. — No smoking. Wear protective gloves/protective clothing/eye protection/face protection. Avoid release to the environment. IF ON SKIN: Wash with plenty of soap and water. Keep out of reach of children. Do not use if there is any change in the appearance of the product (color, odor, ...). Glass bottle: handle with care. Do not collect multiple times in the flacon with the same pipette or reuse the solution deposited in a cup for different patients: risk of inter-patient contamination. Do not mix with other products. Do not swallow. Do not breathe the product.

6 - INSTRUCTIONS FOR USE⁶: The tooth to be treated must be isolated by a dam. Clean mechanically as well as possible: the access cavity, the pulp chamber, and the canal entrance; dry. Collect with a single-use disposable pipette or place the product in a cup. Place using a pipette or a cotton swab, a sufficient quantity of liquid in the pulp chamber to wet it completely (1 to 2 drops) but not to overflow. Leave for 5 to 10 minutes for optimum effect: the prolonged contact of DESOCCLUSOL ORANGE with the cement makes the work easier. Find the canal entrance by using a probe. Start removing the filling using a manual or rotary file forcing DESOCCLUSOL ORANGE towards the cement. Start forcing along the first few millimetres carefully. Progressively add DESOCCLUSOL ORANGE in accordance with its penetration in the cement in order to dilute it more and more. Remove the instrument occasionally to wipe it using a sterile compress. This facilitates the removal of the most thickened parts of the solution and to clear the instrument's coils from the debris covering it. The quantities of liquid thus removed are replaced by DESOCCLUSOL ORANGE. Once the filling removed, finish by careful mechanical cleaning, dry with air and with sterile paper points.

7 - STORAGE⁷: Close well after use. Store at a temperature below + 25°C in a well ventilated area, protected from light and all sources of ignition or sparks. Do not store in a refrigerator. Expiry date: 2 years from the manufacturing date. Do not use past the expiry date shown on the external packaging.

8 - DISPOSAL⁸: Recycle or dispose of according to current legislations, rather by a waste collector or an approved company. Do not dispose of in the environment. Professional dental product.

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