
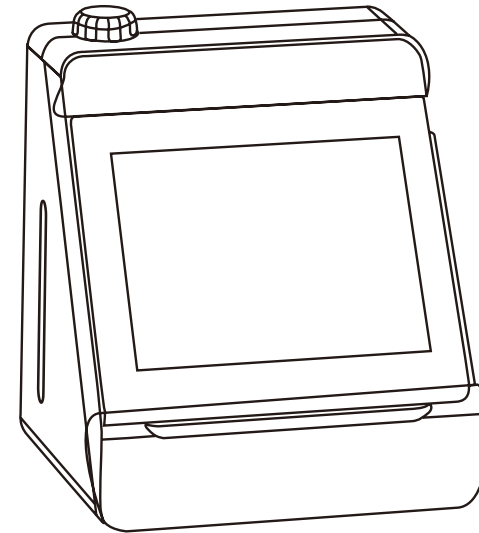
VELOCE

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Quantity: 1 set
N.W.: 7.2 kg
G.W.: 10.5 kg
L.W.H.: 41x40x45 cm

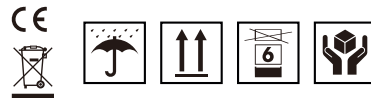 Ningbo Rundeer Control Equipment Co., Ltd.
No.456, Tonghui Road, Jiangbei Investment & Pioneering
Park C, Ningbo, 315033, China.
Tel: +86-574-27709922 Fax: +86-574-27709923

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ENGLISH

Foreword

Thank you very much to choose our Handpiece Lubricating Machine. The device can clean and lubricate the handpieces very easily and safely. In order to give full play to machine's function and use the device effectively and safely, please read this user manual carefully before using the device. And keep the user manual well, so that you can check at any time.

User Records

You will find the product name, model number and serial number from the label which is stuck at the back side of the device. Please check and record the serial number in below blank space. Please advise the serial number when you try to contact with your local distributor.

Product Name: Handpiece Lubricating Machine

Model Number: LUB-120

Serial Number: _____

Contents

I. Basic Characteristics.....	4
II. Parameter.....	5
III. Accessory Name.....	5
IV. Installation Of Components.....	6
V. Operation Steps.....	9
VI. Maintenance.....	13
VII. Troubleshooting.....	15
VIII. Warranty.....	15

Our company is responsible for device's safety, reliability and performance, within the following conditions.

1. The device's installation, debugging and after service work should be done by professional personnel approved by our company or personnel who obtains national corresponding technical level.

2. The power supply of the device should comply with related national regulations and device's usage requirement.

3. The device's operation should be done by qualified or approved personnel, the operator should have the professional medical application skills and operate based on this user manual.

Safety Warning

1) Please read below safety clause before you correctly use and operate this handpiece lubricating machine LUB-120

2) The following instructions can guide you on how to use the device safely, to avoid harms to you and other people caused by improper operation. All the marks are classified according to different levels of danger, please operate according to below instructions.

Danger Precautions	to remind you about the improper operations which may result in death and serious injury
Warning Precautions	to remind you about the improper operation which may result in physical injury and machine damages.
Safety Precautions	to remind you about the improper operation which may result in physical injury and machine damages

Danger Precautions

1) Prohibit to install, use and fill the product with oil in explosive space or in the place where is close to the fire source.

2) Prohibit to install and use the device under the direct sunlight or in the environment where the temperature is over 40°C(104 Fahrenheit).

3) Prohibit to use the device in other uses that are not mentioned in this user manual, except the cleaning and lubricating use purpose. Please keep the device in the ventilated environment regularly, if there is peculiar smell inside the device, please make it ventilation.

Warning Precautions

1) Don't touch the power cable with wet hands to avoid electric shock.

2) Don't pour the water onto the machine to avoid electric shock caused by fire disaster or short circuit.

3) Don't disassemble or change the device. Please contact with the local distributor if you need after service.

4) If there is smoke or smell of burning resin coming from the machine, please turn off the electricity and unplug the socket immediately, and at the same time contact with the local distributor.

5) Don't make the device drop from the height, and put it on the horizontal surface.

Safety Precautions

1) Don't touch the power cable with wet hands to avoid electric shock.

2) Don't pour the water onto the machine to avoid electric shock caused by fire disaster or short circuit.

3) Don't disassemble or change the device. Please contact with the local distributor if you need after service.

4) If there is smoke or smell of burning resin coming from the machine, please turn off the electricity and unplug the socket immediately, and at the same time contact with the local distributor.

5) Don't make the device drop from the height, and put it on the horizontal surface.

6) After use, please turn off the power and cut off the air supply.

7) The air supply (such as air compressor) and air filter supplied to the machine should be kept clean and dry. The moisture in air compressor and air filter should be drained regularly, because the moisture in the cavity mixed with the maintenance oil will cause harm to the machine.

8) If the equipment will be left without use for a long time, please remove the air pipe and power cord.

9) When storing and maintaining lubricating oil, please keep it away from oxidants, flammable substances and strong acid chemicals as much as possible

10). Please purchase substitutes specified by our company, such as lubricating oil, etc. Any unspecified parts may cause damage and malfunction of the machine.

I. Basic Characteristics

1. Ensure the cleaning and oiling of the handpiece;

2. Available to clean and oil three handpiece at the same time;

3. The rotating gear in our handpiece can effectively clean and lubricate from different angles.

4. After completing the cleaning and lubricating steps, pressing the air compartment can remove the excess residue in the handpiece.

5. Choose short mode, long mode and extra-long mode to lubricate according to different handpiece.

6. Oil mist filter cotton can prevent the diffusion of oil mist to a minimum.

II. Parameter

1. Model: LUB-120
2. Rated voltage: 100V ~ 240V ~; 50/60
3. Output power: 60VA
4. Working pressure: 0.4 ~ 0.6MPa (4.0 ~ 6.0kgf / cm²) (60 ~ 80pcs)
5. Oil drum capacity: 300ml
6. Size: (mm):310 270 350

III. Accessory Name

1) The names of parts are shown in Figure 1 and Figure 2:

Figure 1

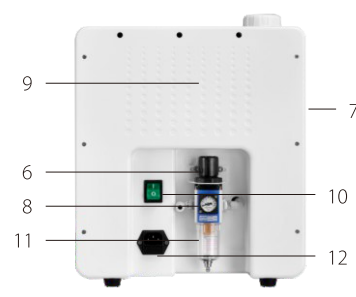
Figure 2

1. Oil cap
 2. Operational panel
 3. Door cover
 4. Oil level gauge
 5. Shell
 6. Operation pressure adjustment button
 7. Pressure gauge
 8. Air filter
 9. Beep hole
 10. Switch
 11. Power socket
 12. Fuse box
- 1) Accessory

The following accessories are equipped with each machine, if any one is missing, please contact your dealer:

1. Lubrication oil_____1 bottle
2. Oil-absorbing cotton_____1 bag
(15 pieces)
3. Oil funnel_____1 piece
4. Power cord_____1 set
5. Y-shaped connector_____1 piece
6. O-ring _____4 pieces
7. Air pipe_____1 piece

IV. Installation Of Components

1. Rated Fuse

Check the voltage whether it is consistent with the rated fuse according to the below chart

Voltage	Rated fuse
100V-240V	T3A/250V

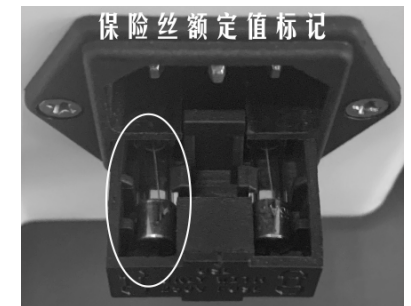
Figure 3

2. Connect The Air Tube

Insert one end of the air tube into the interface of the air filter and connect the other end directly to the air compressor for connection. If you need to connect other medical equipment, please use the attached y-type connector to connect.

Figure 4

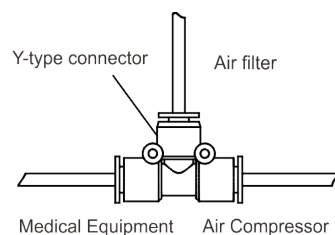
Figure 5

Warning Precautions

- 1) Working pressure ranges from 0.4~0.6MPa(4.0~6.0kgf/cm²)(60~80psi)
- 2) Working pressure is no less than 0.4MPa(3.5kgf/cm²)(50psi)
- 3) Do not wind or bend the air tube
- 4) If you can not install the air tube, please contact with the supplier.
- 5) The air source (like air compressor) and air filter which is supplied to the machine should be kept clean and dry. Regularly drain moisture from air compressor and air filter. Because the moisture inside the chamber will be mixed with the lubricating oil which will do harm to the machine.

3. Add Oil /Oil Drainage

(1) Add oil

Turn on the machine, rotate the refueling cap, put the refueling funnel into the refueling port and pour in the lubricating oil, stop refueling when the oil level reaches the "FULL" position, then remove the refueling funnel and tighten the refueling cap.

Figure 6

! Attention: Please add the lubricating oil in time when the oil level is below the "EMPTY" position

Warning Precautions

- 1) Before refueling, please check that there is no clutter in the refueling funnel to avoid blocking the lubricating system
- 2) Pay attention to the bottom of the machine drain bolts whether they are tightened, bolts do not tighten will lead to lubricating oil leakage
- 3) Be sure to refuel at start up
- 4) To avoid oil leakage, please pour into the oil slowly during refueling. Do not exceed the "FULL" position.
- 5) Make sure the cap is screwed tight after refueling
- 6) Not using the machine for a long time or in the process of transportation, please empty the oil storage tank.
- 7) Do not add the lubricating oil in the space where there is a danger of explosion and near the source of fire.

4. Connect The Power

Please insert the power supply into the socket on the back of the machine.

Figure 7

V. Operation Steps

1. How to install diferent types of handpieces

(1) Installation of high-speed pneumatic turbine handpiece

(Based on 1S09168 standard)

1. Remove the needle from the handpiece
2. Open the door cover, install the handpiece to the connector and tighten the nut (As shown in Figure 8)
3. Close the door carefully.

Figure 8

! Safety Precautions: Be sure to remove the needle before installing the hadnpiece into the connector of the lubrication device.

2) Installation of low-speed handpieces (based on 1S03964 standard)

Straight handpieces

1. Open the door cover, (as shown in Figure 9)

Install the handpiece into the low-speed connector until the handpiece and the interface are locked.

2. Close the door carefully.

Figure 9

Angled phone

1. Remove the needle from the handpiece.
2. Open the door cover, (as shown in Figure 9)

Install the handpiece into the low-speed connector until the handpiece and the interface are locked.

3. When installing, "please turn the pin jack of the angled handpiece towards the door cover, and carefully close the door cover (as shown in Figure 10)

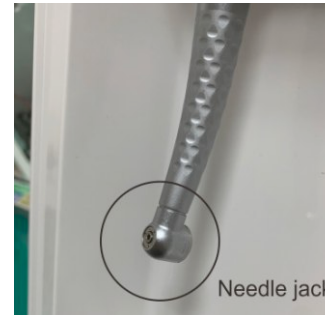
Figure 10

Safety Precautions

- 1) Make sure the needle has been removed before installing the hadnpiece into connector, and you can also install a standard clamping rod for maintenance. Before installing the right-angle handpiece, the needle needs to be locked.
- 2) The pin jack of the angled handpiece needs to face the door cover. If the direction is wrong, the door cover will not close.
- 3) When installing the high-speed manual turbo handpiece, be sure to insert to the connector in place, and then tighten the nut to avoid damage to the handpiece or nut thread.
- 4) After the installation of the low-speed handpiece, please gently pull the handpiece to make sure it is installed in place

2. Cleaning And Lubrication Steps

1. After turning on the power and turning on the power switch, the power indicator lights (see Figure 11).

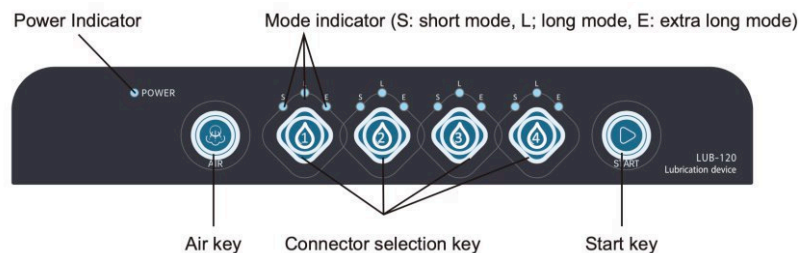
Figure 11

2. Select the 1st, 2nd and 3rd keys according to the handpiece installed in the corresponding connector. Press once to select "S" for short mode, press twice to select "L" mode for long mode, and press three times for "E" for extra long mode. These modes are indicated by S, L, and E lights.

3. If you have two or more handpiece, please select the correct mode for each handpiece.

4. Press the start button, the machine will automatically start the cleaning and oil filling procedures. During the operation of the machine, the indicator lights of different modes will keep flashing.

5. After the machine finishes cleaning and filling the handpiece, the mode indicator will stop flashing and the buzzer will give an audible prompt.

6. Press the air button, at this time, air will blow out air from the joint interface to remove excess oil left in the handpiece.

Mode	Clean oil filling time	Air cleaning time
Short mode	About 35s	About 60s
Long mode	About 45s	About 80s
Extra long mode	About 55s	About 90s

⚠ Safety Precautions

- 1) Use the correct power supply: 100V-240V ~
- 2) Select the corresponding mode according to different types of handpiece. The correct mode can clean the oil from the handpiece.
- 3) When the mode indicator is not lit, the machine will not operate normally.
- 4) After purchasing the product or if it has not been used for a long time, please long press the START button for 10 seconds before use.

Icon Description

!	Note		Product Serial Number
⚠	See attached files	○	Off (disconnected from the power supply)
🧑	Class B equipment	I	On (connected to the power supply)

Handpiece Disassembly Steps

1) Disassembly of gas turbine handpiece (According to the standard ISO9168):

Mode indicator lights up again, and after the cleaning and lubrication process is completed, open the cover (As Figure 12 shows). Then loosen the nut and pull out the handpiece from the connection.

Figure 12

2) Disassembly of low-speed handpiece (According to the standard ISO3964):

Mode indicator lights up again, and after the cleaning and lubrication process is completed, open the cover (As Figure 13 shows). Then loosen the nut and pull out the handpiece from the connection.

Figure 13

⚠ Safety Precautions

The lubricant remaining on the bottom of the door cover will leak. Use a cotton cloth to wipe off the lubricating oil in the bottom and the door cover. And deal with these cotton cloth that belong to medical waste products according to the law.

VI. Maintenance

(1) Replacement of oil mist filter cotton and oil-absorbing cotton. (As fig 14 shows)

Safety Precautions

Replacement of oil mist filter cotton and oil-absorbing cotton after about 50 uses.

Deal with these cotton that belong to medical waste products according to the law.

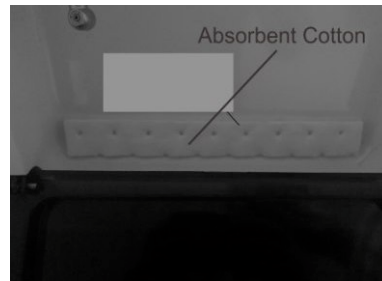
Figure 14

(2) O-ring replacement on low-speed handpiece connection

Use your fingers to take out the O-ring and put the new O-ring into the groove. (As fig 15 shows)

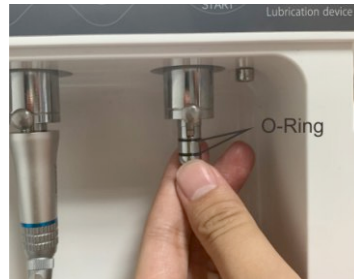
Figure 15

(3) Replacement Of Air Filter

- 1) Remove the air tube from the connection. (As fig 16 shows)
- 2) Take out the air filter after removing the screw. (As fig 17 shows)
- 3) Install a new filter, and insert the air tube into the connection as much as possible.

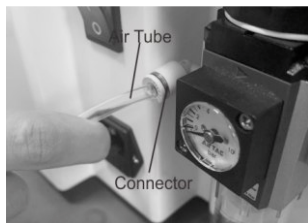
Figure 16

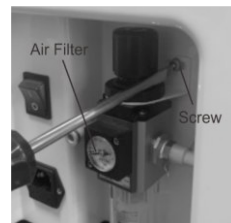
Figure 17

Notes:

Only remove the filter when the air source is completely closed.

Do not install the filter in reverse.

Use period: 8 year

(4) Replacement of air filter components, as shown in figure 18-19

Figure 18

Figure 19

Air filter for drainage, as shown in figure 20

Replacement of fuse, as shown in figure 21

Figure 20

Figure 21

VII. Trouble Shooting

If the machine is faulty, refer to the table below for similar conditions before maintenance is required. If there is no corresponding situation, contact your agent for repair.

Situation	Possible reason	Trouble shooting
No power	The power cord is not connected to the external power supply	Check external power
	Whether the power cord is plugged into the power interface of the machine	Check whether the power cord and the machine power plug are secure
	Whether the power switch is on	Check power switch
	Whether the fuse is blown	Check fuse
The machine cannot be cleaned and oiled	Whether the mode indicator is on	Press the mode key
	Whether the air pipe is connected	Check the trachea for air pressure
	Are handpieces installed	Check handpieces

VIII. Warranty

The product has defects in materials and workmanship during normal installation, use and maintenance, the manufacturer gives the buyer quality assurance, the maintenance oil, filter cotton, O-ring and other components that are consumables are not covered by the warranty. If the product fails within 30 days after installation, please contact the dealer immediately and show the purchase invoice.

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