bestdent

DISCOS PULIDORES

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EN INSTRUCTIONS FOR USE

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DE POLIERSCHEIBEN

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EN POLISHING DISCS

Classification: Class 2a Medical Device

- Discs – (single use only)
- Mandrels (multi-use device)

Besdent Polishing Discs with RA Mandrels are used by the Dental Surgeon for the finishing and polishing of Composite restorations.

The Discs are produced in 2 different shapes and 4 different grits and give best performance when applied at slow speed (between 10,000 and 30,000RPM), under slight pressure.

Besdent Polishing Discs with RA Mandrels are Class 2a Medical devices and are for use in the mouth only by (or under the instruction of) a qualified dental professional. They are manufactured in accordance with the standard ISO 1797, and should only be used in conjunction with a rotary hand piece that conforms to ISO 14457. The operator should ensure that the mandrel is correctly installed in the hand piece and the disc is correctly located onto the Mandrel prior to commencement of any procedure.

INSTRUCTIONS

- 1) Always use Besdent Polishing Discs with a slow speed hand piece (Less than 30,000 RPM). Affix the disc to the mandrel by gently pushing the eyelet piece on to the mandrel until the disc is securely in place.
- 2) The polishing motion should be constant and uni-directional, e.g. commencing at the gingiva and moving outwards over the restoration. A back and forth movement over the composite – enamel margin is not recommended.
- 3) Use light pressure when polishing, let the discs do the work.
- 4) Keep the restoration surface and disc dry while polishing. A dry surface will produce a smoother, more uniform surface.
- 5) Remove Besdent Polishing Discs from the Mandrel by: -
 - A) By positioning a thumbnail under the disc eyelet portion and pushing the disc away from the hand piece (i.e. Pop off) or;
 - B) By grasping the disc and eyelet and peeling the disc upwards away from the handpiece.

PROPER USE

- Only use hand piece, angles and turbines that are technically and hygienically flawless, maintained and cleaned,
- Turbine and hand piece must be concentric and true running.
- The instruments must be clamped correctly.
- Do not use the discs at speeds in excess of 30,000RPM.
- Do not operate the mandrel without the disc attached
- Instruments that are deformed or no longer run true should not be used and must be disposed of.
- Always wear safety goggles.
- Avoid touching composite with the mandrel or disc eyelet because discolouration may occur. This discolouration can be removed by repetition of the polishing steps. The smaller, low profile, mandrel head and unique eyelet design reduces this risk of composite and eyelet contact.
- Use the discs in the correct order, incorrect procedures can result in a reduction in the polishing quality.
- Wear a face mask to prevent biological transfer
- Incorrect use produces poor results and increases the risk. These products must only be used by qualified staff.

SAFETY PRECAUTIONS

All these dental instruments were developed and manufactured for their specific dental surgical application. Incorrect use may harm tissue, cause premature wear, destroy the instruments and endanger the operator, patient or third parties.

PRESSURE

- Excessive pressure must be avoided at all times.
- Excessive pressure may cause heat build up which may damage the disc.

Excessive pressure may cause overheating, which injures the pulp. In extreme cases, the instrument may even fracture

DISPOSE OF WORN INSTRUMENTS

- Fractured and incorrectly shaped discs cause vibration.
- Bent or non-concentric rotary instruments must also be disposed of.

STORAGE, DISINFECTION, CLEANING AND STERILISATION

- The discs do not require sterilisation, however, the multi-use mandrel does. The rotary instrument should be kept in its original packaging at room temperature and protected against dust and moisture until used for the first time.
- Rotary instruments should be kept in hygienic stands, dishes or other suitable containers and they must be protected against dust, moisture and recontamination during storage. If the instruments are not used right away, it is advisable to keep them in their original packaging.
- The mandrels should be sterilised for a holding time of not less than 20 minutes at a maximum temperature of 132°C.
- The method of use, reaction time and suitability of disinfectants and cleaning agents for certain types of instruments are covered by the manufacturers' instructions.

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