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tempolink® clear

GB
Instructions for use

Temporary luting cement, dual-hardening, for crowns and bridges, eugenol-free – resin based, with triclosan

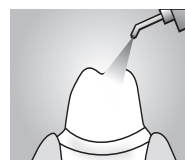
Fig. 1

1. Cleaning the preparation

Before cementing, clean the preparations with water. If the preparation has been disinfected beforehand, remove all traces of disinfectant from the surface. Dry the preparation area and dry the temporary restoration (Fig. 1).

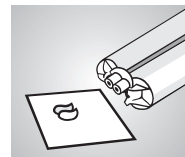
Fig. 2

2. Mixing and applying

Dispensing is performed with the mini-mix system. Insert plunger into the body of the cartridge. Remove the cap from the cartridge by turning. Before fitting the mixing cannula, dispense a small quantity of material until material emerges from the two openings uniformly (Fig. 2). Mount the mixing cannula by using the guides on the cannula and the cartridge and lock in place by rotating in the opposite direction (Fig. 3). Feed material applying even pressure. Prior to use, dispense a small amount of material and visually check that the base and catalyst paste are mixed homogeneously (Fig. 4). Only then dosing can be performed individually. After use, leave the mixing cannula on the cartridge until the next application.

Fig. 3

3. Inserting the temporary restoration

Apply a thin layer of **tempolink® clear** to the interior of the temporary restorations (Fig. 5). Directly after that, position the temporary restorations on the preparations by applying slight pressure (Fig. 6), then press them on firmly. Remove any excess material with a suitable instrument (Fig. 7).

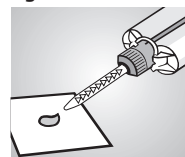
Fig. 4

4. Working time / light curing

The working time at room temperature is approx. 60 sec. Curing at body temperature takes approx. 5 min. and can be delayed outside the mouth. **Setting can be assisted and speeded up at any time by light curing.** For this purpose we recommend an exposure time of 20 sec. per surface (Fig. 8). If the temporary restorations are very opaque, extend light curing to 30 - 40 sec.

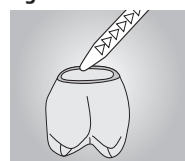
Fig. 5

Important working hints

- **tempolink® clear** is compatible with permanent cements.
- **tempolink® clear** does not have a delaying effect on the curing of composite cement or composite restorative material.
- In the case of composite-based and build-up restorations isolate these areas by applying a thin layer of glycerine or Vaseline in order to prevent adhesion of **tempolink® clear**.
- Where movement is impeded or blocked, discard the cartridge. Do not apply force.
- Before using **tempolink® clear** do not apply any methacrylate-based bonder, dentine sealant or desensitizer.
- The use of **smartprotect® / smartprotect® soft** is recommended before cementation.
- **tempolink® clear** does not contain eugenol.
- Manual mixing of the components is not recommended.
- Avoid contact of unmixed components with skin or mucosa. Wear suitable eye protection and gloves!
- Do not leave any residues of material in the mouth.
- Please follow the instructions of the safety data sheet!

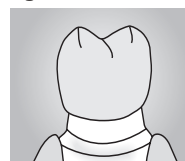
Fig. 6

Only for dental use by qualified personnel.

Fig. 7

Dual-hardening system

light curing:

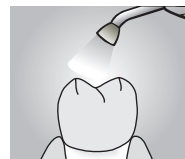
Fig. 8

self curing:

Applications:

- Temporary luting of crowns and bridges
- Temporary luting of permanent restorations

Contra-indication:

- In very rare cases **tempolink® clear** may cause sensitization. If such reactions should occur, discontinue using **tempolink® clear**.

Technical data:

- **Mixed volume:** 5 ml (mini-mix)
- **Mixing ratio:** 4:1
- **Colour code:** Base: transparent Catalyst: opaque
- **Mixing time:** omitted (mini-mix)
- **Working time:** approx. 60 sec.*
- **Setting time in the mouth:** light curing: 20 sec. per surface self curing: 5 min.
- **Application:** At 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity
- **Storage:**

* from beginning of mixing at 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity. Increased temperatures accelerate, decreased temperatures retard a. m. times.

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