Single-Component Adhesive**INTENTION OF USE:**

To enhance the anchor of the composite as part of the bonding system with the **bestdent** etchant.

COMPOSITION:

PMGDM in acetone
Bidestilled Water
Diurethane
HEMA
Photo initiator
Stabilizer

INSTRUCTIONS FOR USE:

Put a drop of dentin enamel bonding in a glass slab.
With a brush or cotton pellet take liquid from is the slab and apply a fine layer of the over the dentin and etched enamel.
When the application of the bonding is finished cure it with light cure lamp for 20 seconds.
Do not touch the remaining wet film of the bonding layer; then proceed to make the restoration with the composite selected.

WARNING:

- Always store the product in a cold area.
- In case of any change on the original function of the product, stop using it and contact the manufacturer.

For Professional Dental use only

Intention of use:

To enhance the anchor of the composite as part of the bonding system with the etchant.

The single-component adhesive is not irritable to the skin. In case of contact with the eyes, to wash immediately with abundant water during at least 15 minutes and if the inconveniences persist consults with a doctor. In case of contact with skin wash the area with soap and water. In case of inhalation clean the nose with water and medical assistance is required. Before the use of these products always use rubber dam.

Some patients can present allergy to the single-component adhesive or any of its components. In this case to suspend its use. Do not storage in temperatures above 24°C. Close the packages immediately after to dispense the product.

Only for use professional dental.

Contraindication:

The material must not be used when the techniques application described are not possible.

Warranty: The manufacturer warranties the product against any fault of manufacture or of the material. The manufacturer does not take responsibility for the use of the product out of the intention of use specified or out of the instructions of use, of the precautions, the warnings, and indicated contraindications.

Being its only obligation the replacement of the defective product.

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