BULK FILL FLOW MICRO HÍBRIDO FOTOPOLIMERIZABLE

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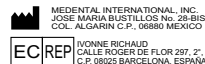
BULK FILL FLOW MICRO HYBRID LIGHT CURE

INTENTION OF USE

- Base and liner for direct restorations of Class I and II.
- For small Class I direct restorations in permanent molars, and for direct restorations of Class I and II of temporary teeth.
- Sealing of pits and fissures.
- Core buildups.

COMPOSITION: Crystalline and fumed silicon dioxide, Bisg-ma, Stabilizer, Photoinitiators.

INSTRUCTIONS FOR USE

TOOTH PREPARATION

Isolate tooth with rubber dam before starting. Follow usual procedures for tooth preparation. Exposed dentin must be protected by a Calcium Hydroxide preparation, glass ionomer or another suitable base can be used to cover the preparation.

Remove temporary restorations. Isolate and clean the tooth of any traces of temporary cement.

ETCHING TOOTH SURFACE ENAMEL

Apply agent etchant to the tooth surface using a cotton swab or brush. Allow to remain it for 20 to 30 seconds. On primary teeth with high fluoride, remain it for 90 to 120 seconds. If using a base ionomer cement base this should not be remain for more than 15 seconds. Rinse the area thoroughly for 20 seconds.

Do not allow the patient to rinse his mouth or allow saliva touch the surface of etched enamel. Dry thoroughly with compressed air from a free source of oil and moisture.

The tooth surface should have a dull, grayish appearance. Otherwise repeat the procedure again for another 20 to 30 additional seconds to the tooth to be engraved.

PRIMING TOOTH SURFACE

Immediately apply a thin but uniform coating dentin/enamel bonding on the surface of etched tooth, with oil and moisture free air flow gently applied over the coated surface to thiner the film. Avoid contact with the gingival tissue or other soft tissue.

If using porcelain faces or veneers, apply a layer of primer for porcelain over the engraved faces or veneers.

RESTORATION

Apply a layer thickness of 4 mm bulk fill in the prepared cavity and polymerize with visible light curing lamp for 40 seconds. If thicker layers are needed pour layers of 4 mm and polymerize each layer for 40 seconds until the cavity is full as necessary. After apply a thin adhesive layer and polymerize for 20 seconds. Give the finish as usual.

STORAGE AND USE

This system is designed to be used at room temperature of approximately 18°-24°C. The lifetime at room temperature is two years unopened cases a year to open cases when syringes are covered immediately after use.

Closed cases can be refrigerated at -4 ° C to extend shelf life, if it were stored in refrigerator, bring to room temperature prior to use.

Do not store materials in proximity of eugenol containing products. Do not expose materials to elevated temperatures or intense light.

WARNING

- In case of any change on the original function of the product, stop using it and contact the manufacturer.

For Professional Dental use only

WARNINGS

The **bestdent** Bulk Fill is not irritable to the skin. In case of contact with the eyes, wash immediately with abundant water during at least 15 minutes and if the inconveniences persist consult a doctor. In case of contact with skin wash the area with soap and water. In case of inhalation clean the nose with water and medical assistance is required. Before the use of these products always use rubber dam.

Some patients can present allergy to the **bestdent** Bulk Fill or any of its components. In this case suspend it use. Do not storage at temperature above 24°C. Close the packages immediately after to dispense the product.

Only for in professional dental use.

Contraindication: The material must not be used when the techniques application described are not possible.

Warranty **bestdent**. warranties the product against any fault of manufacture or of the material. **bestdent** does not take responsibility for the use of the product out of the intention of use specified or out of the instructions of use, or out of the precautions, the warnings, and indicated contraindications. Being its only obligation the replacement of the defective product.

Information Post-production:

If the use of these product on a patient or any one else some anomaly occurs please send detailed report to MEDENTAL INTERNATIONAL INC. medental_inc@hotmail.es

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