**VOCO**

Ceramic Bond

In accordance with DIN EN ISO 26874

Instructions for use

Product description:

Ceramic Bond is a one-component silanisation agent for durable adhesive bonding of porcelain, also zirconium dioxide, to composites as well as for the pre-treatment of glass fibre reinforced composite posts.

Indication:

Ceramic Bond is used as a porcelain coupling agent to achieve an optimal bond strength through chemical connection of silicate, zirconium dioxide and aluminium oxide ceramics as well as glass fibre reinforced composite posts to composite materials.

Method of Use/application:

For ceramics surfaces:

Silanising the surface of all-ceramic and zirconium dioxide restorations creates optimum conditions for long-term bonding of ceramic and composite.

- Once the fit of the porcelain restoration has been checked, etch or sandblast its bonding surfaces to clean them, rinse carefully with water and dry with oil-free air.
- Apply **Ceramic Bond** onto the conditioned surfaces with a brush, let act for 60 s and dry carefully with oil-free air.
- Apply the luting composite as described in the respective instructions for use and place the restoration.

Glass-fibre reinforced posts:

- After preparation of the root canal, try the fit of the glass-fibre reinforced post, shorten the post extraorally, then clean with alcohol and dry with oil-free air.
- Coat the post with **Ceramic Bond** and let dry for 60 s.
- Cement the post in the root canal according to the instructions of the manufacturer.

Indications and precautions:

Ceramic Bond contains organic acid, 3-methacryloxypropyltrimethoxysilane and acetone. **Ceramic Bond** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.

Contamination of the silanised bonding surface (e.g. saliva or blood) may reduce the bonding strength of **Ceramic Bond**. Contaminated surfaces must be cleaned again with etchant, rinsed and dried. Close the bottle after use.

Do not allow liquid **Ceramic Bond** to contact the skin/mucosa or eyes as this may have a slightly irritating effect.

Storage:

Store **Ceramic Bond** at temperatures of 4 °C to 23 °C. Refrigeration is recommended. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VOCO GmbH