



Components

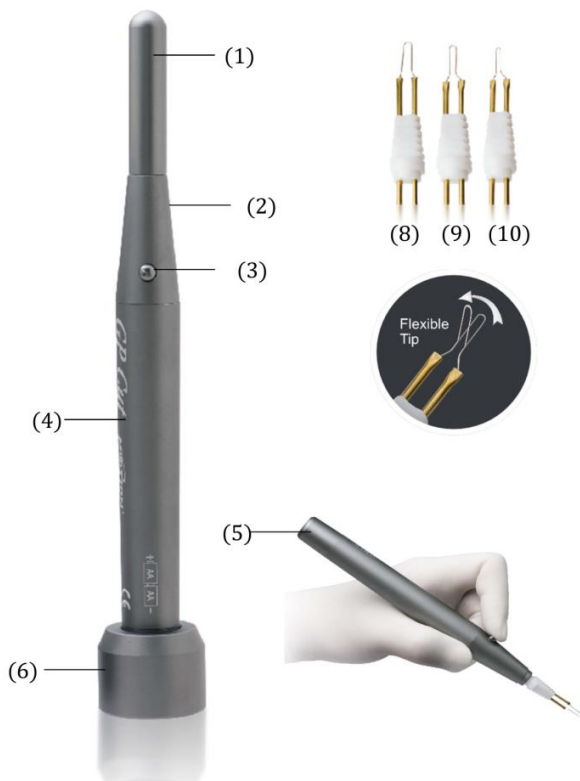

- (1) Cap
- (2) Front
- (3) Power Button
- (4) Handle
- (5) Battery cover
- (6) Stand Base
- (7) Two AA batteries (not shown)
- (8) Large Tips (2.0 mm)
- (9) Medium Tips (1.2mm)
- (10) Small tips (0.5mm)

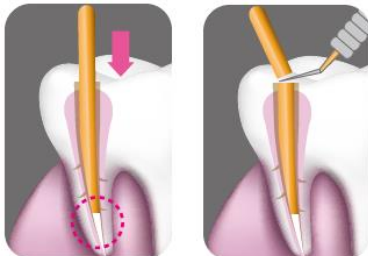

Specification

Dimension (LxW)	20cm x 1.85 cm (tips include)
Voltage	2 AA batteries (DC 3V)
Tip Type	3 Tips (L: 2.0mm; M: 1.2mm; S: 0.5mm)
Temp. Reach	Over 500 degree
Cool down period	less than 1 second
Stand base size	3 cm x 2.5 cm

GP CUT Feature

1. One button only, two batteries power supply, easy use design.
2. Three tip size available.
3. One second hip up to 500 degree high temperature.
4. Fast tip cool down design, safety use.
5. Flexible tip design, cutting smoothly.

Simple to Use

1. Install two AA batteries when first using the GP Cut device.
2. Insert the selected tip on the front of GP Cut. The tip is flexible design to let you cut smoothly.
3. Press the power button and the tip can heat up to 500 °C immediately.
4. Cut Gutta Percha.
5. Release the power button and the tip can cool down fast.
6. Put tip cap back after using.
7. Put GP Cut back on stand base.

CAUTION:

- (1) DO NOT touch tip during operation due to high temperature.
- (2) DO NOT press power button over 5 seconds.
- (3) NO water sprays on unit.

NOTE:

- (1) If you can't cut Gutta Percha quickly, please replace battery immediately.
- (2) The tip can be used until it cut off.

The device has been tested to meet the following standards: IEC 60601-1, IEC 60601-1-2, 93/42/EEC, and ISO 13485.

Manufacturer:

Motion Dental Equipment Corp.

Address: 6F-2 & -3, No. 27, Lane 61, Sec. 1, Guangfu Rd, Sanchong District, New Taipei City, Taiwan (R.O.C)


TEL : +866-2-2278-2199 FAX: +886-2-8511-2492

Website: <http://www.motiodental.com> E-mail: motion@twmotion.com

Componentes



- | | |
|------|--------------------------------|
| (1) | Tapón |
| (2) | Front |
| (3) | Botón de encendido |
| (4) | Mango |
| (5) | Cubierta baterías |
| (6) | Base soporte |
| (7) | Dos baterías AA (no incluidas) |
| (8) | Punta grande (L 2.0 mm) |
| (9) | Punta mediana (M 1.2 mm) |
| (10) | Punta pequeña (S 0.5 mm) |

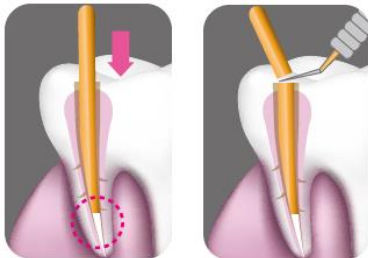

Especificaciones

Dimensiones (LxW)	20 x 1.85 cm (punta incluida)
Voltaje	2 baterías AA (3V cc)
Puntas	3 puntas (L: 2.0 mm; M: 1.2 mm; S: 0.5 mm)
Temperatura	>500 °C
Enfriamiento	Menos de 1 segundo
Soporte	3 cm x 2.5 cm

Características

1. Un solo botón, dos baterías, diseño de fácil uso.
2. Tres tamaños de punta disponibles
3. Tarda un segundo en alcanzar los 500 .
4. Punta diseñada para un rápido enfriamiento, uso seguro.
5. Diseño de punta flexible, corte suave.

Fácil de usar

1. Instale dos baterías AA cuando utilice por primera vez el aparato.
2. Inserte la punta seleccionada en la parte frontal. La punta tiene un diseño flexible para que pueda cortar sin problemas.
3. Presione el botón de encendido y la punta se calentará inmediatamente hasta 500 °C.
4. Corte la gutapercha.
5. Suelte el botón y la punta se enfriará rápidamente
6. Cubra la punta con el tapón después del uso.
7. Coloque el aparato en la base del soporte

PRECAUCIÓN:

- (1) No toque la punta durante el uso debido a su alta temperatura.
- (2) No mantenga el botón pulsado más de 5 segundos.
- (3) No salpique agua sobre el aparato.

NOTA:

- (1) Si no puede cortar la gutapercha rápidamente, por favor sustituya las baterías inmediatamente.
- (2) La punta puede usarse hasta que deje de cortar.

El dispositivo ha sido probado para cumplir con los siguientes estándares: IEC 60601-1, IEC 60601-1-2, 93/42/EEC, and ISO 13485.

Fabricante:

Motion Dental Equipment Corp.

Dirección: 6F-2 & -3, No. 27, Lane 61, Sec. 1, Guangfu Rd., Sanchong District, New Taipei City, Taiwan (R.O.C)

TEL : +866-2-2278-2199 FAX: +886-2-8511-2492

Website: <http://www.motiodental.com> E-mail: motion@twmotion.com