- DE** GEBRAUCHSANWEISUNG
- EN** INSTRUCTIONS FOR USE
- ES** INSTRUCCIONES DE USO
- FR** MODE D'EMPLOI
- IT** ISTRUZIONI PER L'USO
- PL** INSTRUKCJA POSTĘPOWANIA
- PT** INSTRUÇÕES DE USO

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**EN ROOT CANAL SEALING MATERIAL**

**General Information**

The Root canal sealing is a polyepoxy resin based, paste type root canal sealer delivered in a 1:1 dual barrel automix syringe. Its chemical composition ensures excellent biocompatibility while its physical properties produce a complete hermetic seal in conjunction with a guttapercha point. Root canal sealing is suitable for use with a single-point method or lateral condensation technique.

**Composition**

Base: Epoxy oligomer resin, Ethylene glycol salicylate, others  
Catalyst: Poly(1,4-butanediol)bis(4-aminobenzoate), Calcium phosphate, others

**Indication**

Obturation of root canals using adhesion properties between root canal walls and root canal filling materials.

**Characteristics**

Working Time	70 min (23°C, 55%)
Setting Time	120 min (37°C, 98%)
Flow	21 mm
Thickness	10 µm
Radiopacity	8.0 mm

**Precautionary Information**

- This product can only be used by dentists and dental professionals.
- In case of accidental contact with eyes or skin, rinse immediately with plenty of water and take appropriate action.
- Do not use the product after the expiration date.

**Canal Preparation**

- Thoroughly prepare and irrigate the root canal using standard endodontic techniques.
- Evacuate liquids from canals and dry with absorbent paper points.
- Select and set aside a sterilized master guttapercha cone for each canal.

**Restoration**

- Remove the cap from the Root canal sealing syringe and extrude a small amount to make sure both base and catalyst

are dispensing equally.

- Mix Root canal sealing on a mixing pad using an automatic mixing tip or extrude with a Eco Tip and manually mix with a spatula for 15-20 seconds until homogenous color and consistency is achieved.
- Coat the preselected master guttapercha cone with a thin layer of the Root canal sealing and very slowly insert it into the canal. Seat it to full working length and confirm tug back at the apex.
- If desired, place additional guttapercha points into the canal using standard condensation techniques.
- Trim excess guttapercha using a hot instrument. Apply a slight amount of vertical condensation, and remove any excess sealer with a moist cotton ball or swab.
- After application, remove the mixing tip or Eco tip from the syringe. Clean the outside of the syringe, place the cap tightly on to syringe hub. Store in a cool and dry place.

**Storage and Use**

This product is designed to be used at room temperature. If refrigerated storage is used, allow product to reach room temperature prior to use. Shelf life at room temperature is 36 months. See outer package for expiration date. Ambient temperatures routinely higher than 25°C/77°F may reduce shelf life. Do not expose restorative materials in proximity to products containing Eugenol. Disinfect the product using an intermediate level disinfection process (liquid contact) as recommended by the CDC and endorsed by the ADA. Guidelines for infection Control in Dental Health-Care Settings – 2003(Vol. 52; No.RR-17), Centers for Disease Control and Prevention.

**Packaging**

- 10 g Dual Syringe x1 (Base 5 g, Catalyst 5 g)
- Mixing Tip ,Eco Tip
- Protective Cap , Mixing pad

**Disposal**

See the Material Safety Data Sheet (available at www.spident.co.kr or through your local subsidiary) for disposal information.

**Customer Information**

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

**Caution**

U.S. Federal Law restricts this device to sale or use on the order of a dental professional.

**Warranty**

Spident® warrants this product will be free from defects in material and manufacture. Spident®. MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Spident®'s sole obligation shall be repair or replacement of the Spident®'s product.

**Limitation of Liability**

Except where prohibited by law, Spident® R will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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SPIDENT CO., LTD.