bestdent

RECONSTRUCTOR DE MUÑONES

- DE** GEBRAUCHSANWEISUNG
- EN** INSTRUCTIONS FOR USE
- ES** INSTRUCCIONES DE USO
- FR** MODE D'EMPLOI
- IT** ISTRUZIONI PER L'USO
- PT** INSTRUÇÕES DE USO

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EN CORE BUILD UP

Composition

Organic Matrix: Bis-GMA, UDMA, TEDGMA
Inorganic Fillers Hydrophobic micro/nano Silica
Additives: Initiator, Catalyst, Stabilizer, Others

Timing

Working Time 1 min 30 sec
Chemical-Cured 4 min (intraoral)
Light-Cured 20 sec @ 2 mm thickness

Indications

Core build-ups

Application

Cavity Preparation: Remove caries and any amalgam or resin filling(s) under which the recurrent decay has occurred.

Retention: Root canal and multiple missing cusps may require preparation for post insertion.

Adhesive System Application: Apply **FineEtch®** (37% phosphoric acid gel) to enamel, exposed dentin and post (if desired). Wait 15 seconds, rinse and air dry. Apply **EsBond®** and **EsBond® Activator** to etched

enamel, dentin and post (if desired). Light cure all surfaces for 20 seconds.

Insertion of Posts: Apply Core Build up into the prepared canal then insert post(s).

Matrix Placement: Place matrix if necessary and wedge appropriately for the restoration.

Placement: Use the **Smart Type Syringe** of Automix + Intraoral tips attached to the **Cartridge Type** to place the restorative into undercut areas of prepared cavity and around inserted post(s). Keeping the syringe tip immersed in the material minimizes air entrapment and surface voids. Contour the restoration using appropriate placement instruments.

Curing: Light cure the entire surface area of the restorative for 20 seconds. For core buildups where a matrix band has been placed, light cure the occlusal for 20 seconds. The recommended depth of material for light curing is not to exceed 4 mm. Chemical-Cured setting time is 4 minutes from the start of mix at oral cavity temperature. For core buildups, any soft axial areas may be light cured or allowed to self-cure following matrix removal.

Finishing: Immediately after curing, the restorative can be contoured using any conventional rotary instruments under water spray.

Storage & Use

This product is designed to be stored and used at room temperature. If refrigerated storage is used allow product to reach room temperature prior to use. Shelf life at room temperature is 24 months. See outer package for expiration date. Ambient temperatures routinely higher than 27°C/80°F may reduce shelf life. Do not expose restorative materials to elevated temperatures, or to intense light. Do not store materials in proximity to products containing eugenol. Disinfect the product using an intermediate level disinfection process (liquid contact) as recommended by the CDC and endorsed by the ADA. Guidelines for infection Control in Dental Health-Care Setting – 2003 (Vol. 52; No RR-17), Centers for Disease Control and Prevention.

Disposal

See the Material Safety Data Sheet (available at www.spident.co.kr or through your local subsidiary) for disposal information.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Caution

U.S. Federal Law restricts this device to sale or use on the order of a dental professional.

Warranty

Spident® warrants this product will be free from defects in material and manufacture. Spident® MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Spident®s sole obligation shall be repair or replacement of the Spident®s product.

Limitation of Liability

Except where prohibited by law, Spident® will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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Composition

Organic Matrix: Bis-GMA, UDMA, TEDGMA

Inorganic Fillers Hydrophobic micro/nano Silica

Additives: Initiator, Catalyst, Stabilizer, Others

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Adhesive System Application: Apply 37% phosphoric acid gel to enamel, exposed dentin and post (if desired). Wait 15 seconds, rinse and air dry. Apply **bonding** and **activator** to etched enamel, dentin and post

(if desired). Light cure all surfaces for 20 seconds.

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Placement: Use the **Smart Type Syringe** or Automix + Intraoral tips attached to the **Cartridge Type** to place the restorative into undercut areas of prepared cavity and around inserted post(s). Keeping the syringe tip immersed in the material minimizes air entrapment and surface voids. Contour the restoration using appropriate placement instruments.

Curing: Light cure the entire surface area of the restorative for 20 seconds. For core buildups where a matrix band has been placed, light cure from the occlusal for 20 seconds. The recommended depth of material for light curing is not to exceed 4 mm. Chemical-Cured setting time is 4 minutes from the start of mix at oral cavity temperature. For core buildups, any soft axial areas may be light cured or allowed to self-cure following matrix removal.

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Storage & Use

This product is designed to be stored and used at room temperature. If refrigerated storage is used allow product to reach room temperature prior to use. Shelf life at room temperature is 24 months. See outer package for expiration date. Ambient temperatures routinely higher than 27°C/80°F may reduce shelf life. Do not expose restorative materials to elevated temperatures, or to intense light. Do not store materials in proximity to products containing eugenol. Disinfect the product using an intermediate level disinfection process (liquid contact) as recommended by the CDC and endorsed by the ADA. Guidelines for Infection Control in Dental Health-Care Setting – 2003 (Vol. 52; No RR-17), Centers for Disease Control and Prevention.

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Warranty

Spident® warrants this product will be free from defects in material and manufacture. Spident® MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Spident®'s sole obligation shall be repair or replacement of the Spident®'s product.

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SPIDENT CO., LTD.