# bestdent

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- DE GEBRAUCHSANWEISUNG
- EN INSTRUCTIONS FOR USE
- ES INSTRUCCIONES DE USO
- FR MODE D'EMPLOI
- IT ISTRUZIONI PER L'USO
- PT INSTRUÇÕES DE USO

## DE FISSURENVERSIEGLER

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## EN FISSURE SEALANT

Fissure Sealant is a light-cured pit & fissure sealant to aid in prevention of caries and enable to penetrate the depth of pit due to the suitable viscosity. They have the excellent characteristics of bonding strength, low polymerization shrinkage and abrasion resistance by adding the nano filler.

### Application range

Sealing the enamel pit and fissure of teeth to aid in prevention of caries.

### Application method

- Prepare the cavity by cleaning and drying.
- Apply a adequate amount Etch to enamel surface to be sealed.

- After ringing by water and drying the teeth, apply the sealant.
- Cure the Fissure Sealant by exposing it to light by curing light unit for 20seconds.

### Package

- Sealant: 1.2ml x 2 syringes, tips.

### Cautions

- Fissure Sealant is only used for oral tissue.
- Avoid contact with eyes.
- In case of contact with the skin or eyes, clean with pure water several times, and then consult with a medical specialist if necessary.
- Do not use expired Fissure Sealant.
- For dental use only.

### Storage

- Do not store it under conditions of high temperature or intense light.
- Close the cap immediately after use.
- Store in dry and cool place.
- Valid for 2years from the manufacturing date.

### Side effect

In rare cases, Fissure Sealant may cause a sensitizing reaction for hypersensitive patients due to any of the ingredients of these materials. In these cases, the sealant must no longer be used.

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