FICHA TÉCNICA

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TECHNICAL DATA SHEET

BESTDENT FISSURE SEALANT		
REFERENCE	63546	
MANUFACTURER	SPIDENT CO., LTD. 203&312 Korea Industrial Complex, 722, Gojan-Dong, Namdong-Gu, Incheon, Korea 405-821	
CLASSIFICATION	Medical Device, Class IIa according to Directive 93/42/EEC	
DESCRIPTION	Light-cured pit and fissure sealant.	
COMPOSITION	The main ingredients are urethane dimethacrylate, triethylene glycol dimethacrylate, bisphenol A ethoxylate dimethacrylate, calcium fluoride.	
SPECIFICACIONES	The product has excellent characteristics of bonding strength, low polymerization shrinkage and abrasion resistance.	
SHELF LIFE	2 years from the manufacturing date.	
PACKAGING	Package with 1,2 ml x 2 syringes + tips	
INTENDED USE	For dental use. It is used for sealing the enamel pit and fissure of teeth to aid in prevention of caries.	
INSTRUCTIONS FOR USE	<ul style="list-style-type: none"> • Prepare the cavity by cleaning and drying. • Apply an adequate amount of etch to enamel surface to be sealed. • After rinsing with water and drying the teeth, apply the sealant. • Cure the sealant by exposing it to light by curing light unit for 20 seconds. 	
SAFETY INFORMATION	In rare cases, the fissure sealant may cause a sensitizing reaction in hypersensitive patients due to any of the ingredients. In these cases, the sealant must no longer be used.	
HANDLING AND STORAGE CONDITIONS	Do not store it under conditions of high temperature or intense light. Close the cap immediately after use. Store in dry and cool place.	