MEDI-KORD A.S.

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MEDI-KORD A.S.

Braided cotton thread for retraction of movable gingiva with Aluminium Sulfate, for use by dentists only!

MEDI-KORD A.S. is braided cotton cord saturated with a solution of aluminium sulfate and dried by a technique which retains the activity of the active ingredient. Normal tissue moisture reactivates the active ingredient, which promptly retracts gingival tissue in a manner that causes no tissue damage, yet fully exposes the gingival margin.

Available in dispenser of 183 cm of uninterrupted thread in 4 sizes: MEDI-KORD A.S. 7, 8, 9, 10.

COMPOSITION: The average concentration of aluminium sulfate for each size / cm is:

MEDI-KORD A.S. 7 0.12 mg/cm · 16 H₂O

MEDI-KORD A.S. 8 0.189 mg/cm · 16 H₂O

MEDI-KORD A.S. 9 0.335 mg/cm · 16 H₂O

MEDI-KORD A.S. 10 0.57 mg/cm · 16 H₂O

The product is not sterile.

INDICATIONS: Use whenever gingival retraction is advisable in dental practice, as preceding impression taking with any elastic impression material; or prior to placement of caps, crowns, bridges; and during tooth preparation.

PRECAUTIONS:

- Read and follow all instructions.
- For professional in-office use only.
- For topical intraoral use only. Do not ingest.
- Keep out of reach of children
- Do not use on patients with aluminium sulfate allergy or sensitivity.

DIRECTIONS FOR USE:

Use as much cord of the appropriate sizes as is necessary to go all the way around the neck of each tooth to be retracted, with a slight overlap. Two loops may be packed around each tooth (see below), if sulcus space allows.

- For most effective use of MEDI-KORD A.S., loop a strand of smaller cord around the tooth and over the crevice.
- Trim the ends for a slight overlap and pack the loop firmly into the sulcus.
- Apply a second loop of larger cord over the smaller cord in the same manner.
- When ready to apply the impression material, remove the larger cord loop only, leaving the smaller cord in place below the gingival margin until the impression procedure has been completed.
- Once the impression procedure has been completed the smaller cord is removed also.
- Alternately, a single loop of cord will suffice.
- MEDI-KORD A.S. completely retracts tissue within five to eight minutes and should then be removed from the mouth.

Dispensed portions of cord are intended for single use only and should be disposed of immediately after use. Reusing dispensed portion puts patients at increased risk for injury or infection.

CONTRAINDICATIONS: There are no known contraindications for the use of MEDI-KORD A.S.. The chemical components of MEDI-KORD A.S. have a long history of safety and effectiveness for the purposes intended.


For product SDS information please see company website at www.lamaisondentairesa.com or contact La Maison Dentaire directly.

La Maison Dentaire SA
Alte Churerstrasse 36, FL-9496 Balzers – Liechtenstein

+30°C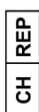

CH-REP / Importer
Medirel SA
via Redondello 20, CH-6986 Agno
Tel. +41 (0)91 604 54 34 – info@medirel.ch

(expiry date is printed on the label)

Rev. 09/2022