# LuxaForm

DMG Chemisch-Pharmazeutische Fabrik GmbH  
 Elbgaustraße 248 · 22547 Hamburg · Germany  
 www.dmg-dental.com  
 091673/#2-2015-10

DE	Gebrauchsinformation	SV	Användningsinformation
EN	Instructions for use	PL	Wskazówki dotyczące użycia
FR	Mode d'emploi	RU	Инструкция по применению
IT	Istruzioni per l'uso	ZH	使用说明书
ES	Instrucciones de empleo	JP	取扱説明書
PT	Instruções de uso	KO	사용 설명서
NL	Gebruiksaanwijzing	TR	Bilgi broşürü
DA	Brugsanvisning		

## Gebrauchsinformation

## Deutsch

### Produktbeschreibung

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Polycaprolacton, Additive, Pigmente

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## English

### Product description

LuxaForm is a thermoplastic molding polymer.

### Indications

- Impressions for provisional repairs

### Basic safety information

- ▶ For dental use only!
- ▶ Keep out of the reach of children!

### Interactions with other materials

- Do not use with MMA-containing provisional crown and bridge material.

## Instructions for use

- Malleable LuxaForm sticks to gloves. Always knead LuxaForm without gloves. For insertion in the mouth, isolate the fingertips of the gloves with Vaseline.
- In order to ensure secure removal, do not model LuxaForm too tightly around the teeth when making an impression in the mouth.
- Block out any undercuts with an appropriate agent, e.g. wax.
- Do not use LuxaForm to make an impression of more than 5 teeth.
- Perform any necessary post-processing of the impression with a scalpel.
- The impressions retain their form for months and can also be re-used later, e.g. if a temporary restoration breaks.

## Recommended use

1. Place one LuxaForm tablet per tooth to be molded in approx. 70 °C/158 °F hot water, until the material becomes soft and clear (approx. 1 minute). For multiple applications, use more tablets as needed. The tablets can be kneaded together once they have become transparent.
2. Shape the transparent material into a cube and press it around the teeth that are being molded.

**Note:** After cooling, the material quickly becomes hard and turns opaque blue. It can be cooled more rapidly by briefly spraying water on it.

3. Remove the hardened LuxaForm from the tooth and lightly isolate it with Vaseline or another separating agent.

**Note:** When using temporary crown and bridge materials, they may not exceed an initial temperature of 25 °C/77 °F.

4. Apply temporary crown and bridge material (e.g. Luxatemp Star) as normal.

## Ingredients

Polycaprolactone, additives, pigments

## Storage

- ▶ Store in a dry place (15 - 25°C/59 - 77°F) at room temperature.
- ▶ Do not use after the expiration date.

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