Rx Only

MD

135°C  
⏏

## Purevac® High Volume Evacuation Mirror Tips

Canule d'aspiration de haut débit avec miroir / Cannula di evacuazione ad alto volume con specchio /  
Spiegelsauger für Hochleistungsabsaugung / Puntas de espejo de evacuación de alto volumen /  
Hoogvolume afzuigspiegeltips

Manufactured For:  
Dentsply Sirona  
1301 Smile Way  
York, PA 17404 USA  
[www.dentsplysirona.com](http://www.dentsplysirona.com)

Dentsply DeTrey GmbH  
De Trey Strasse 1  
Konstanz D-78467  
Germany

11080 Rev. 2 (0120)

## Instructions for Use

Mode d'emploi / Istruzioni per l'uso / Gebrauchsanweisung / Instrucciones de uso / Gebruiksaanwijzing

### Indications for Use

The Purevac® High Volume Evacuation Mirror Tip allows for evacuation, visualization, retraction and illumination during dental procedures. The mirror tip is intended to be used by trained dental professionals only.

### Contraindications

None known.

### Warnings

The mirror tip must be sterilized before first use and between patients to prevent cross-contamination. Follow validated reprocessing instructions provided.

### Precautions

The mirror tip is designed to last up to 100 uses. Ensure the mirror tip is secured in the valve before use. Do not use brushes for cleaning which may scratch the mirror or plastic. Take precaution during use with other dental instruments to avoid damage to the mirror and plastic. Be sure to dry the mirror surfaces thoroughly prior to sterilization. Dispose of the mirror tip in accordance with government agency regulations.

### Adverse Reactions

None known.

### Residual Risks

Use of the device beyond its useful life may cause damage and increase risk of patient cross-contamination.

Failure to follow the validated reprocessing instructions provided may result in exposing the patient to cytotoxic residuals and/or cross-contamination.

### Step by Step Instructions

Always sterilize the mirror tip before first use and between patients. Insert the mirror tip directly into high volume evacuation valve. The mirror tip is compatible with standard 15 mm systems per ISO 10637:1999. Ensure the mirror tip is secured in the valve before use.

### Reprocessing Instructions

There are two possible cleaning and disinfection procedures recommended between patient use to prevent cross-contamination: (1) Ultrasonic Cleaning and Disinfection or (2) Automated Cleaning and Disinfection. It is only required to follow one of these two methods as part of the overall reprocessing procedure

Point of Use: Turn suction off and remove mirror tip from valve.

Ultrasonic Cleaning and Disinfection: Remove visible soil by rinsing the interior and exterior of the mirror tip under tap water of at least drinking quality for a minimum of 30 seconds. Prepare instrument cleaning or cleaning/disinfection solution per the manufacturer's recommendation. Immerse the mirror tip in an ultrasonic bath containing the prepared solution for the contact time specified by the solution manufacturer. The recommended solutions are PracticeProtect™ Instrument Disinfection & Cleaning Liquid and Resurge® Instrument Cleaning

Solution. Rinse the mirror tip under tap water of at least drinking quality for a minimum of 30 seconds. Allow the mirror tip to dry. Visually inspect the mirror tip for visible soil, and repeat the cleaning and disinfection procedure if necessary.

Automated Cleaning and Disinfection: A thermal disinfectant that complies with ISO 15883 may be used for automatic cleaning and disinfection. Follow the manufacturer's instructions to select the solutions and programs.

Sterilization: Ensure that the mirror tip, particularly the mirror, is completely dry before sterilization. A lint-free, disposable cloth may be used to dry the mirror tip. Enclose the mirror tip into an appropriately sized sterilization pouch intended for steam sterilization. The mirror tip must be sterilized using a steam sterilizer. A pre-vacuum sterilizer may be used at 135°C for 3 minutes or 121°C for 15 minutes. A positive steam flush pressure pulse sterilizer may be used at 134°C for 3.5 minutes. A gravity displacement steam sterilizer may be used at 135°C for 10 minutes. Do not remove the mirror tip from the sterilization pouch until ready for use to prevent device contamination.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the Competent Authority of the Member State.

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