PHILIPS

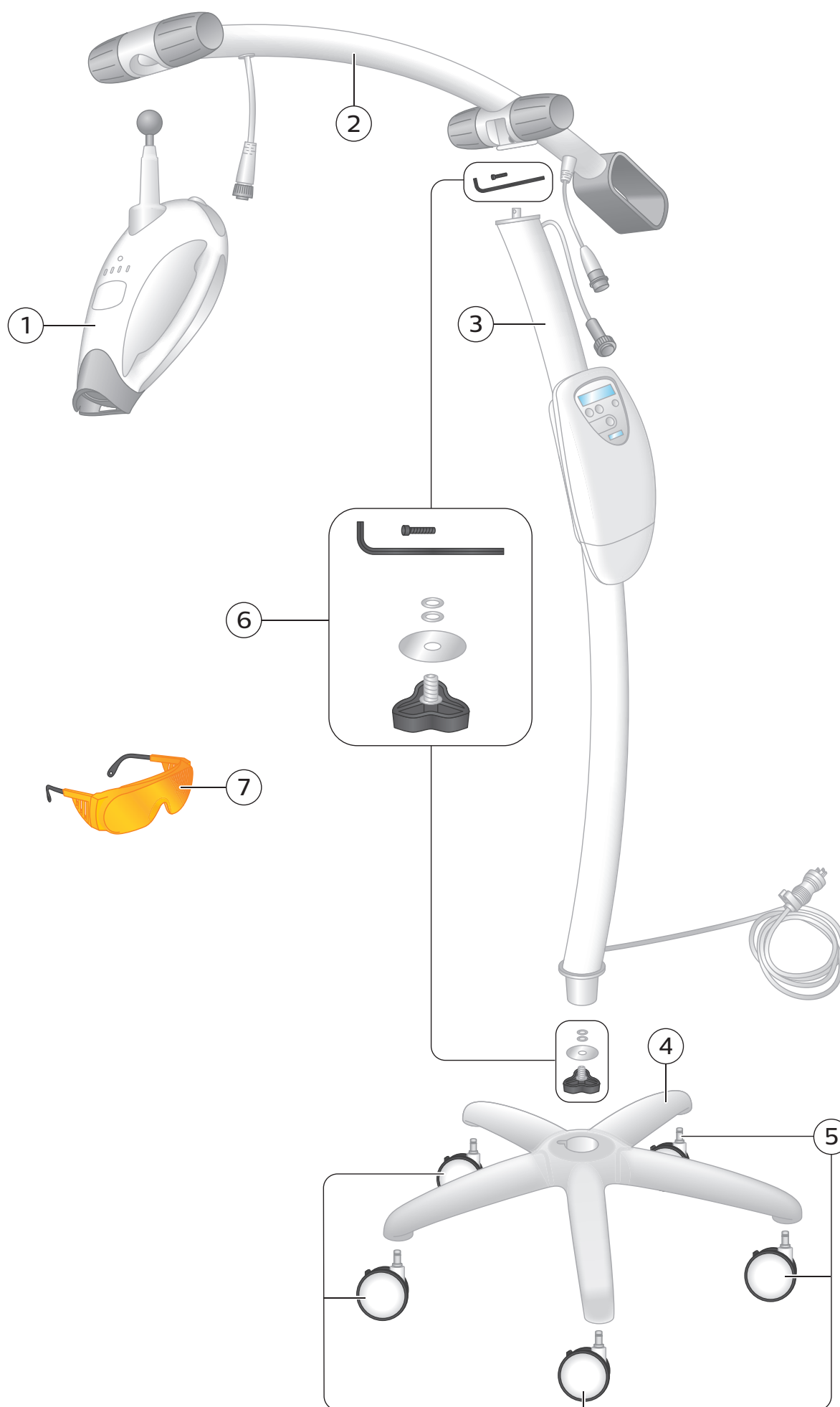
ZOOM!

WhiteSpeed

Philips Zoom!
WhiteSpeed Whitening LED Accelerator
Instructions for use

EN	English	6
CS	Čeština	13
DA	Dansk	20
DE	Deutsch	27
ES	Español	35
FR	Français	42
HR	Hrvatski	49
IT	Italiano	56
LT	Latviešu	63
NL	Nederlands	70
NO	Norsk	77
PL	Polski	84
PO	Português	91
RO	Română	98
FI	Suomi	105
SV	Svenska	112
TR	Türkçe	119
EL	Ελληνικά	126
BG	Български	133

Introduction

Welcome dental professionals!

The Philips Zoom! Training Center is your destination for earning the certification required to offer Zoom! WhiteSpeed Light Accelerated Whitening Treatments at your practice.

Visit **www.zoomtraining.com** to participate in a full course of 11 modules to get your office up to speed with Philips Zoom! Whitespeed Light Accelerated Whitening Treatments (treatments).

Note: The Zoom! training course is offered in English only.

Important safety information

Do not attempt to use the Philips Zoom! WhiteSpeed Accelerator Light (accelerator light) without a thorough understanding of these instructions for use, and the instructions for use for the Philips Zoom! Light Accelerated Procedure Kits (procedure kits). Failure to do so may result in injury to you or your patients, or damage to the product. Only allow trained and qualified personnel to operate the system.

Caution

- Zoom! protective eyewear must be used during operation to ensure the safety of everyone in the treatment area. The intense light can pose hazards for the eyes. Protective safety goggles or spectacles that filter out wavelengths between 400 and 505 nanometers (nm) must be worn while operating this device.
- Never look directly into the light.
- Never direct the light output of the accelerator light in any direction other than the area being treated.
- Before activating the light, confirm the accelerator light head is properly positioned and aimed at the treatment area.
- Do not attempt to gain access to the inside of the device. Service is limited to authorized service representatives of Philips. Any attempt to gain access to the inside of the device by unauthorized personnel or without proper authorization is prohibited, may result in injury, and will void any warranty or service obligations by Philips.

Electromagnetic fields (EMF) and electromagnetic compatibility (EMC)

The accelerator light complies with relevant international and national law and standards on electromagnetic compatibility (EMC) for this type of product when used as intended. Such laws and standards define both the permissible electromagnetic emission levels from product and its required immunity to electromagnetic interference from external sources.

Other electronic products exceeding the limits defined in such EMC standards could, under unusual circumstances, affect the operation of the product.

- Medical electrical products need special precautions regarding electromagnetic fields (EMF), and need to be installed and put into service according to the EMF information provided in these instructions for use.
- The use of accessories and cables other than those specified may result in increased emission or decreased immunity levels.
- The product should not be used adjacent to other products and if adjacent use is necessary, it should be observed to verify normal operation.
- To avoid the risk of electrical shock, do not remove any panels from the accelerator light.
- Refer servicing to qualified personnel.
- Never operate the device with a damaged power cord or plug.

Product overview

Each accelerator light has been thoroughly inspected and tested at the factory. Upon arrival at your facility, inspect the shipping container and the device for damage. Any damage should be reported immediately to your Philips representative. In the case of a damaged shipment, please save all packaging materials.

Contents

- 1 Head
 - 2 Boom
 - 3 Stand with power pack
 - 4 Base
 - 5 Casters (5)
 - 6 Parts bag
 - Allen wrench
 - Screw
 - Bolt with knob
 - Lock washer in 2 parts (2)
 - Large fender washer
 - 7 Protective eyewear (3)
- Not shown: Warranty card

Intended use

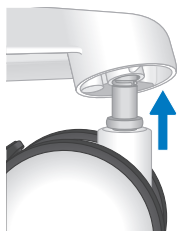
The accelerator light is intended for use as part of Philips Zoom! WhiteSpeed Light Accelerated Tooth Whitening System (system). The system consists of an accelerator light, Philips Zoom! WhiteSpeed In-Office Light Accelerated Tooth Whitening Gels (whitening gels) and a procedure kit. The accelerator light delivers the optimal spectrum of light energy to accelerate the tooth whitening process, delivering up to 6-8 shades* of teeth whitening. Wavelength emissions are between 400 and 505 nm and the energy intensity delivered to the teeth can be set at three levels depending on patient and case type. The accelerator light must be used at all times in accordance with the terms and conditions of the Letter of Agreements (LOA) provided and signed prior to delivery of the accelerator light.

**Based on the Vitapan Classical shade guide arranged in value order. VITA and Vitapan are trademarks of Vita Zahnfabrik, AG Germany.*


Preparing for use

Accelerator light assembly


- 1 Remove all items from the box.
- 2 Snap the casters into the feet of the base and lock the casters.

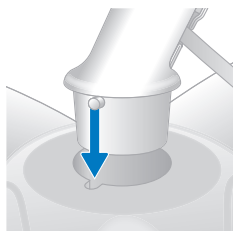
- 3 Place the base upright on the floor. Insert the stand into the base by aligning the pin in the stand with the slot in the base and pushing the stand securely into place.

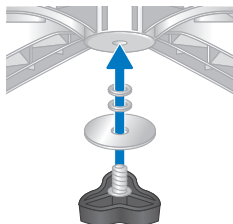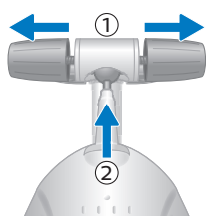
- 4 Assemble the lock washer with the big teeth facing each other and place it on the bolt, followed by the fender washer. Reaching under the base, screw the bolt into the bottom of the base, securing it to the stand.

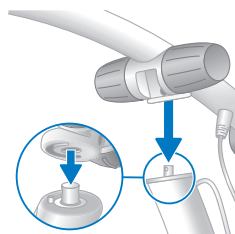
- 5 Loosen the knobs on the side of the socket at the end of the boom 4-5 turns, then push the head into the socket.

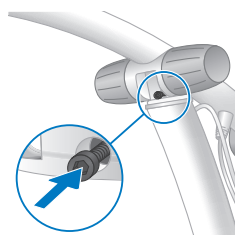
- 6 Place the boom onto the stand by carefully aligning the hole in the boom with the hole in the stand.

- 7 Insert the screw into the hole and tighten it. Keep tightening the screw until it is completely in the pocket. The screw should be flush with or inset into the pocket.

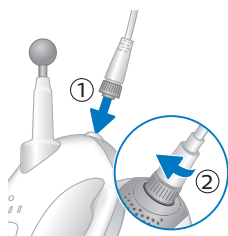IMPORTANT: The screw has glue on the threads (yellow in color) that releases as the screw is tightened. Therefore it is important to finish tightening the screw upon insertion or the glue may harden and stop the screw from being fully inserted.

- 8 Connect the electrical cord of the boom to the cord of the stand and the head. Turn the head connector clockwise until you feel it lock into place.

- 9 When you are ready to test the accelerator light, switch on the device by setting the switch on the power pack to 'O'.

Accelerator light setup

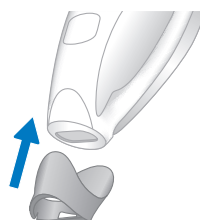
Plug the power cord of the base directly into a properly-rated power outlet.

Caution: The inlet (the end of the cord) must be accessible in order to disconnect the power if a problem occurs.

The accelerator light is now ready for use with a procedure kit. Please refer to the procedure kit instructions for use for further setup and whitening gel application instructions.

Zoom! Light Accelerated In-Office Whitening Treatment

Light guide

The light guide is designed to ensure proper distance and alignment to maintain correct light exposure on the smile zone.

- 1 Remove the light guide from the procedure kit and place it on the end of the accelerator light head.
- 2 Switch on the accelerator light using the power button located on the side of the power pack. When the light guide is properly attached, the power pack display will read 'Guide Status-Guide Attached' followed by 'X of X Sessions'.

Each new light guide is loaded with four 15-minute acceleration sessions.

Aligning the head and light guide to the retractor

- 1 Once the whitening gel application is complete, raise the back of the chair so the patient is in a slightly reclined position (~45 degree angle) and able to swallow comfortably while maintaining light pressure on the bite block. See the procedure kit instructions for use for whitening gel application instructions.
- 2 Slide the accelerator light into place next to the patient. The boom should be movable, but stable.
- 3 Grasp the accelerator light head with one hand and bring it close to the patient's mouth. Use your other hand to lift or lower the boom as needed to make sure the accelerator light head is aimed directly at the patient's teeth.
- 4 Use both hands to finalize positioning of the accelerator light head so the light guide and IsoPrep retractor (included in procedure kit) slots are aligned.
- 5 Tighten the head and boom knobs.
- 6 Lock the casters in position. While engaged, the light guide and IsoPrep retractor will allow the accelerator light head to follow minor movements of the patient's head to ensure proper alignment.

Running a whitening procedure cycle

Each treatment consists of three 15-minute whitening sessions with a pause after each session to allow the dental professional to remove and replace the whitening gel. Each light guide includes a fourth session for use on patients with shades A3 or darker.

- 1 Ensure the patient is properly prepared according to the procedure kit instructions for use and that all protective barriers are in place.
- 2 Press the 'Intensity' button to toggle between high, medium, and low light intensities until the desired intensity is selected. This setting can be changed at any time during the whitening procedure.

Tip: Use the high setting for maximum whitening and the low setting for maximum heat and sensitivity management.

Note: Lower light intensity may reduce results.

- 3 Press the 'Start' button to switch on the light and activate the session timer.
- 4 Inform the patient that the progress of each session is tracked by four white lights on top of the accelerator light head. Each light indicates one quarter of a session.
- 5 Once the light is on, a countdown is displayed. The accelerator light beeps once when three minutes of the session are remaining and beeps three times in the final three seconds of the 15-minute session. When the session is finished, the light goes out and you hear a long beep.
- 6 If a session must be interrupted, press the 'Pause' button. The remaining session time flashes on the display. To reactivate the session, press either the 'Pause' or 'Start' button.

DO NOT PRESS THE 'STOP' BUTTON AT ANY TIME DURING THIS PROCESS OR AN ENTIRE 15-MINUTE SESSION WILL BE LOST.

- 7 At the end of each 15-minute session, the display indicates the number of remaining sessions left on the light guide. Press 'Start' to activate the light for the next session.
- 8 When all sessions on a light guide are finished, the display reads 'Sessions Complete' for three seconds, followed by 'Guide Status - Replace Guide'. Detach the light guide from the IsoPrep retractor and pull the accelerator light away from the patient.

Tip: After each session, make sure the patient's lips and gums are consistently and thoroughly covered to help prevent any exposure of soft tissue to the LED lights and whitening gel.

Care and maintenance

Clean and disinfect the accelerator light after each whitening session.

- Do not spray or pour liquids directly onto the system.
- Apply cleaners to a soft cloth or paper towel before using.
- Do not use abrasive cleaners or solvents.
- The following cleaners are recommended: Sanicloth, CaviWipes, SciCan, Birex, BioSurf, Cidex and IPA. Follow the instructions of the cleaning products to ensure proper cleaning of the unit.
- For optimum performance, make sure that the output window on the head remains clean and free of particles or streaks.
- The accelerator light has no user-serviceable or user-replaceable parts with the exception of two replaceable fuses that are inside the power pack housing.

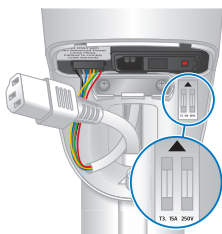
Storage

The boom is designed to fold down to minimize the overall accelerator light footprint for convenient storage or to move the device. To do so, loosen the boom knobs and fold the boom down and in towards the stand.


Fuse replacement

CAUTION: Unplug the accelerator light from the power outlet before replacing fuses.


- 1 Use a Phillips-head screwdriver to remove the screw at the bottom of the power pack housing. Set the screw aside for reassembly.
- 2 Remove the door from the housing.
- 3 Pull the white power cord from its connection and let it hang freely.
- 4 Look for the fuse label near the upper right-hand side to identify the fuse location.

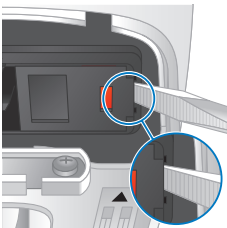- 5 Using a small thin-bladed flathead screwdriver, pry open the right-hand side of the fuse door.

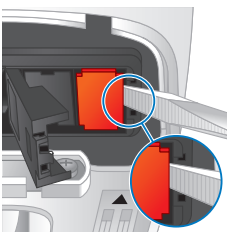
- 6 Using a small thin-bladed flathead screwdriver, pry out the red fuse rack.
- 7 Remove both fuses from the red fuse rack - note their location in the fuse rack.
- 8 Snap in the new fuses at the top of the fuse rack. They should be closer to the side with the tabs (top).

- 9 Reinsert the fuse rack into the fuse slot with the tabs first. Push the fuse rack in until it is securely in place.

Note: You will not be able to close the fuse door if the fuse rack is not inserted correctly.

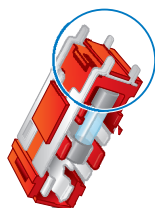- 10 Close the fuse door.
- 11 Reinsert the power cord firmly into the power pack housing.
- 12 Reinsert the screw into the door and tighten with a Phillips-head screwdriver.

Disposal

Your product is designed and manufactured with high quality materials and components, which can be recycled and reused. For recycling information, please contact your local waste management facilities or visit www.recycle.philips.com.

Recycling

- This symbol means that this product shall not be disposed of with normal household waste (2012/19/EU).
- Follow your country’s rules for the separate collection of electrical and electronic products. Correct disposal helps prevent negative consequences for the environment and human health.

Troubleshooting

Problem	Possible cause	Solution
Display reads*SYSTEM ERROR CALL CUSTOMER SERVICE	- One of the cables between power pack and head is not plugged in	- Make sure all 3 connectors are plugged in (head, back of the boom, into the power pack) - If the error code remains, call customer service
Display reads*GUIDE STATUS* IMPROPER GUIDE	- New light guide was first placed on Zoom! Advanced Power or older system	- Replace the light guide with a new light guide that was not first placed on a Zoom! Advanced Power or older system
Display reads*GUIDE STATUS* REPLACE GUIDE before the light guide is empty	- One or more sessions were interrupted by switching off the device - One or more sessions were interrupted by a power outage - One or more sessions were interrupted by light guide disconnection	- If one or more sessions are interrupted by any of the 3 possible causes listed, the lost session cannot be recovered
Display reads*GUIDE STATUS* ATTACH GUIDEWhen a light guide is attached	- The light guide is not fully seated on thehead - The light guide was placed on the head upside down - The pins on the underside of the head are damaged	- Pull the light guide off and reinstall correctly - Place the light guide on the head with the word 'TOP' facing up - Call customer service for a replacement head

Specifications

Philips Zoom! WhiteSpeed accelerator light

Input ratings	100-240 V ~, 0.4~0.2 A, 50-60 Hz
Classification	Class 1 Type B, IPX0, Equipment not suitable for use in the presence of flammable mixtures
Operating mode	Continuous
Operating	Ambient temperature: 10 °C - 40 °C (50 °F - 104 °F) Relative humidity: 5 % to 95 % not including condensation Atmospheric pressure: 70 kPa to 106 kPa Altitude maximum: 3,050 m (10,000 ft)
Transport and storage	Ambient temperature: -40 °C - 70 °C (-40 °F - 158 °F) Relative humidity: 5 % to 95 % not including condensation Atmospheric pressure: 50 kPa to 106 kPa
Light source	LED Array
Bandwidth	400 - 505 nanometers normal Low: 50 mW/cm² ± 15 % Medium: 120 mW/cm² ± 15 % High: 190 mW/cm² ± 15 %

Cooling	Air cooled
Height	136 cm (53.5 in)
Weight	11 kg (24 lbs)

Guidance and manufacturer’s declaration – electromagnetic emissions

The accelerator light is intended for use in the electromagnetic environment specified below. The customer or the user should assure that it is used in such an environment.

Emissions	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The accelerator light uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The wireless transmitter in the foot pedal is battery powered only, and does not connect to public mains.
RF emissions CISPR 11	Class A	The Zoom! WhiteSpeed accelerator light is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes, provided the following warning is heeded: Warning: This equipment/system is intended for use by healthcare professionals only. This equipment/system may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as re-orienting or relocating the Zoom! WhiteSpeed accelerator light or shielding the location.
Power line harmonics IEC/EN 61000-3-2	Class A	
Power line flicker IEC/EN 61000-3-3	Complies	

Explanation of symbols

Type B applied part

Manufacturer

Date of manufacture

Caution

Serial number

Hazardous voltage

Consult instructions for use

Unit must be disposed at an appropriate facility

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All rights reserved
Philips GmbH