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**GROSSBRITANNIEN – GB**

**MECTRON Prophylaxis Powder soft M mint / L lemon Instructions for use**

**To be used with Air–polishing devices.**

**Always refer to the Manual of use of your prophylactic device. Shake the bottle before filling the device with powder. Do not use Prophylaxis Powder soft for applications different from the intended use.**

**Description of the product**
Mectron Prophylaxis Powder soft based on sodium bicarbonate is intended for professional use in supra–gingival treatment only. Mectron Prophylaxis Powder soft is sugar–free. Lemon taste/peppermint taste.

**Intended use**

- Mectron Prophylaxis Powder soft is indicated fort the following types of professional treatment:
- Removal of supra–gingival bacterial plaque;
  - Removal of stains on the tooth surface;
  - Plaque removal prior to fluoride treatment;
  - Plaque and stain removal prior to whitening procedure;
  - Prophylaxis (plaque and stain removal) on patients with ongoing orthodontic treatment;
  - Cleaning prior to bonding ortho brackets;
  - Surface preparation prior to placing composite restorations;
  - Surface preparation prior to bonding, cementation of inlays, crowns and veneers.

**Kontraindikations**

Do not use the device on patients with diseases related to the respiratory system such as chronic bronchitis, asthma, emphysema, etc, as the stream of air and powder could cause breathing difficulties in these patients.

Patients who use contact lenses must remove them before undergoing treatment with air–polishers

Do not point the air/powder/water stream at the eyes. Failure of precaution may cause serious damage. Use of goggles by all concerned (patient, dental hygienist and dentist) is recommended while carrying out the treatments listed above.

The dentist and dental hygienist must wear masks to limit inhalation of the powder.

Do not point the air/powder/water stream at composites, fillings or bridges. Failure to comply could cause damage to these items.

Do not point the air/powder/water stream at soft tissues, nor at the gingival sulcus. Failure to complying with this requirement may cause gingival tissue emphysema (emphysema of the mucous membrane and/or subcutaneous emphysema).

Protect the patient’s lips with vaseline during the treatment.

**Storage**

Store in cool, dry place, in closed containers, far from acids and any heatources.
The optimum temperature and moisture conditions for storage are:
Temperature: –10 °C – 35 °C / 14 °F – 95 °C
Relative humidity: 10 % – 70 %

**Expiry**

Do not use the product after the expiry date stated on the container.

**Classification**

Device compliant with Regulation EU MDR 2017/745: Class I

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