FIG

TempoCem

Inhalt | Contents

Gebrauchsinformation – DEUTSCH	4
Instructions for use – ENGLISH	12
Mode d'emploi – FRANÇAIS	20
Istruzioni per l'uso – ITALIANO	28
Instrucciones de uso – ESPAÑOL	36
Instruções de utilização – PORTUGUÊS	44
Gebruiksaanwijzing – NEDERLANDS	52
Brugsanvisning – DANSK	60
Bruksanvisning – SVENSKA	68
Instrukcja użycia – POLSKI	76
Инструкция по применению – РУССКИЙ	84

Produktbeschreibung

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Product description

TempoCem and TempoCem Soft are automix cements based on zinc oxide/eugenol. TempoCem flows easily, allowing correct placement of the temporary restoration. It is resistant to saliva and stable in the mouth. It withstands the stress of mastication sufficiently, but allows easy removal of the temporary restoration. TempoCem Soft cures more slowly than TempoCem and has a lower end hardness. This allows easier removal of temporary crowns and bridges with very high friction.

Indications

TempoCem

- Temporary cementation of provisional and permanent crowns and bridges, inlays and onlays.
- Temporary cementation of implant-borne dental prosthesis.

TempoCem Soft

- Temporary cementation of provisional and permanent crowns and bridges, inlays and onlays.

Contra-indications

Do not use the material in cases of allergies to any of the substances contained or, as can occur in rare cases, in the event of contact allergy.

Basic safety instructions

- Only for dental use!
- Keep out of reach of children!
- Avoid contact with skin and eyes! In the event of accidental contact, rinse thoroughly with plenty of water immediately and consult a physician if necessary.
- Contains rosin.

Application System

- Automix cartridge: see “Using the Automix cartridge” on page 15
- Smartmix injector: see “Using the Smartmix Injector” on page 16

Please note: When using a cartridge or injector for the first time, squeeze out a pea-sized amount of material and discard it.

Timing

TempoCem

≈1:00 min	Working time
4:00 min	Curing time in the mouth

TempoCem Soft

≈02:30 min	Working time
08:00 min	Curing time in the mouth

Please note: The times given apply at a room temperature of 23 °C and a normal relative air humidity of 50 %. Higher temperatures shorten, lower temperatures increase these times.

Advice on application

Cements containing eugenol can inhibit the curing of composites, resin-based attachment cements and bonds.

- ▶ Check whether it might be more beneficial to use the eugenol-free attachment cement TempoCemNE (DMG).
- ▶ Make sure that any residual cement is completely removed from the impression or stump or, if necessary, from the implant post, following removal of the temporary restoration.

Recommended Application

Temporary cementation – General

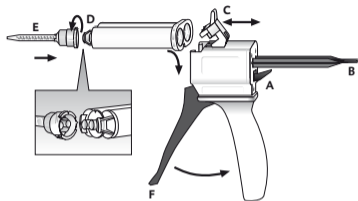
1. Apply TempoCem/TempoCem Soft to the dry temporary restoration.
2. Insert the temporary restoration within 1:00 min or 2:30 min for TempoCem-Soft after the start of mixing and attach to the dry, prepared tooth using slight pressure.

3. After curing, remove excess cement with a probe.
It is easy to remove excess cement in large chunks.
4. Remove interdental excess by flossing carefully.
The light color of the material makes it easy to distinguish from teeth.

Temporary cementation of implant-bourne dental prosthesis.

1. Apply TempoCem to the superstructure.
2. Insert the superstructure within 1:00 min of start of mixing and attach to the dry implant post using slight pressure.
3. After curing, remove excess cement with an implant scaler. Follow the manufacturer's instructions for the implant precisely!
4. Remove interdental excess by flossing carefully.
The light color of the material makes it easy to distinguish from teeth.

Using the Automix cartridge

Insert cartridge

1. Push up the lever [A] at the back of the Automix dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C], insert the cartridge and lock it using the plastic lever [C].

Please note: Make sure that the notches on the cartridge line up with those on the Automix dispenser.

Attach mixing tip

1. Remove the cartridge cap [D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

Please note: Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.

Apply material

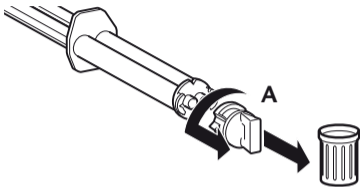
- ▶ When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [F] of the Automix dispenser and it is then applied directly.

Please note: After the application, leave the used mixing tip on the cartridge to cap unused material!


Remove cartridge

1. Push up the lever [A] at the back of the Automix dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

Using the Smartmix Injector

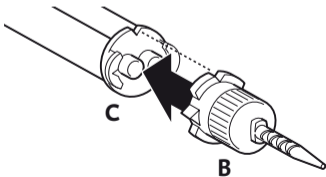
1. Before putting on the mixing cannula, remove the cap [A] or used mixing tip after twisting it 90° counterclockwise, and then discard it.

2. Attach a new mixing tip [B].

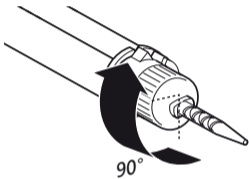Please note: Make sure that the notches on the mixing tip line up with those on the Smartmix injector [C].

3. Turn the mixing tip clockwise 90° to lock it.
4. The material is mixed in the cannula when it is expressed and it can be applied directly.

Please note: When using an injector for the first time, squeeze out a pea-sized amount of material and discard it!

Leave the used mixing cannula on the Smartmix injector after use. It serves as a cap!

Disinfection of the Automix dispenser

The Automix dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute). The Automix dispenser is also autoclavable.

Side effects

Certain studies hypothesise that there is a correlation between eugenol and pulp damage.

- ▶ Therefore, check the necessity of a pulp protection in each individual case.

Composition

- Catalyst paste: Natural resins, eugenol, additives
- Base paste: Zinc oxide, paraffin, additives

Classification

ISO 3107:1988 (because of the automatic mixing, the curing time is shorter than usual)

Storage and shelf life

- ▶ Store at room temperature (15 to 25 °C/59 to 77 °F) in a dry place!
- ▶ Do not store in the refrigerator!
- ▶ Protect against exposure to direct sunlight!
- ▶ After the application, leave the used mixing tip on the Automix cartridge to cap unused material!
- ▶ Leave the used mixing cannula on the Smartmix injector after use. It serves as a cap!
- ▶ Do not use after the expiration date!

Available product sizes

Refill packs

1 cartridge @ 63g paste, 35 Automix tips	
TempoCem	REF 110247*
TempoCem Soft	REF 110265*

2 Smartmix double syringes containing 11 g paste , 20 Smartmix-Tips	
TempoCem	REF 212129*

*Not available in the United States

Accessories

1 Automix-Dispenser Type 25 1:1	REF 110253
50 Automix-Tips Blue 1:1	REF 110266
50 Smartmix-Tips Short	REF 212040

Only available in the United States

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DMG Chemisch-Pharmazeutische Fabrik GmbH