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More information available at:
www.proclinic-products.com

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PROCLINIC EXPERT HOME USE WHITENING KIT

Contents:

- 10 syringes with a 16% carbamide peroxide concentration

Avoid contact with eyes. In case of contact with eyes, rinse immediately with plenty of water. 16% carbamide peroxide ~ 4.44% H₂O₂ – Not intended for use by individuals under 18 years old. – Sale exclusively for odontologists. – During each cycle, the first use will be restricted for odontologists or will be used under a direct supervision providing that a similar safety is assured. Subsequently, it will be supplied to the patient to complete the cycle.

Instructions:

- Place the product in the frontal face of the tray (the one that will be in contact with the surface which is going to be bleached). Repeat the process in the upper tray and the lower tray.
- Place the trays over the teeth without biting excessively. Avoid the ingestion of the product.
- After a period of 40-50 minutes, remove the trays and rinse the mouth with plenty of water.
- Repeat the process daily until finishing the product. Apply just before going to sleep.

Recommendations for the patient:

1. Avoid food with natural or chemical coloring.
2. Avoid drinking beverages like juices, coffee, tea, chocolate, wine, infusion, alcohol, etc.
3. Avoid orange, red or green food.
4. Do not use colored mouthwash or toothpaste as they contain artificial coloring and extracts.
5. Avoid smoking at least during the first days of the treatment and permanently to obtain long-lasting results.

6. Maintain a good oral hygiene.
7. Follow a "white diet" at least during the following week after the treatment. It will consist of the ingestion of beverages and food without coloring, such as dairy products, rice, etc.
8. Avoid acid flavors which irritate the gums.
9. In case of sensitivity, rinse the mouth with plenty of warm water.

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