# Gingival resin barrier

## Instructions for use

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### EN Instructions for use.

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### GINGIVALE KUNSTSTOFFBARRIERE

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### GINGIVAL RESIN BARRIER

#### Description:

White light-cured protective resin. When cured with the light, it creates a semi-plastic barrier which protects the gingival tissues during the treatment.

#### Instructions for use:

1. Apply the cheek retractor so the patient feels comfortable. Cotton rolls can also be used.
2. Remove the black lid of the syringe and place the application tip correctly.
3. Clean and dry the teeth, specially the gingival tissue around the teeth where the whitening product will be used.
4. Press the end of the syringe over a tissue paper until the content comes out.
5. Apply the product starting at the end of the upper arch creating a 6 mm thickness barrier (1 mm of the barrier should be over the enamel). In case of an existing fissure, cover it well. Ensure that all necks are correctly covered. Always protect one more tooth than the number of teeth

to be bleached.

6. Then, cure the resin by sections using a curing light. It should be cured during 5-10 seconds.
7. Repeat the process in the lower arch.
8. Remove the tip from the syringe and place the lid again. Each syringe has enough product for more than one patient.
9. After the treatment, clean and rinse well the teeth.
10. Remove the protective resin sliding the end of a probe between the gums and the resin. Check that not product is left in the interdental area. Use a probe or floss and rinse again.
11. Keep the product in a fresh, dry and dark place.

#### Caution:

This product must be used just in patients with a good dental and gingival health. Do not use this product in patients who may be sensitive or allergic to resin. It is recommended to try it firstly in a small area.

#### Ingredients:

Dangerous component: OSHA PELACGH TLV%  
Monomers of aromatic aliphatic dimethylacrylate

#### Emergency:

In case of contact with the eyes, rinse with plenty of water.

In case of contact with the skin, rinse with water and soap.

In case of accidental ingestion, consult a doctor.

This document is provided for informational purposes only.

Proclinc will not be liable for any incorrect use of the products or any consequences arising from an inappropriate use.

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