IT

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EN

IRRIGATION SET FOR MACHINERY

Irrigation set for machinery driven irrigation system to be used with dispensing liquid coolants in oral surgery. Non toxic. Apyrogenic. Free of latex components and phthalates.

Instructions for use:

- 1 Remove the cap form the spike.
- 2 Insert the spike into the bag or connect the irrigation system throught the luer lock. The spike with air filter "closed and open" allows to regulate the flow rate.
- 3 Where foreseen,use roller/clip (3A) to adjust flow rate.
- 4 Insert the irrigation set in the peristaltic pump, check the flow direction and use the appropriate connection at the end of silicone tube.
- 5 Connect the final part of the irrigation set to the contra-angle.
- 6 Multimedical irrigations sets offer the following options:
a. single way internal irrigation
b. single way external irrigation
- 7 Fix the irrigation set to the micro motor using the clips included.

The irrigation set sterilized with ethylene oxide is for single use only. Do not resterilize. Do not re-use. Keep in clean and dry place, protect from moisture, direct rays of the sun and source of heat. Discard after

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